

PRODUCT SIZING

Ref No. 2415-XX

Measure from center of shoulder blades around shoulder, back to center.

Size	XX	Min	Max	Min	Max
PED	05	8"	14"	20cm	36cm
XS	10	14"	20"	36cm	51cm
S	20	20"	26"	51cm	66cm
M	30	26"	32"	66cm	81cm
L	40	32"	38"	81cm	97cm
XL	50	38"	44"	97cm	112cm

OPTIONAL COLD THERAPY ACCESSORIES

Product Name	Size	Ref No.
FirstIce® Direct Stick™ Cold Therapy Pack	6 x 7"	100-0607

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

© 2022 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *282463*



REV A 8/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



CLAVICLE SUPPORT

INDICATIONS


Patient Population: Adults and Children with identified indications

- Clavicle fractures

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.



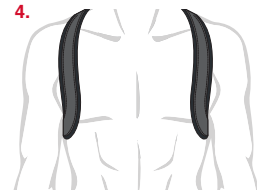

1. Place straps over shoulders and adjust D-Ring height for support and comfort as needed.
2. Secure strap for unaffected side by sliding strap through D-Ring. Repeat for affected side.
3. Adjust to ensure the shoulders are in neutral anatomical position.
4. Secure straps simultaneously. Patient should be in a resting position with shoulders back.
WARNING: Do not hyperextend.

Optional Cold Therapy Accessory

5. Apply optional Direct Stick™ Cold Therapy on compatible straps as desired.

*Scan for application
and accessories.*

