

## PRODUCT SIZING

Ref No. 403-XX

Measure circumference of leg 6" above mid-patella with leg fully extended

Size	XX	Min		Max	
XS	10	13"	33cm	16"	41cm
S	20	16"	41cm	19"	48cm
M	30	19"	48cm	21"	53cm
L	40	21"	53cm	24"	61cm
XL	50	24"	61cm	27"	69cm
XXL	60	27"	69cm	30"	76cm
XXXL	70	30"	76cm	33"	84cm

**Note:** Silicon Butress may require increasing the brace size

# COMPRESSION KNEE SUPPORT

## INDICATIONS:

**Patient Population: Adults and Children with identified indications**

- Knee Ligament Injuries
- Meniscus Injuries
- Knee Sprains and Strains

**WARNING:** This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

**CARE:** Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2023 Professional Products, Inc.  
www.ezywrap.com  
Made in the USA

**R&ID# \*286472\***




REV A 3/23

54 Hugh Adams Rd., Defuniak Springs, FL 32435  
National: (800) 234-9004  
International: (850) 892-5731  
Fax: (800) 274-9006




a.

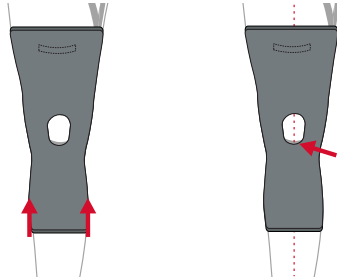

a. Pull brace up over knee and center on patella

b. Center brace on back of leg and pull up over knee

b.

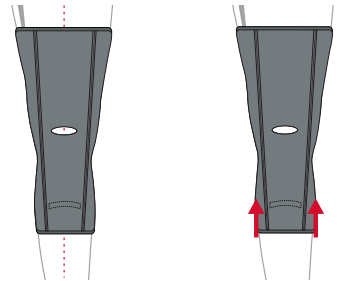

*Scan for application  
and accessories.*

