

PRODUCT SIZING

Ref No. 450-XX

Measure circumference of leg 6" above mid-patella with leg fully extended

Size	XX	Min		Max	
XS	10	13"	33cm	16"	41cm
S	20	16"	41cm	19"	48cm
M	30	19"	48cm	21"	53cm
L	40	21"	53cm	24"	61cm
XL	50	24"	61cm	27"	69cm
XXL	60	27"	69cm	30"	76cm
XXXL	70	30"	76cm	33"	84cm

INFERIOR PATELLA STABILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Chondromalacia
- Patellofemoral Malalignment
- Acute Patella Injuries and Chronic Conditions

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2023 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *286994*




REV A 3/23


54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006




1.

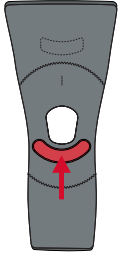

2.

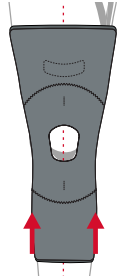

3.

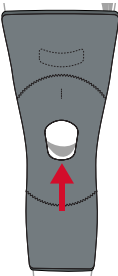

1. Place silicone buttress to support bottom of patella
2. Center brace and pull up over the knee
3. Refine silicone buttress alignment with bottom of the patella

*Scan for application
and accessories.*

