

PRODUCT SIZING

Ref No. 535-XX

Measure the circumference of leg 6" above mid-patella with leg fully extended

Size	XX	Min		Max	
XS	10	12"	31cm	16"	41cm
S	20	15"	39cm	18"	46cm
M	30	17"	44cm	20"	51cm
L	40	20"	51cm	23"	59cm
XL	50	23"	59cm	26"	66cm
XXL	60	26"	66cm	29"	74cm
XXXL	70	29"	74cm	32"	81cm

HINGED LATERAL-J PATELLA STABILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Patella Subluxation
- Chondromalacia
- Lateral Patellofemoral Malalignment
- Acute Patella Injuries and Chronic Conditions

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2023 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *285845*




REV B 5/23


54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



1.

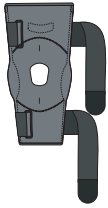


2.

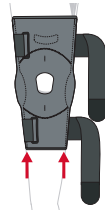

1. Loosen all straps

2. Center brace and pull up over the knee


3.

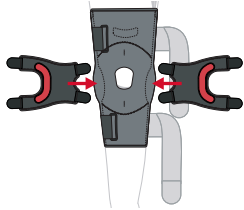

3. Determine direction of support desired for patella stabilization

4. Place silicone buttress and apply the patella strap laterally

4.

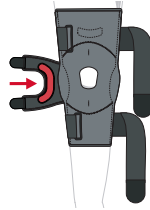

5. Fix removable straps in the desired location and secure the upper strap, then the lower strap through the links


5.



Customization

6. Remove hinges from pocket and shape as desired

6.

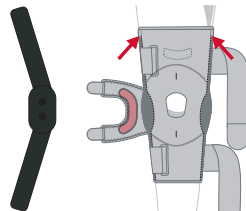

*Scan for application
and accessories.*

