

PRODUCT SIZING

Ref No. 7370-XXX

Measure circumference of relaxed mid-bicep

Size	Ref No.	Max	Max
Advanced Elbow Compression Wrap	7370-XXX	16"	41cm

Bundle Options	Pack Size	Packs	XXX
Wrap Only	-	0	000

First Ice® System (Wrap, Packs, & Tote-Bag)	6" x 12"	2	102
First Gel® System (Wrap, Packs, & Tote-Bag)	6" x 12"	2	202

Note: All options are available in single and 12-pack case quantities, wraps only in 48 packs

FIRST ICE® & FIRST GEL® (PACK ONLY)

Integrated Hot or Cold Therapy

Product Name	Pack Size	Storage	Packs	Ref No.
First Ice® Direct Stick™ Cold Therapy Packs	6" x 12"	Zip-Lock	2	130-0612
		Direct Freeze Package (DFP)	4	144-0612
First Gel® Direct Stick™ Hot / Cold Therapy Packs	6" x 12"	Zip-Lock	4	230-0612

Note: DFP decreases pack freeze time and facilitates freezer storage

© 2023 Professional Products, Inc.
www.ezywrap.com
Made in the USA

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006

R&ID# *286925*



REV A 5/23



ADVANCED ELBOW THERAPY SYSTEM

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-Operative pain and inflammation
- Post-Injury pain and inflammation
- Rehabilitation pain and inflammation
- Acute elbow injuries and chronic elbow conditions


WARNING: This product is intended for single-patient use at the direction of a medical professional and does not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CAUTION: Review all instructions included with Hot or Cold Therapy, First Ice® or First Gel® before use. Hot or Cold Therapy is intended for external use only.


CARE: Hand-wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.




1.

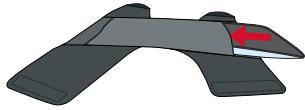


1a.

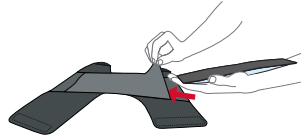


2.

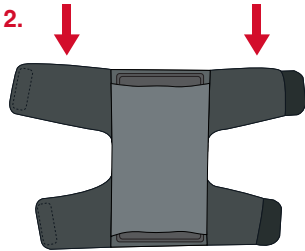

3.

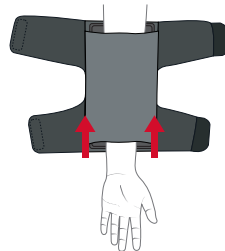

4.

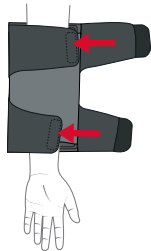

5.



1. Insert Direct Stick™ pack inside the sleeve with the black barrier facing the skin
 - a. Guide the pack into the wrap using your hand and covering the hooks
2. Orient wrap so the wide strap is on top
3. Insert hand into sleeve and pull up arm to desired position on elbow
4. Close top and bottom strap with quick-assist hook
5. Readjust straps for desired compression

Note: This elbow system is intended to be applied over any post-operative dressing when present.