

## PRODUCT SIZING

Ref No. 803-XX

Measure using shoe size

Size	XX	US Men	US Women	EU Men	EU Women
XXS	5	5 - 6	6.5 - 7.5	37 - 40	37 - 38
XS	10	6 - 7.5	7.5 - 9.0	40 - 41	38 - 40
S	20	7.5 - 9	9.0 - 10.5	41 - 43	40 - 41
M	30	9 - 11	10.5 - 12.5	43 - 45	41 - 43
L	40	11 - 13	12.5 - 14.5	45 - 47	43 - 45
XL	50	13 - 15	14.5 - 16.5	47 - 49	45 - 47
XXL	60	15+	16.5+	49+	47+

## OPTIONAL ACCESSORIES

Product Name	Size	Ref No.
Removable Plastic Stays	Universal	150-00

# SPEED LACER ANKLE ORTHOSIS

## INDICATIONS:

**Patient Population: Adults and Children with identified indications**

- Ankle sprains
- Strains or post-injury
- Post-operatively

**WARNING:** This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

**CARE:** Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2023 Professional Products, Inc.  
www.ezywrap.com  
Made in the USA

R&ID# \*282555\*



REV B 5/23

54 Hugh Adams Rd., Defuniak Springs, FL 32435  
National: (800) 234-9004  
International: (850) 892-5731  
Fax: (800) 274-9006



## INSTRUCTIONS FOR USE

US & Foreign Patents Pending

Model No. 803


1.



2.



3.

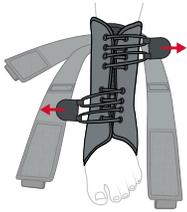

4.



5.



1a.



1. Loosen all laces prior to application.
2. Insert foot into orthosis and heel into heel relief.
3. Secure speed lacer tabs to brace body.
4. Apply the figure 8 strap on the side requiring support and place underfoot hook to lock in targeted control
5. Apply top band for desired support.
6. Readjust as necessary.

### Optional Accessories

- 1a. Insert plastic stay for additional stability.

*Scan for application  
and accessories.*

