

## PRODUCT SIZING

Ref No. 939-XX

Measure circumference of thumb at knuckle and wrist at narrowest.

Size	XX	Thumb at Knuckle		Wrist at Narrowest	
XS	10	2.1 – 2.5"	5 – 6cm	4.5 – 5.0"	11 – 13cm
REG	35	2.5 – 3.4"	6 – 8cm	5.0 – 8.5"	13 – 22cm
XL	50	3.4 – 3.6"	8 – 9cm	5.0 – 8.5"	13 – 22cm

# WAHOO THUMB ORTHOSIS

## INDICATIONS:

### Patient Population: Adults and Children with identified indications

- CMC Stabilization
- Thumb Injuries
- Arthritis

**WARNING:** This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

**CARE:** Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2022 Professional Products, Inc.  
www.ezywrap.com  
Made in the USA




R&ID# \*283032\*



REV A 9/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435  
National: (800) 234-9004  
International: (850) 892-5731  
Fax: (800) 274-9006



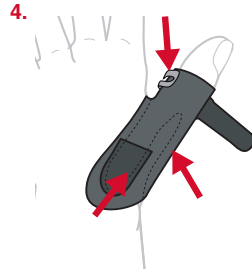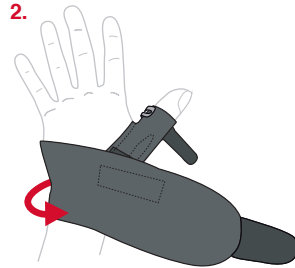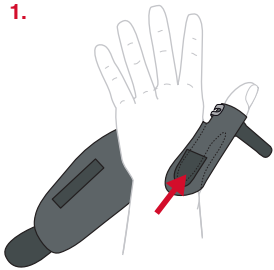

1. Insert thumb into thumb opening.
2. Secure wrist strap.
3. Secure thumb strap.

### Customization

4. Remove, stack or shape stays for desired function.

*Scan for application and accessories.*

