EZY[®] PRO BRACE

CE

Proudly

MADE IN USA

Thoroughly read the following information prior to application.
Product function requires proper application.

WARNING: Because of the nature of the sport, the possibility of injury is inherent in gymnastics. The equipment is intended for use only by properly trained and qualified participants under supervised conditions. When you use our products, know your limitations and the limitations of your equipment. Keep your equipment in good condition by inspecting for loose fittings or damage and test for stability before each use.

- One of the unique features of this brace consists of its palmar pad (Figure 1). This pad is positioned in the lower half of your palm and serves three purposes:
 1. Having a pad in the palm of your hand during a support position will raise the wrist a small distance from the contact surface, enough to decrease the angle of extension at the wrist joint. This "opening up" of the wrist extension angle relieves the pressures inside the wrist joint during hyperextension.
 2. The palmar pad is wedge-shaped (thicker on the radial side than on the ulnar side) in order to compensate for the imbalance between the radius and ulna. This condition is called positive ulnar variance. The medial wedging distributes the contact forces inside the wrist joint more evenly.
 3. The palmar pad can also serve as a cushion acting to absorb pressure on events where high impact forces on the wrist are common (e.g. vault, floor exercise, pommel horse ...)

The brace is designed such that the palmar pad can be detached from the dorsal support by its Velcro® attachment. (Figure 3) This allows the gymnast to use the brace in a variety of activities:

With palmar pad: on events where the surface is flat, e.g. floor exercise, vault, mushroom, ends of pommel horse ...

Without palmar pad: (Figure 2) on apparatus where the palm of the hand needs to be exposed, but where dorsal support is still desired, e.g. p-bars, pommel horse, balance beam.

- **Apply Without Palmar pad:** Loop Velcro® closure through metal buckle and secure. (Figure 4 & 5).
- **Apply With Palmar pad:** Align dorsal (Figure 6) and palmar pads central to wrist. Attach Velcro® to palmar Velcro® pad. Loop Velcro® closure through metal buckle and secure. Secure thumb and fifth metacarpal straps to dorsal pad with elastic Velcro® tabs. (Figure 7)

CARE INSTRUCTIONS

INDICATIONS

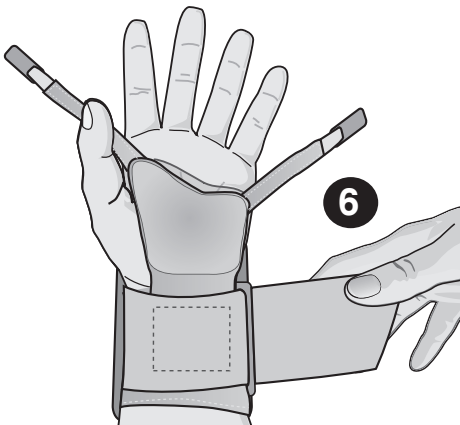
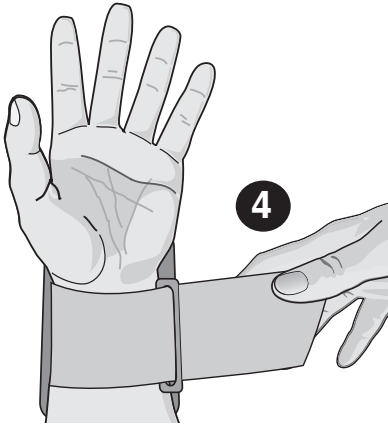
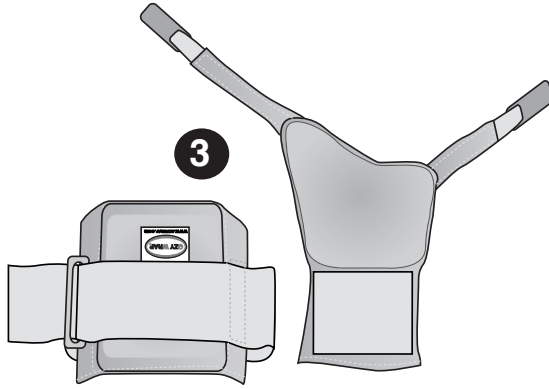
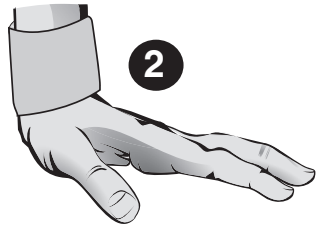
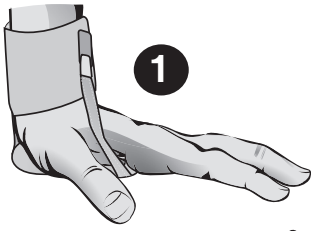
To limit the training lost due to wrist injuries, and to increase the quantity and quality of gymnastics.

CARE INSTRUCTIONS

Hand wash using cold water and mild detergent. Do not bleach. Rinse thoroughly, removing all soap residue to avoid skin irritation. Allow to AIR DRY. Do not twist or wring. Reshape, dry flat

WARNING: INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.

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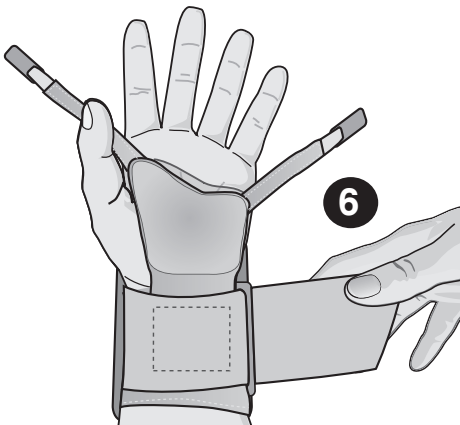
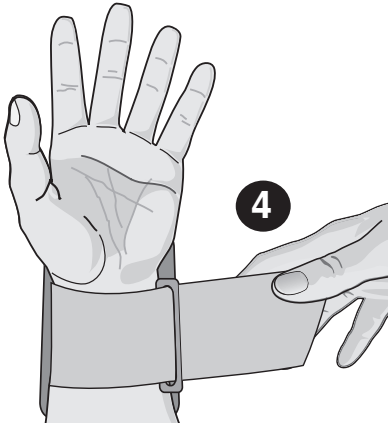
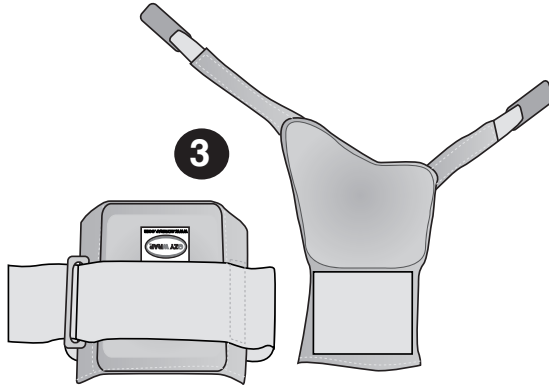
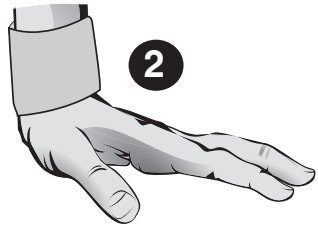
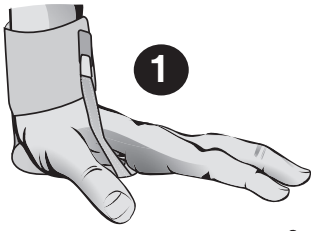
EZY PRO BRACE™ GYMNAST WRIST SUPPORTS (PAIR)


Wrist pain from repetitive dorsiflexion and compression during pommel horse exercises is common among gymnasts. The lateral wedge of the palmar pad of the brace will compensate for positive ulnar variance, distributing contact forces more evenly across the radioulnar carpal joint. Research was shown that wrist joint dorsiflexion angle was significantly reduced by the Ezy Pro Brace™ with and without the palmar pad. However, ulnocarpal joint intraarticular peak pressure was reduced only by the brace with pad. Prevention of pathologic wrist changes requires intervention in pressure attenuation, which was achieved with the Ezy Pro Brace™ with palmar pad. This brace may decrease the cumulative effects of repetitive stress of pommel horse exercise training.*

Helps relieve and prevent wrist pain by providing strong support in wrist extension and hyper-extension. With palmar pad: on events where the surface is flat, e.g. floor exercise, vault, mushroom, ends of pommel horse ect. Without palmar pad: on apparatus where the palm of the hand needs to be exposed, but where dorsal support is still desired, e.g. p-bars, pommel horse, balance beam.


- Materials: Breathable padded material, Beige
- Patented Pad design is detachable enabling the palm of the hand to be exposed during strenuous Gymnastic activities.
- U.S. Patent No. 4,881,533.
- Velcro® closure

| Measure the circumference of the wrist joint & the span of the palm | | | | |
|---|-----------|------------|---------------------------|------------|
| Size | Palm Span | | Wrist Joint Circumference | |
| XSmall | 2½ - 3" | 6 - 8 cm | 4¾ - 6" | 12 - 15 cm |
| Small | 3 - 3½" | 8 - 9 cm | 6 - 6½" | 15 - 17 cm |
| Medium | 3½ - 4" | 9 - 10 cm | 6½ - 7" | 17 - 18 cm |
| Large | 4 - 4½" | 10 - 11 cm | 7 - 7½" | 18 - 19 cm |

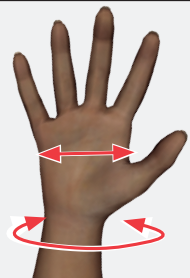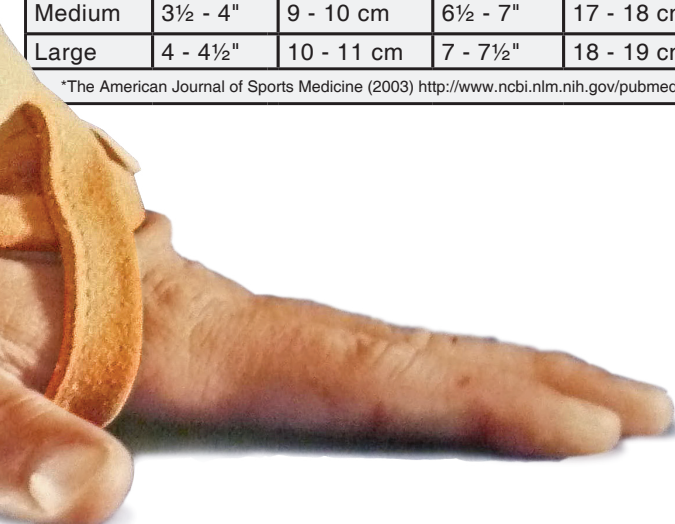*The American Journal of Sports Medicine (2003) <http://www.ncbi.nlm.nih.gov/pubmed/12975195#>

WWW.EZYWRAP.COM

**PROFESSIONAL PRODUCTS, INC.
54 HUGH ADAMS ROAD
DEFUNIAK SPRINGS, FL 32435**

**USA & CANADA (800) 234-9004
INTERNATIONAL (850) 892-5731
FAX (800) 274-9006**

COPYRIGHT © 2018 PROFESSIONAL PRODUCTS, INC.

R& ID# *277040*

B / B