THE BOXER ORTHOSIS

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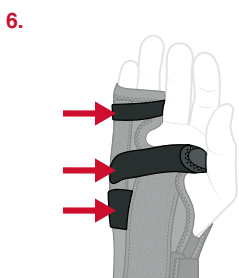
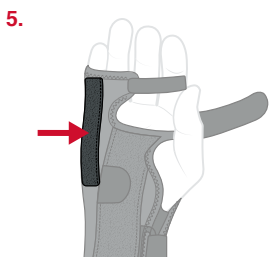
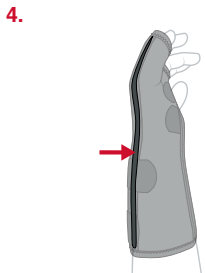
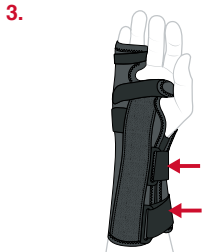
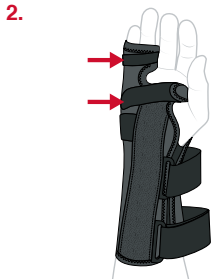
ENGLISH

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Fractures of the 4th and 5th metacarpals
- Fractures of the 2nd and 3rd metacarpals

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

INSTRUCTIONS FOR USE

1. Loosen straps and apply wrist into palmar pocket guiding affected fingers into metacarpal pocket of the brace.
2. Align palmar stay to center of wrist and secure finger then palmar straps.
3. Secure wrist strap then forearm straps.

Customization

4. Adjust dorsal metacarpal stay to desired angle and place on center of affected fingers.
5. Install optional palmar finger stay for additional support.
6. Readjust stay straps as desired for support.

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© 2024 Professional Products, Inc.