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
CRYOTHERAPY

featuring **FIRSTICE** technology

Comprese FirstICE® De froid

FirstICE® Terapia Fría

FirstICE® Kältetherapie

<p>OUTPATIENT / SPORT AND INDUSTRIAL USE: Follow the instructions of your healthcare provider, sport or industrial device instructions.</p> <p>WARNING: This product is only to be used as directed. Observe skin condition during use. If redness, pain or swelling develops or a change in skin sensitivity occurs, discontinue use immediately. If irritation, redness, pain or swelling occurs, discontinue use immediately. If any and/or muscular reaction might be associated with increased skin sensitivity and irritation. Discontinue therapy until the exact cause of the problem is determined.</p> <p>CAUTION: FIRSTICE will not burn or injure skin tissue, but direct application of a cold pack with increased skin sensitivity to the skin may cause discomfort. The recommended application is over a thin dressing or applied to a FIRSTICE device. FIRSTICE is non-toxic, and is not intended for internal use.</p> <p>Made in U.S.A. (800) 234-9004 www.ezywrap.com/firstice</p>	 CRYOTHERAPY featuring FIRSTICE technology	  INPATIENT USE: 1. Place in freezer at least (12) hours prior in initial application. 2. Apply to the affected area. 3. Replace with frozen pack/s per physician's protocol or after 3 - 4 hours. 4. Return used packs to freezer. 5. Repeat per physician's protocol.	<p>OUTPATIENT / SPORT AND INDUSTRIAL USE: Follow the instructions of your healthcare provider, sport or industrial device instructions.</p> <p>WARNING: This product is only to be used as directed. Observe skin condition during use. If redness, pain or swelling develops or a change in skin sensitivity occurs, discontinue use immediately. If irritation, redness, pain or swelling occurs, discontinue use immediately. If any and/or muscular reaction might be associated with increased skin sensitivity and irritation. Discontinue therapy until the exact cause of the problem is determined.</p> <p>CAUTION: FIRSTICE will not burn or injure skin tissue, but direct application of a cold pack with increased skin sensitivity to the skin may cause discomfort. The recommended application is over a thin dressing or applied to a FIRSTICE device. FIRSTICE is non-toxic, and is not intended for internal use.</p> <p>Made in U.S.A. (800) 234-9004 www.ezywrap.com/firstice</p>	 CRYOTHERAPY featuring FIRSTICE technology	  INPATIENT USE: 1. Place in freezer at least (12) hours prior in initial application. 2. Apply to the affected area. 3. Replace with frozen pack/s per physician's protocol or after 3 - 4 hours. 4. Return used packs to freezer. 5. Repeat per physician's protocol.	<p>OUTPATIENT / SPORT AND INDUSTRIAL USE: Follow the instructions of your healthcare provider, sport or industrial device instructions.</p> <p>WARNING: This product is only to be used as directed. Observe skin condition during use. If redness, pain or swelling develops or a change in skin sensitivity occurs, discontinue use immediately. If irritation, redness, pain or swelling occurs, discontinue use immediately. If any and/or muscular reaction might be associated with increased skin sensitivity and irritation. Discontinue therapy until the exact cause of the problem is determined.</p> <p>CAUTION: FIRSTICE will not burn or injure skin tissue, but direct application of a cold pack with increased skin sensitivity to the skin may cause discomfort. The recommended application is over a thin dressing or applied to a FIRSTICE device. FIRSTICE is non-toxic, and is not intended for internal use.</p>
---	--	---	---	--	---	---

Proudly

MADE IN USA

INDICATIONS:

Inflammation / pain following injury or operatio.

Patient Population: Adults and children with identified indications.

FIRSTICE® COLD THERAPY PACK INPATIENT INSTRUCTIONS

Place in freezer at least (4) hours prior to initial application.

- To allow for proper freezing, do not stack.
- Place packs (Fig. 1) in Direct Freeze Package (DFP) (Fig. 2) to avoid cross contamination and improve freezing times.
- Take FirstICE® out of freezer.
- Gently massage packs to loosen granules. (Fig. 3)

Load pack/s into or onto FirstICE® device and apply to desired area.

- Attach Velcro® side of pack to support. (Fig. 4)
- Apply soft side of FirstICE® Direct Stick™ (DS™) pack to affected area. (Fig. 5)
- White hole allows air to release and should not be obstructed.

Replace with frozen pack/s per physician's protocol or after 2 - 4 hours.

Return used pack/s to freezer.

- With Direct Freeze Package (DFP), Re-Freeze 2 to 3 hours
- Without Direct Freeze Package (DFP) DO NOT STACK, Re-Freeze 3 to 4 hours.


Repeat per physician's protocol. FirstICE® is safe to place directly on skin after sutures are removed and incision is healed. Add a barrier at physician's discretion.

OUTPATIENT / SPORT AND INDUSTRIAL USE:


Follow the instructions of your healthcare provider, sport or industrial device instructions.


1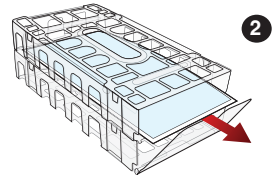


2

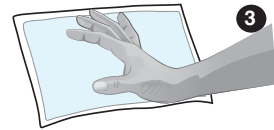3

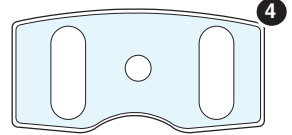4

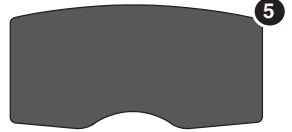5

Warning: This product is intended for single patient use at the direction of a medical professional and does not prevent or reduce the risk of injury. This product is not microwave safe. Do not directly heat. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

Caution: This product is non-toxic. Do not apply directly to the skin without appropriate barrier. Dispose of product if the product is punctured or torn.

Storage: This product is sensitive to direct light and heat. Store in a cool, dry place with the white valve unobstructed and facing up. Do Not Stack. For best results, store in the Direct Freeze Package (DFP) to allow for crystallization.

Care: Hand wash using standard decontamination protocols of diluted disinfectants and rinse thoroughly. Reshape flat and allow to air / towel dry only. Do not twist.

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Copyright © 2022 Professional Products, Inc.
Professional Products, Inc.
MDSS GmbH