



Design & Manufacturing from Norway

Øvre Langelandsvei 19//2213 Kongsvinger//Norway
+47 62 81 48 98 // sale@sagadental.no // www.sagadental.no

INSTRUCTION FOR USE

1 GENERAL INFO

DEVICE NAME

Rhodium Coated Mirrors

INTENDED USE

Intended for clinical photography in dentistry

2 STERILIZATION ROUTINE

- Can be chemically sterilized in Cetylcide II, after that wash under running water prior to use.
- Best mode of treating the mirror is to wash them in the Disinfector at 80°.
- Then sterilize them in the dry heat sterilizer at 121°C
- Black spot due to the water drying on the surface is to be cleaned with a wet sponge.

NOTE

Do not autoclave rhodium coated mirror in the autoclave

Many of the old sterilizer, temperature goes much higher than 131°C, if the thermostat is not regularly checked for temperature.

3 STORAGE & SHELF LIFE

Store preferably in lint free cloth or similar to avoid scratching.

4 MATERIAL DATA

Please refer separate Material Data Sheet.

5 DISPOSAL

Saga Dental Supply AS suggests the product to be disposed of according to country or state regulation.

6 FEEDBACK

Contact Saga Dental Supply AS in cases of warranty issues, customer feedback or adverse incidents.

7 CONTACT INFO

Our contact info is found in the letterhead of the IFU, our invoice, our products and website.

Saga Dental Supply AS
Øvre Langelandsvei 19
2213 Kongsvinger
Norway

Phone: +47 62 81 48 98
Email: sale@sagadental.no
Web: www.sagadental.no