

**Product reference :
4PFM40 Schoeller Masks Medical Face Mask**



CE These face masks bear CE marking to demonstrate compliance with MDD 93/42/EEC Annex I Health & Safety requirements -

EN 14683 : 2019 Medical Face masks

Bacteria filtration efficiency **BFE ≥ 95%**

Microbial cleanliness ≤ 30 **Breathability < 40**

CLASS 1
MEDICAL DEVICE

TYPE I

Suitable for use in situations where moderate to light amounts of fluid or aerosols are produced. **Please note this product is not classified as PPE**


Splash Resistance (not tested)
- NOT APPLICABLE TO TYPE 1

These medical face masks are intended to be worn by patients and other persons to reduce the risk of spreading of infections, particularly in epidemic or pandemic situations

- NON STERILE
- SINGLE USE ONLY

4 layer structural protection - comfortable & breathable - shaped nose clip - ear loops - fitted to face

Instruction for use:

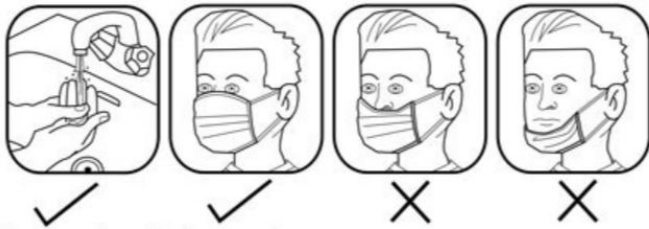

How to put on the face mask -

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Remove mask from package, take out the mask, hold by the two ends/ ear loops with the nose bridge at the top - white side should be facing towards the inside to touch the face, blue side should be facing to the outside.
- Cover the nose, mouth and jaw with the mask and loop the ear loops over each ear.
- Adjust the position of the mask, mould or pinch the stiff edge over the bridge of the nose for a comfortable fit.
- The mask should cover the nose, mouth and chin at all times during wearing.
- At no time should the mask hang around the neck of the wearer.

How to remove the face mask -

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Avoid touching the main part of the mask as this is contaminated, only touch the ear loops.
- Hold both of the ear loops and gently lift and remove the mask.
- Dispose of the mask according to the requirements of the hospital or environmental protection department.
- Clean your hands with soap and water or hand sanitizer.

Warnings & cautions

- Please read the instructions on how to put on the face mask and how to remove the face mask, carefully before use.
- This product is single use, it should not be re-used.
- In case of liquid splashing, mask damage, moisture and/ or significant increase in respiratory resistance during wear please remove and replace immediately.
- Please check the packaging before use, if the packaging is damaged - do not use.
- Non sterile masks should not be used in areas with strict microbiological index control.

Storage method:

This product should be stored in a cool, dry, corrosive gas free, well ventilated and clean environment.

Manufacturer -

Empteezy Ltd
4 Muir Road
Livingston
Scotland
EH54 5DR
www.empteezy.co.uk
Contact : advice@empteezy.co.uk
Tel: +44 (0)1506 430309

EU Representative -

Empteezy Benelux
Populierstraat 3
8800 Roeselare
BE 0474553395
Belgium
www.empteezy.be
Contact: info@empteezy.be
Tel: +32 51 63 68 54

Test House -

HeX Lab
Porte des Batisseurs, 145
B-7730 Estaimpuis
www.hex-group.eu
Contact : info@hex-group.eu
FI ID Number: 201780
Report ID Number:
EMPT01-2-02-200716-1-PDIFF-14683