

**Product reference :
3PMM50 Schoeller Masks Medical Face Mask**



CE These face masks bear CE marking to demonstrate compliance with MDD 93/42/EEC Annex I Health & Safety requirements -

EN 14683 : 2019 Medical Face masks

Bacteria filtration efficiency **BFE ≥ 98%**

Microbial cleanliness ≤ 30 **Breathability < 40**

CLASS 1
MEDICAL DEVICE

TYPE IIR

Suitable for use in situations where moderate to light amounts of fluid or aerosols are produced. **Please note this product is not classified as PPE**

Splash Resistance ≥ 16.0


These medical face masks are intended to be worn by patients and other persons to reduce the risk of spreading of infections, particularly in epidemic or pandemic situations

**- NON STERILE
- SINGLE USE ONLY**

3 layer structural protection - comfortable & breathable - shaped nose clip - ear loops - fitted to face

Instruction for use:

Warnings & cautions

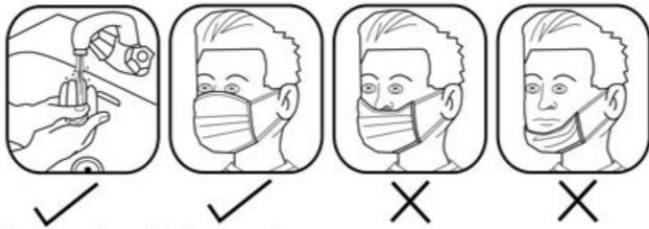

How to put on the face mask -

1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Remove mask from package, take out the mask, hold by the two ends/ ear loops with the nose bridge at the top - white side should be facing towards the inside to touch the face, blue side should be facing to the outside.
3. Cover the nose, mouth and jaw with the mask and loop the ear loops over each ear.
4. Adjust the position of the mask, mould or pinch the stiff edge over the bridge of the nose for a comfortable fit.
5. The mask should cover the nose, mouth and chin at all times during wearing.
6. At no time should the mask hang around the neck of the wearer.

How to remove the face mask -

1. Clean your hands with soap and water or hand sanitizer before touching the mask.
2. Avoid touching the main part of the mask as this is contaminated, only touch the ear loops.
3. Hold both of the ear loops and gently lift and remove the mask.
4. Dispose of the mask according to the requirements of the hospital or environmental protection department.
5. Clean your hands with soap and water or hand sanitizer.

1. Please read the instructions on how to put on the face mask and how to remove the face mask, carefully before use.
2. This product is single use, it should not be re-used.
3. In case of liquid splashing, mask damage, moisture and/ or significant increase in respiratory resistance during wear please remove and replace immediately.
4. Please check the packaging before use, if the packaging is damaged - do not use.
5. Non sterile masks should not be used in areas with strict microbiological index control.

Storage method:

This product should be stored in a cool, dry, corrosive gas free, well ventilated and clean environment.

Manufacturer -

Empteezy Ltd
4 Muir Road
Livingston
Scotland
EH54 5DR
www.empteezy.co.uk
Contact : advice@empteezy.co.uk
Tel: +44 (0)1506 430309

EU Representative -

Empteezy Benelux
Populierstraat 3
8800 Roeselare
BE 0474553395
Belgium
www.empteezy.be
Contact: info@empteezy.be
Tel: +32 51 63 68 54

Test House -

HeX Lab
Porte des Batisseurs, 145
B-7730 Estaimpuis
www.hex-group.eu
Contact : info@hex-group.eu
FI ID Number: 202223
Report ID Number:
EMPT01-2-03-200911-1-PDIFF-14683