# ROUSE

## REMOTE CONTROLLED KEGEL MASSAGER

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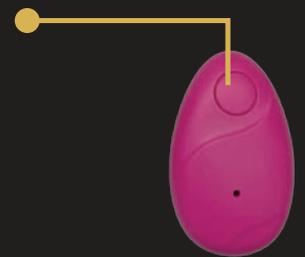
## INSTRUCTIONS FOR USE / MODE D'EMPLOI

1. To turn on the device, press and hold the button for 3 seconds. Light on device illuminates.
2. To start vibrator pattern, press and hold button for 1 second. To change, press button for 1 second. Continue to press button to cycle through all modes – light changes with each.
3. To turn off device, press and hold button for 3 seconds. Light on device shuts off.  
Can be used with water-based lubricants.  
Do not use with silicone-based lubricants.

### REMOTE CONTROL INSTRUCTIONS

1. Remove plastic piece from remote battery panel before use.
2. To turn on remote, press and hold button for 1-3 seconds. Light on remote illuminates.
3. Note, device must already be on for remote to operate the device. To change vibrator pattern on device, press button on remote for 1 second. Continue to press button on remote to cycle through vibration modes.
4. To turn off remote, press and hold button for 3 seconds. Light on remote shuts off.  
Note, turning off the remote does not turn off the device; the device must be turned off separately.

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## HOW TO CHARGE THE BATTERY / COMMENT CHARGER LA BATTERIE

1. Before charging, make sure the outlet is free from water or lubricant residue.
2. To charge the device, insert the cord into the side of the device and plug the USB cord into a power source.
3. Unplug the device before use.
4. To replace the remote control battery, open the panel located on the back of the remote.

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## CLEANING, MAINTENANCE AND STORAGE / NETTOYAGE, MAINTENANCE ET STOCKAGE

---

- Using and maintaining the **SKYN™ ROUSE REMOTE CONTROLLED KEGEL MASSAGER** correctly will prolong the lifespan of the device.
- Clean thoroughly before first use and after each subsequent use. Clean gently with mild soap and water. Do NOT clean with any cleaner containing alcohol.

**CAUTION:** THIS DEVICE IS NOT WATERPROOF. IT MUST NOT BE SUBMERGED IN WATER OR USED IN THE SHOWER OR BATH.

After cleaning, pat dry. Keep somewhere dry and cool.

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## DISPOSAL INSTRUCTIONS / COMMENT JETER

---

Dispose of this device according to your local regulations.  
For more information, please contact your municipal waste disposal office.

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