

2nd Gen 7-in-1 Professional Hydro Dermabrasion Machine

USER MANUAL



Welcome

**to your 2nd Gen 7-in-1 Professional Hydro Dermabrasion
Machine**



TABLE OF CONTENTS

PREFACE

I. PRODUCT INFORMATION & SPECIFICATION	2-4
II. PRODUCT APPEARANCE	5
III. WARNINGS	6-7
IV. MACHINE OPERATION	
a. Hydrogen Oxygen Hydration Handle	8
b. Hydrogen Oxygen Cleansing Handle	9
c. Cold Hammer Repairing Handle	10
d. Multipolar RF Lifting Handle	11
e. Ultrasonic Import Handle	12
f. Ultrasonic Skin Scrubber Handle	13
g. LED Light Therapy Mask	14
V. TROUBLESHOOTING	15

PREFACE

Thank you for choosing **REJUVA FRESH** 2nd Gen 7-in-1 Professional Hydro Dermabrasion Machine. This machine gives your spa clients the glowing skin they crave. In a single treatment session, you can thoroughly refresh, revitalize and nourish, revealing a healthier, younger-looking facial complexion.

This machine is an upgraded version of the 1st Generation 7 in 1 Professional Hydro Dermabrasion Machine. The upgrade includes a new design of the hydrogen oxygen hydration probe, which improves the skincare product penetration to up to 80%. This also includes a radio frequency probe that has 9 poles (upgraded from 2 poles), with more RF crossover points and expanded active area. Using high grade raw materials, the machine body was redesigned for that aesthetic look!

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com.

Thank you again, and please enjoy your Rejuva Fresh 2nd Gen 7-in-1 Professional Hydro Dermabrasion Machine!

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

This machine uses hydro dermabrasion. This type of treatment makes the water molecules quickly penetrate the dermis to achieve the effect of skin rejuvenation and whitening, giving you a fresh face! Included in this machine are six handles:

1. **Hydrogen Oxygen Hydrating**: Quick penetration for deep moisturization
2. **Ultrasonic Import**: Promotes micro circulation, cell renewal & repair, and helps skin nutrition be more fully absorbed
3. **Cold Hammer Whitening**: Calm down the skin after RF treatment, refresh skin & reduce puffy eyes
4. **Skin Scrubber**: Remove dead skin and excess cuticle
5. **Hydrogen Oxygen Cleansing**: Deep clean, remove grease, acne, blackheads and dirt
6. **RF Lifting**: Increase collagen production, skin tightening and firming
7. **LED Light Therapy Mask**: Treat acne, reduce redness and wrinkles



I.2 Product Specification





Machine Power	350 W
Voltage	100 V/110 V/ 220 V Voltage selected according to your region
Pressure	80KPa-106KPa
Vacuum Suction	70cm HG
Machine Size	25 x 40 x 38 cm

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Item	Quantity	Image
Main Unit	1 set	 A blue and white medical device with a control panel and two large blue reservoirs at the bottom.
Hydrogen Oxygen Hydration Handle	1	 A white handheld device with a nozzle and a grey handle.
Hydrogen Oxygen Cleansing Handle	1	 A white handheld device with a nozzle and a grey handle, similar to the hydration handle but with a different nozzle design.
Multipolar RF Lifting Handle	1	 A white handheld device with a circular faceplate containing several small circular electrodes.
Ultrasonic Import Handle		 A white handheld device with a circular faceplate and a metal ring around the edge.

I. PRODUCT INFORMATION & SPECIFICATION

Item	Quantity	Image
Cold Hammer Repairing Handle	1	
Ultrasonic Skin Scrubber Handle	1	
LED Light Therapy Mask	1	
Power Cable	1	

II. PRODUCT APPEARANCE

II.1 Front view



III. WARNINGS




DO NOT use if experiencing fever or if with infectious disease.



DO NOT use if pregnant, menstruating, conceiving, or breastfeeding



DO NOT USE if with: heart disease, hypertension, thyroid, malignant tumor, renal failure, epilepsy and muscle strain

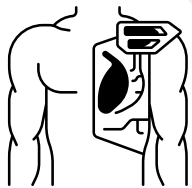

DO NOT use if with with metal or electronic implants (metal contraceptive ring, cardiac pacemaker, cardiac defibrillator, nerve stimulator, drug pump and stent)

III. WARNINGS



DO NOT use if had undergone major surgery,
Diabetic and with cerebrovascular disease (stroke patients)




DO NOT use if with artificial bone or have any implants.



Before starting the treatment,
remove any metal jewelry you are wearing.

IV. MACHINE OPERATION

a. Hydrogen Oxygen Hydration Handle

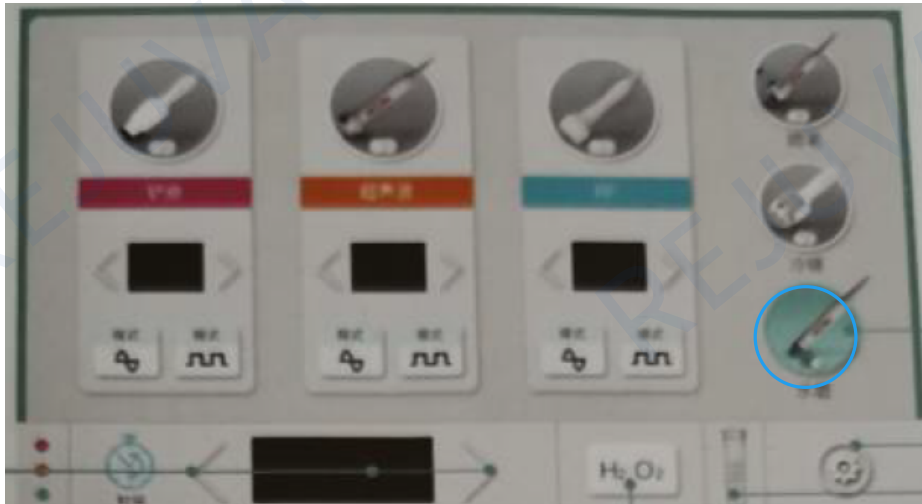


HOW TO USE

1. On the interface, select Hydrogen Oxygen Hydration Handle.
2. Remove makeup and clean the face.
3. Twist the middle knob and select "D (H₂ O₂)" Bottle
4. Tap "H₂ O₂" button on the screen and wait for 3 minutes.
NOTE: There should be enough H₂ O₂ on the Bottle.
5. Adjust the vacuum suction and water flow, then proceed the treatment of the whole face for 10 minutes.
6. After the treatment, twist the middle knob to select "C" Bottle to clean the Hydrogen Oxygen Hydrating Handle. Twist the knob to "Flow Adjust" to maximize the water flow and then maximize suction. Turn ON the device, cover the handle with the lid to cover the Hydrogen Oxygen Hydrating Handle. This will allow the device to clean automatically.

NOTE: When the liquid in the bottles A,B,C and D are all used up, fill the bottles with hot water and clean using the same process as step 6 to avoid blocked conduit.

IV. MACHINE OPERATION

b. Hydrogen Oxygen Cleansing Handle

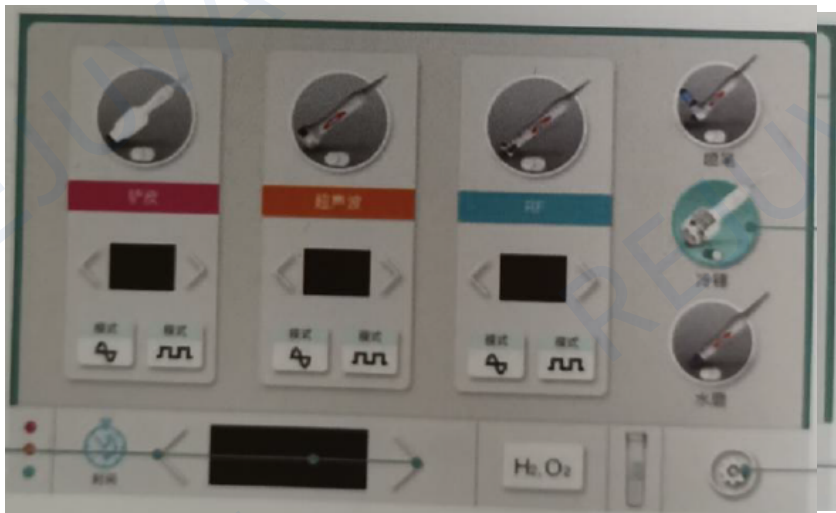


HOW TO USE

1. On the interface, select Hydrogen Oxygen Cleansing Handle.
2. Clean the treatment area.
3. Fill the bottle of the Hydrogen Oxygen Cleansing Handle with diluted essence liquid.
4. Tap the button for Hydrogen Oxygen Cleansing Handle.
5. Use the handle on the treatment area, press the switch button to start the treatment.

IV. MACHINE OPERATION

c. Cold Hammer Repairing Handle

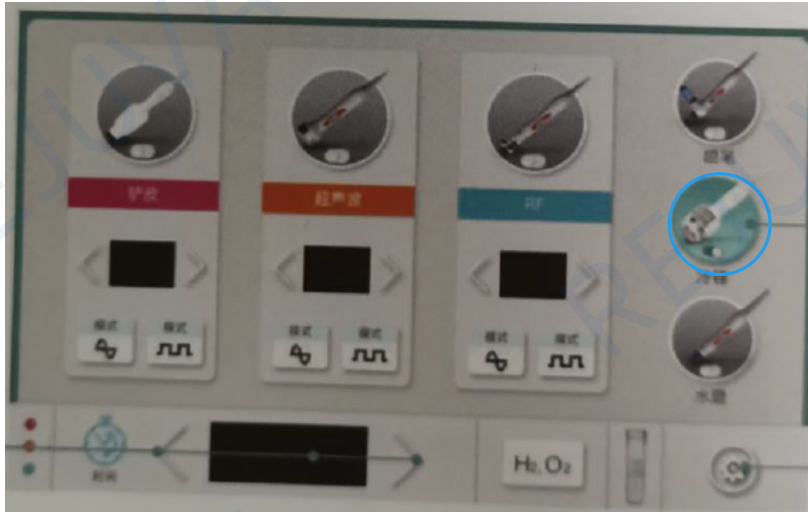


HOW TO USE

1. On the interface, select Cold Hammer Repairing Handle.
2. Clean the treatment area.
3. Move the handle in contact with the skin and start the treatment.

IV. MACHINE OPERATION

d. Multipolar RF Lifting Handle

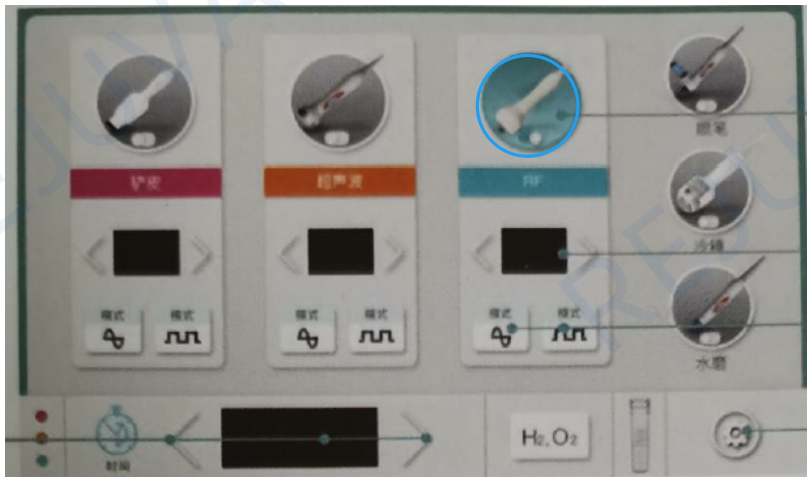


HOW TO USE

1. On the interface, select Multipolar RF Lifting Handle.
2. Clean the treatment area.
3. Apply gel on the treatment area.
4. Adjust the working time, RF level, and modes.
5. Move the handle close to the skin, tap Multipolar RF Lifting on the screen or press START button on the handle to start.
6. Move in circular motion, from down to up in circles at the neck. If doing treatment on the neck, the customer needs to lift his head up . Each treatment area takes 10-15 minutes.

IV. MACHINE OPERATION

e. Ultrasonic Import Handle

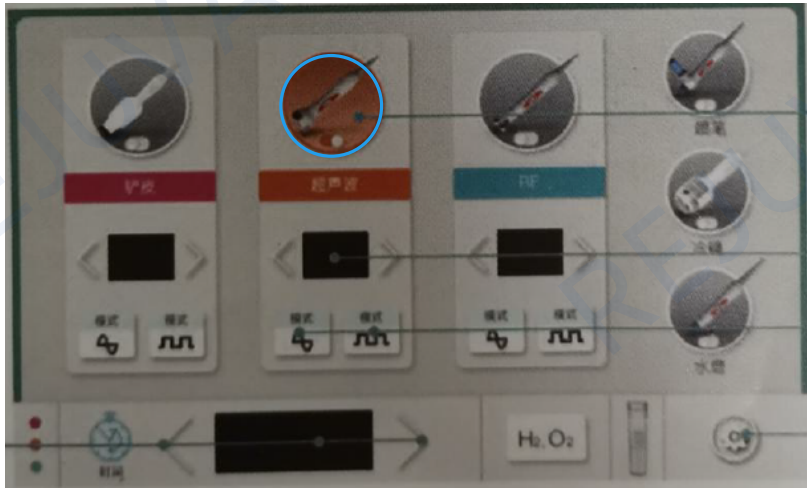


HOW TO USE

1. On the interface, select Ultrasonic Import Handle.
2. Clean the treatment area.
3. Apply essence on the treatment area.
4. Adjust the working time, ultrasound level, and modes.
5. Move the handle close to the skin, tap Ultrasonic Import on the screen then press START button on the handle to start.
6. Move slowly in circular motion along the contour lines of the treatment area for 5-8 minutes.

IV. MACHINE OPERATION

f. Ultrasonic Skin Scrubber Handle

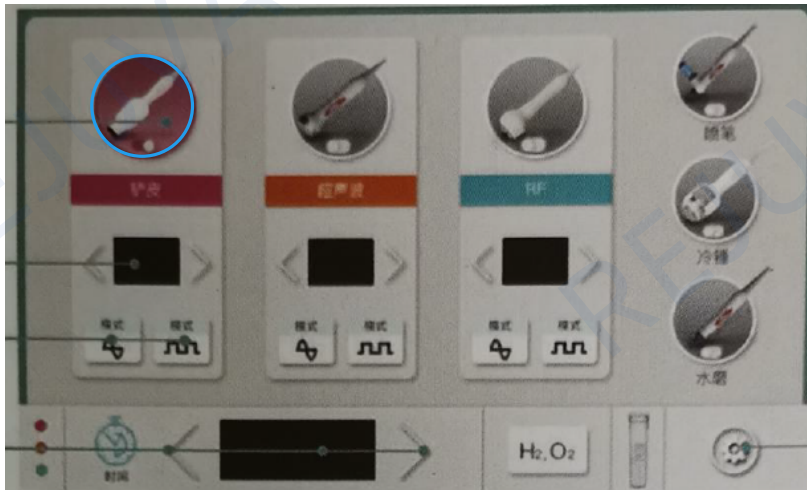

HOW TO USE

1. On the interface, select Ultrasonic Skin Scrubber Handle.
2. Adjust the working time and modes, press START to start the treatment.
3. Clean the treatment area.
4. Sterilize the scrubber.

REMOVE DIRT IN THE CUTICULAR LAYER


1. Apply enough clean water on the skin of the treatment area.
2. Tilt the handle 45 degrees close to the skin from the bottom to the tap with moderate speed.
3. Treatment time is 20 minutes.

MASSAGE THE LYMPH

1. Apply enough rose water on the skin.
2. Move the handle close to the skin, tap Ultrasonic Skin Scrubber on the screen then press START button on the handle to start.
3. The scrubber head slides flat the skin according to the texture of the muscles.

IV. MACHINE OPERATION

g. LED Light Therapy Mask

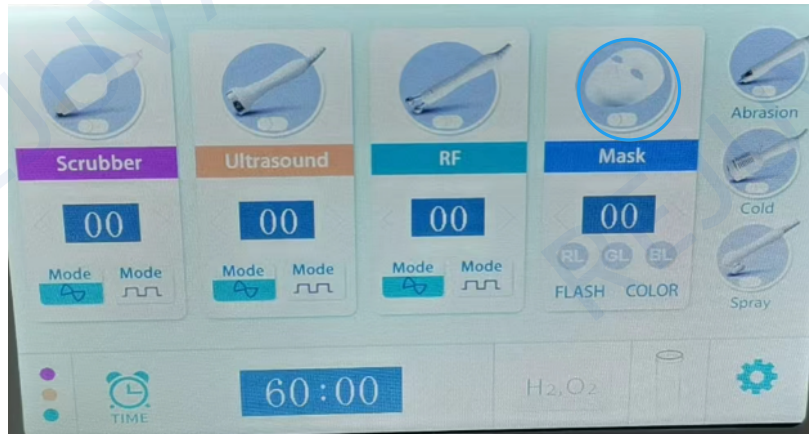

HOW TO USE

1. On the interface, select LED Light Therapy Mask.
2. Clean the face.
3. Select the FLASH COLOR and adjust time press START to start the treatment. Suggested treatment time is 10 minutes.



RED(630nm)

Increase cell activity, speed up metabolism, and promote collagen formation

(520nm)GREEN

Neutralization, balance and stability, can reduce skin oil secretion



BLUE(470nm)

Bactericidal anti-inflammatory, inhibit inflammation, repair skin

(590nm)YELLOW

Improve cell oxygen exchange, replenish energy to skin cells



PURPLE

Dual frequency light, repairing acne.

CYAN

Enhances cell energy and promotes metabolism



WHITE

Improve fine lines and loose skin.

V. TROUBLESHOOTING

If the device is not working properly, do not dis-assemble.

Please check below troubleshooting and solutions for common problems encountered:

Trouble	Cause	Solution
There is no response after turning on.	The power plug is not plugged into the socket; The power switch is not on; The power fuse blows out.	Check whether the plug and socket are plugged in properly.
Some handle cannot be used properly.	The "Start" button for the handle was not tapped; The probe was installed improperly; The probe was damaged; The jack at the back of the device is not properly installed.	Tap the "Start" for the handle; Check whether the probe is installed properly; Change the probe or silicone ring; Install again.
Liquid leakage.	Damaged or missing silicone ring in sealing position; Improper installation.	Change the silicone ring in sealing position; Send it back to manufacturer for repair.



REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
1-800-249-1310
rejuvafresh.com