

8in1 Hydra Facial Machine

User Manual



Machine Introduction



Tips & Sealing Ring for Cleaner Handle



Accessories Description

1.

Bubble cleaner handle(with 8 tips)

First clean skin with bubble cleaner handle. Suck skin from middle to both sides, slide quickly, do not stay too long. For sensitive skin, use flat tip and vortex tip. For large pores and T zone, use the tip with matte surface.



2.

Ultrasound handle

The Ultrasound import products deep into subcutaneous tissues, promote microcirculation and skin renewal. Products can be penetrate deep into the skin, speed up absorption

3.

RF radio frequency handle

Open the pores, positioning tissue heating, open the pores to promote subcutaneous collagen contraction and taut, relieve wrinkles, produce new collagen, the skin will be more compact later.



4.

Ice repair handle

Ice repair handle can froze skin, shrink pores the ice energy can penetrate from epidermis to deep skin, instantly promote strong contraction of skin pores, thus a layer of natural freezing mask is formed in the skin surface to lock nutrition facts

5.

Micro-electric therapy handle

BIO face lifting, increase cell permeability, help body better absorption of nutrients and water, oxygenated blood. enhance collagen synthesis level, reduce wrinkles, so that the skin is more soft, flexible and shiny.





6. Oxygen sprayer

It can inject oxygen and spurt oxygen just with one machine. Therefore, it solves the difficulties of hypoxia. It injects 98% purified oxygen and activates nutrition distillate to the deep-seated skin through impulsing force in order to accelerate the metabolism of cells, whiten and tender the skin.

1 Red light(650-730nm)

1

Will stimulate the production of collagen, collagen is an essential protein used to repair damaged tissue and to replace old tissue, best for fine lines, and will also help to reduce large pores.



2 Blue light (430-450nm)

2

Anti-inflammatory and removing Acne. The blue light furthermore has calming action which is very effective for hypersensitivity.



3 Green light(525-550nm)

3

The balance of the green light can balance color pigment, reduce fine lines, nutrient aging skin, speed up the healing process of the wound, and lighten the scar.



4 Purple light with wave at 640nm


4

combine with red and blue, it can strengthen the protein fiber tissue, shrink pores, at the same time, eliminate face yellow, wrinkles, premature acne-prone skin; Accelerate the blood circulation function, promote the activation of skin tissue.




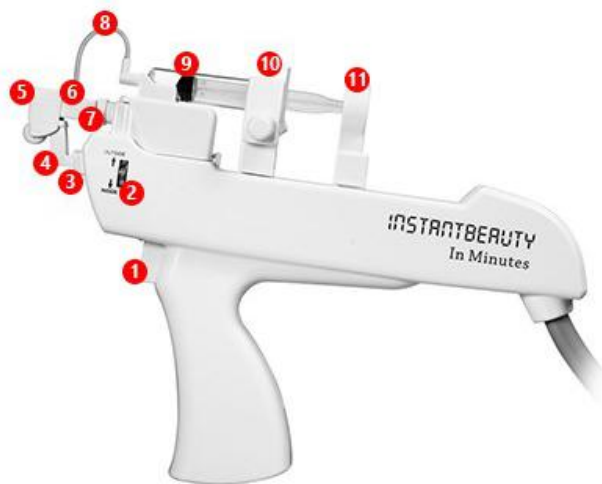
6 Cyan light with wave at 640nm

In combination with green and blue, it can adjust the gland function of skin, improve oily skin, black, pink, acne, At the same time, promote the synthesis of protein and collagen, activate the skin, the relaxation of skin contraction.



7 Laser light with wave at 510nm

can calm, soothing, prevent allergy to desensitization, and so on.

HANDPIECE OPERATION	
1	Switch
2	Micro Crystal Head Scale Adjustment
3	Total Scale Adjustment 4mm
4	Micro Crystal Head Holder
5	Micro Crystal Head Cover
6	Micro Crystal Head
7	Connecting Rod
8	Micro Crystal Head Connecting Tube
9	Micro Crystal Tube
10	Micro Crystal Tube Clip
11	Micro Crystal Tube Propeller

The needle free injection gun using Vacuum Vanadium Titanium open the skin channel, inject some products with precise dose and time in skin Mesoderm, there by achieve whitening, skin rejuvenation, shrink pores and other effects

Treatment Step

1. install the handpiece to the machine
2. Add water or solution for the two bottles
3. press On/off red button to turn on machine
4. Machine screen will shows like following
5. Click the screen, enter the function option screen ,shows below

小气泡水光嫩肤仪

8in1 Hydra Facial Machine




6. Choose the function you need to do treatment

Hydra Peel Water Dermabrasion(Cleaner Handle)



Hydra dermabrasion completely changed the traditional way, which is hand cleaning skin relying on individual's practice skills.


Hydro microdermabrasion uses vacuum suction mode controlled by intelligent process, through the combination of products and equipment to improve skin texture. The main advantage of the hydra-dermabrasion machines lies in that it is much gentle by using only water, and enables treatment diversification by using with skin care products for different functions like essential oil, whitening products, lactic acid, salicylic acid and more, to achieve various aims on customers' request.

firstly use hydra cleaner function ,click  , and then enter hydra cleaner function screen



1). Install hydra tips to the hydra cleaner handle ,like this

2). Click  and  to adjust the time , suggest 10 minutes.

- 3). Click  to let the hydra cleaner handle working.
 4). If hydra cleaner handle have a little water comes out ,Rotate this



button to adjust the water level and pressure



- 5). Before treatment, we suggest to use finger to block the tips for few seconds, start to do treatment when water come out from pipe.



Ultrasonic(Ultrasound handle)


- 1.By the high frequency vibration (1MHz-3MHz), generate ultrasonic with strong penetrability. It is a special instrument that could radiate an alternate wave and spread around. It is much stronger than the normal sonic wave, with high frequency, good directivity, strong penetrability and great expandability.
- 2.Because of the high frequency vibration, it will do soft and micro massage to the skin cells, accelerate the blood circulation, facilitate the metabolism, activated the cells and improved the absorption of nutrition.
- 3.High frequency micro massage could eliminate the aged cells, expel the toxin and fine the wrinkles. Accompany usage with cosmetic product or medicine, it could treat and improve the skin problems.
- 4.The frequency of ultrasonic could also generate resonance vibration of the cells, consume the fat and improve water absorbing capacity of cells, rejuvenate the skin and recover the elasticity.



Click , and then enter Ultrasonic function screen




1). Click  and  to adjust the time and energy , suggest 10 minutes, energy start from 5, can increase accordingly

2). Click , the ultrasound handle will show light, can start to do treatment now.

Tips 1: Before using this function, clean the skin where would be treated by cleaner.

Tips 2: After cleaning the skin, choose the lotion or essence according to the skin condition and disease to achieve the best effect.

Tips 3: During treatment, the beautician should hold the probe steadily, rotate slowly and evenly, let the skin have a sufficient ultrasonic treatment.

3). Press  after finishing the treatment, keep the essence on the skin and let it penetrate thoroughly.

BIO Face Lift(Micro electric therapy handle)

1. BIO can inhibit recession of human skills ,designed by micro current programe and the human body bioelecticty to resonate, which stimulate cell cell activation ,promote metabolism ,recover the strongness and vitality of organism .



2. Accelerate the blood circulation of micro-vascular, improving cell permeability, moisture, making the skin smooth, elastic.


3. Current micro-current stimulation is as deep as glial tissue under the skin and muscle tissue. Stimulate and restore dermal elastic fibers so that stretch wrinkles.

4.The cause of skin aging is slow metabolism, reduction of electric energy, cells repair capacity is insufficient to fight aging of the body caused by internal or external factors. Therefore, recharging energy to recover normal metabolic function is an effective method of delaying aging.

Click  to enter BIO function



1). Click  and  to adjust the time and energy , suggest 10 minutes, energy start from 5, can increase accordingly

2). Click , the BIO handle will show light, can start to do treatment now.

Tips 1: Use special cream in the face such as gels, and then hold the BIO handle with your hand.

Tips 2: Move the probe from the center parts of the face to the outside, moving from bottom to top.

Tips 3: Face lifting methods: from the bottom jaw to ear, from the nose to the temples, moving outward from the central forehead.

Tips 4: Eye lifting methods: from the inner corner to temple, probe should not be too close to the areas around the eyes, and you should be gentle in the operation.

RF Radio Frequency

with the thermal conduction of radio frequency electric wave which can stimulate the ions and electric colloid particles in the organism moving or shaking with high speed and produce thermal power to heat the dermal layer with molecule resonance. The whole process never relates to hurt people ' s skin. When the temperature increases to 45 °C -60 °C , there much more collagen are stimulated to be secreted, then fill with the collagen gaps and realign in the deep collagen organization. After the new collagen structure rebuilt, the skin become firming , elasticity and glossy, wrinkles also move away.




Main Function

1. Increase skin and muscular organization temperature about 4-5°C and stimulate cells' functionality.
2. Increase blood flow speed about 4-5times and substance purification. Helpful on the nutrient, antibody, white blood cell and lymph circulation.
3. Increase skin elasticity to 5-10 times, and make the ligament, joint capsules and scar organization more strong.
4. Significantly reduced joint stiffness: compared with physiotherapy, heat therapy
5. can reduce the joint stiffness and inconvenience.
5. Pain relief: For pain caused by coronary heart disease, it can reduce pain if using thermoelectric probe congestion. Increasing the heating pain place or nerve around can reduce the pain of Muscles, ligaments and organizational links.
6. Relaxing the muscles atrophy: reducing Al broke motor nerve fibers can relieve muscle atrophy.
7. Stimulating the therapy of chronic inflammation: the way to stimulate the veins and inner loop which help to absorb inflammatory fibroid can increase oxygen, nutrients, white blood cells and antibodies.
8. Stimulating wound treatment: according to the increase of blood flow making the arteriolar and capillary expanded can stimulate skin circulation and physiotherapy



Click  to enter rf function



- 1). Click  and  to adjust the time and energy , we advise 10 minutes, energy start from 5
- 2). Click  , the BIO handle will show light, can start to do treatment now.

Treatment Pictures



Step1: Apply RF oil evenly on the face and keep circle moving at even pace from jowl to basal part of the ear.



Step2: Lift from corners of the mouth to the part of cheek in front of ear.



Step3: Move from inner to outer corner of eye.



Step1: Lift from upward side of collarbone to lower jaw bone.



Step2: Lift from upward side of collarbone to lower jaw bone keep clear of prominentia laryngea.



Daub the cooling gel equably



Move circularly and uniformly




from wrist to elbow





Move circularly and uniformly on the arm, can prevent the arm skin sagging


Cooling hammer(Ice repair handle)

It is a derivative instrument of medical instrument, safe and reliable.
Cold hammer has cooling effect, tighten the skin texture, anti-wrinkle.
Do the treatment about 8 minutes

Click  to enter cooling hammer function




1). Click  and  to adjust the energy , we advise 8 minutes, energy start from 5, can increase accordingly

2). Click  to let the cooling hammer hydra peel start work. No light for this handle.

Treatment on the face:

Start from ophryon and spread away, move upwards. As the photos shows.

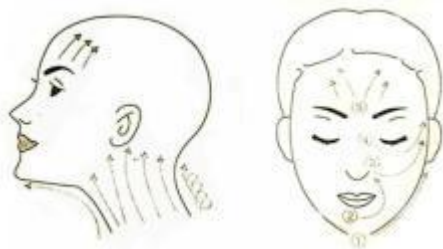


1. From lower jaw to ear.
2. From lower jaw to nose.
3. From nose to Temple.
4. From inner corner to the Temple, do not get too close to the eyes and be tender.

Treatment on the body:

Move the hammer upwards from the bottom like drawing circles.

While treat on the breast of female, move around the pectoralis major muscle, like the right photo shows.

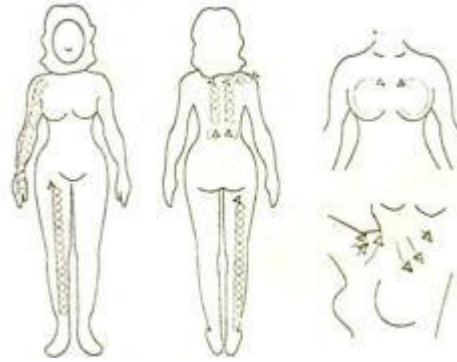

Oxygen Spray Gun

Needle free water oxygen injection beauty equipment adopts traceless and harm ess noninvasive method.extracting the oxygen in hte air, and comporessed to acti ve factor by molecuar filter, the oxygen release to skin and inject into skin by The machine.it is a effective and safe way to send oxygen to skin, so to remove wrinkle s and smooth skin.



Click  to enter oxygen spray function function




1). fill essence fluid or related skin care products into bottle of gun, don't fill cream or powder, not easy to spray out, gun will get stuck.



2). Click  and  to adjust the time , screw on the gun to adjust dosage



rotate the

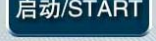3). Click  to let the spray gun ready for working, press switch



on gun to do treatment. After treatment, fill some pure water into bottle for cleaning bottle, and spray water out.



LED mask

LED phototherapy skin mask instrument that uses the principle of light irradiation, both simple and safe.



Click  to enter mask function.



1). Click  and  to adjust the time&color&intensity. Please refer to page4-5 for color functions. Advise time 15minutes, intensity start from can increase accordingly.


2). Before using LED mask, put silk mask or medical repair mask on face, then put LED mask. Like following.




Crystallite injection

Different from mesotherapy guns that use needles, we use a microchip with multiple pins to deliver nutrition and essentials, which can be reached to subcutaneous layer under high pressures from the pistol.




A. Click  to enter injection function



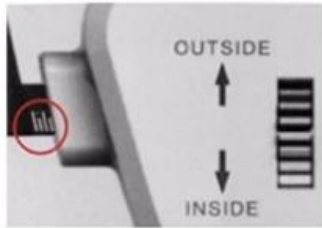


1 mm

Depth fit for thin skin location such as forehead and risorius

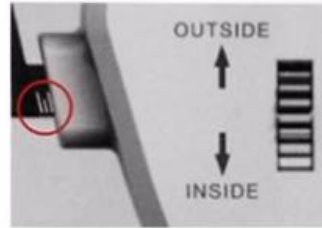

2 mm

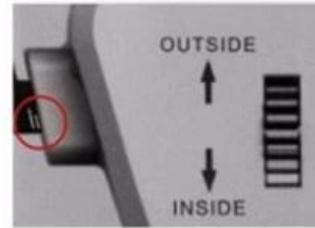

Fit for face, neck and other normal skin





3 mm

3mm fit for thick and serious problem skin

1). Suck serum or hyaluronic acid into tube, Install the injection probe to the pistol, press retreat on screen, put the tube into pistol, press forward to fix it.

2). Click  and  to adjust time speed and dosage, the time is based on speed and dosage, if the serum is diluted, set lower speed and dosage, if the serum is thick, set higher speed and dosage.

3). Click  to let the pistol ready for working. Press switch on pistol to do treatment until the serum finish. Press  to stop.



