

9 in 1 Cavitation Lipo Laser Body Sculpting Machine

USER MANUAL



Welcome

to your **9 in 1 Cavitation Lipo Laser Body Sculpting Machine**



TABLE OF CONTENTS

PREFACE

I. PRODUCT INFORMATION & SPECIFICATION	2-5
II. PRODUCT APPEARANCE	6
III. WARNINGS	7-8
IV. RECOMMENDATIONS	9
V. TREATMENT SELECTION	
V.1 Cavitation	10-11
V.2 CRYO Sculpting	12
V.3 Vacuum RF	13
V.4 10 Polar RF	14
V.5 8 Polar RF	15
V.6 Tripolar RF	16
V.7 Bipolar RF for Face and Around the Eye	17
V.8 Laser Lipolysis	18
VI. RECOMMENDED TREATMENT POSITION	19-20

PREFACE

Thank you for choosing REJUVA FRESH 9 in 1 Cavitation Lipo Laser Body Sculpting Machine. This stylish slimming machine is designed to achieve the most effective body contouring, including reduction of fat & cellulite on your targeted areas plus skin firming & tightening.

Lipo laser emits low levels of laser energy which creates a chemical signal in the fat cells breaking down the stored triglycerides into free fatty acids, glycerol and water molecules. This is similar to the natural response initiated by the body when it needs to use its stored energy reserves. These free fatty acids is then transported throughout the body by the lymphatic system to supply the body with energy. Again, just as the body would respond in times of caloric scarcity.

The use of this machine is safe, minimally-invasive procedure, and easy to operate. There are no bleedings, no pain, and side effects.

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com.

Thank you again, and please enjoy your body sculpting machine!

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

This machine uses collective strong sound wave head, strong sound waves of 40 KHz is emitted to vibrate fat cells at top speed and produce numerous vacuum air pockets inside and outside the fat cells. Robustly impacting the fat cells to generate introverted blast and disintegrate triglyceride into glycerol and free fatty acids, exhausting the integrated glycerol and free fatty acids through hepatoenteral circulation.

Finally, vacuum RF and energy electrode are used for positioning and tightening fat. In physics, this is known as "cavitation" . Micropore introverted blast inside and outside cell would lead to enhanced molecular motion and a higher energy level. This will finally cause fat cells rupture thereby achieve the effects of body building and losing weight.

I.2 Product Specification

Machine Power	320 W
Cavitation Power	75 W
Cavitation Frequency	40K / 80K
Laser Wavelength	635-650 nm
Laser Power	5 MW per diode
Vacuum Pressure	-20 KPa
Radio Frequency (RF)	5 MHz

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Main Unit	1 set	
Vacuum RF	1	
40 KHz Cavitation Head	1	
Ten Polar RF Head	1	

I. PRODUCT INFORMATION & SPECIFICATION


Eight Polar RF Lipolaser	1	
Tripolar RF for Face	1	
Bipolar RF for Face and Around the Eye	1	
Cryo Sculpting	1	

I. PRODUCT INFORMATION & SPECIFICATION

Laser Positioning Plates	6 Large 2 Small	
Power Cord	1	

II. PRODUCT APPEARANCE

II.1 Front view

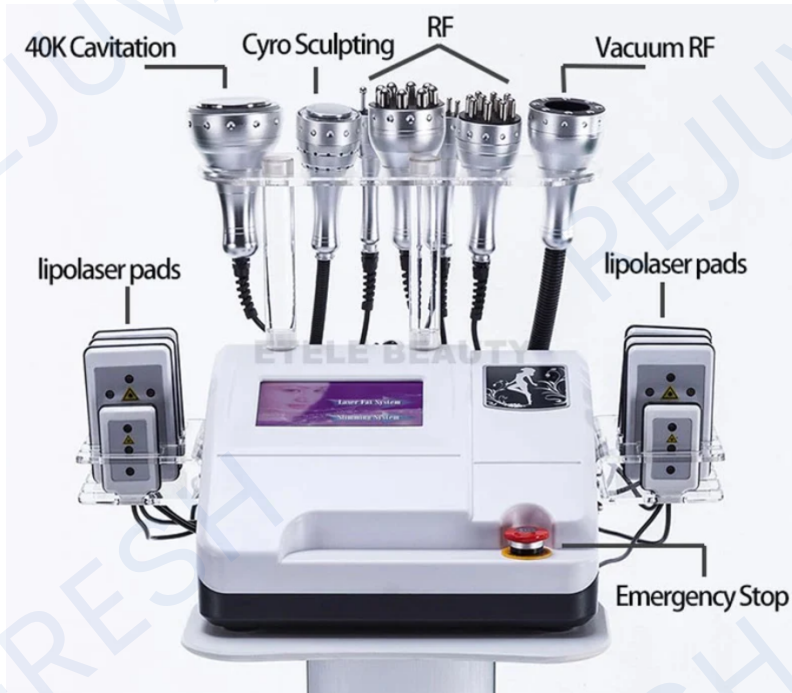

II.2 Back view



Unscrew this knob and use the funnel to put water inside the machine. Before using CRYO, make sure that the machine is filled with water.

III. WARNINGS

III.1 Please follow these precautions before/during/after use



DO NOT use if pregnant, menstruating, conceiving, or breastfeeding



DO NOT use if epileptic



DO NOT use if there are open wounds or any skin infections such as eczema or dermatology issues



DO NOT use if there are heart conditions (such as with arrhythmias) especially those with pacemakers

III. WARNINGS



DO NOT use if with kidney disease



DO NOT use if have metal on any part of the body



Before using the instrument,
remove any metal jewelry you are wearing

IV. RECOMMENDATIONS

1. Be sure to use the ultrasonic special gel.

2. Handle the heads with care, avoid knocks.

3. Use and prepare enough gel to avoid overheating the heads.

4. Don't linger too long on a single region and avoid the bones.

5. Don't use disinfectants to disinfect the heads, use wet cotton or dry towel instead.


6. Make sure the power supply is connected and plugged well.

7. If the machine is not for use for a long period of time, switch it off and unplug the power supply.

8. The machine should be rested every hour of continuous operation for about 10 minutes.

V. TREATMENT SELECTION

V.1 Cavitation

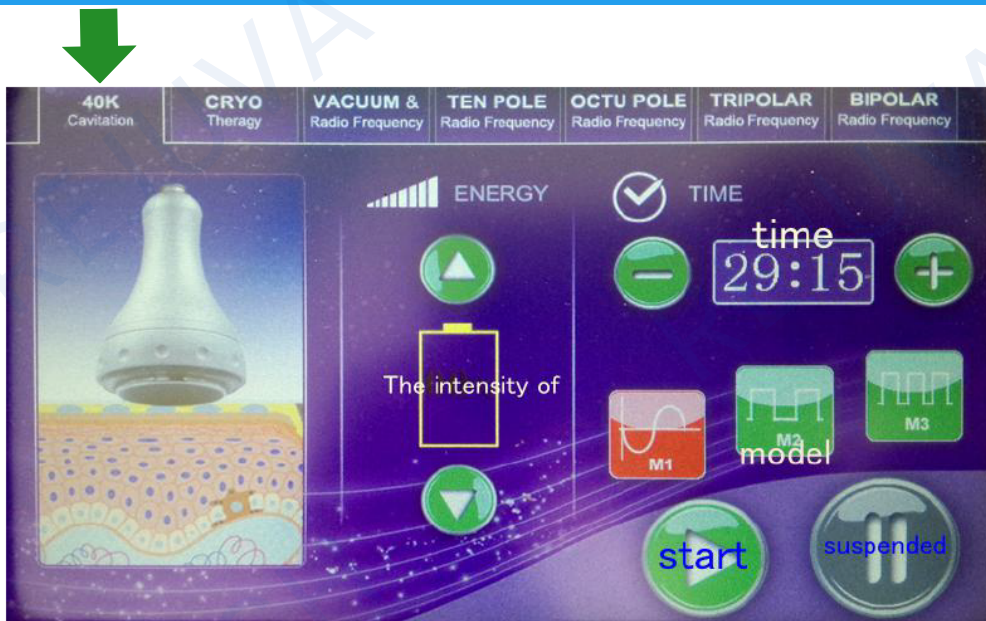

1. Select the appropriate treatment and install the corresponding handle.

Handle: 40 KHz Cavitation Head

Treatment: Cavitation

Proceed with the treatment.

There is 4 MODE of M1, M2, and M3 as described as follows:

"M1" is continuous working for 3 sec and discontinuous for 2 sec.

"M2" is continuous working for 2 sec and discontinuous for 2 sec.

"M3" is continuous working for 1 sec and discontinuous for 1 sec.

2. Apply slimming cream or cooling gel to the part of skin to be treated. Operate with the 40KHz cavitation head for 15-20 minutes with mainly the spiral movement on body.

V. TREATMENT SELECTION

V.1 Cavitation

3. Press "-" "+" to adjust the TREATMENT TIME.
4. Below the TREATMENT TIME is the FREQUENCY INTENSITY LEVEL.
5. Clicking "-" to decrease the strength or click "+" to increase it.
6. Press "START" to start the strong sound wave explosion fat head.
7. Use the Cavitation Head to massage the specific part of skin to be treated circularly after applying the slimming cream for about 15 to 20 minutes. It could quicken the blood circulation in the treated part of skin and has the effect of decomposing fat.


HELPFUL TIPS:



- Keep close contact between the probe and the skin.
- Adjust the ENERGY LEVEL from low level to high level.
- Pay attention to the temperature of the cavitation handle when using it, and stop using it for ten minutes after half an hour of use.
- Cavitation is forbidden to work without gel.

V. TREATMENT SELECTION

V.2 CRYO Sculpting

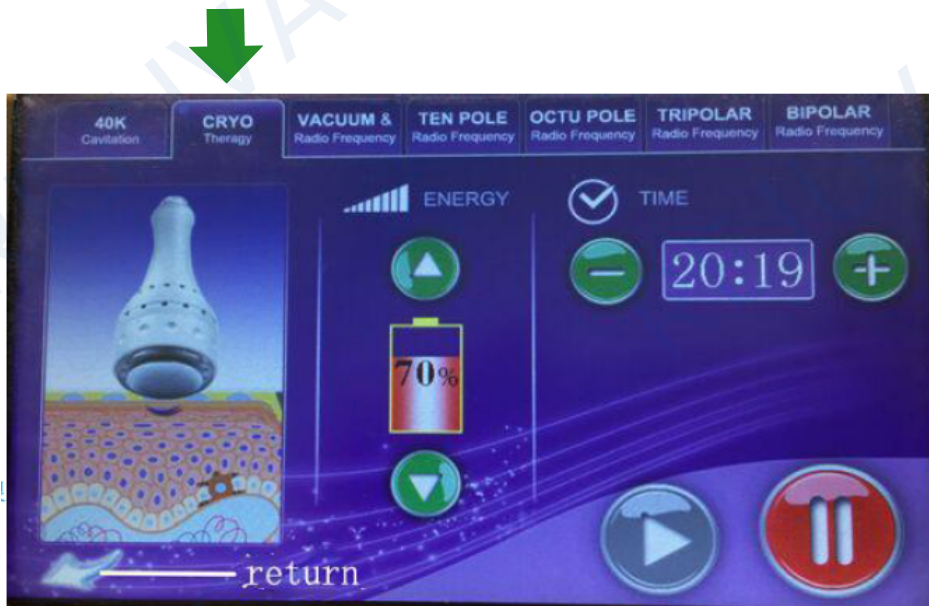

1. Select the appropriate treatment and install the corresponding handle.

Handle: Cryo Sculpting Head


Treatment: CRYO Therapy

Proceed with the treatment.

2. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability).
3. Press "START" key to begin the treatment.
4. Apply the slimming cream on the area being treated.
5. After 30 to 40 minutes, use the Cryo Sculpting Head to massage the specific part of skin circularly. This would quicken the blood circulation in the part of skin to be treated and has the effect of decomposing fat.

V. TREATMENT SELECTION

V.3 Vacuum RF

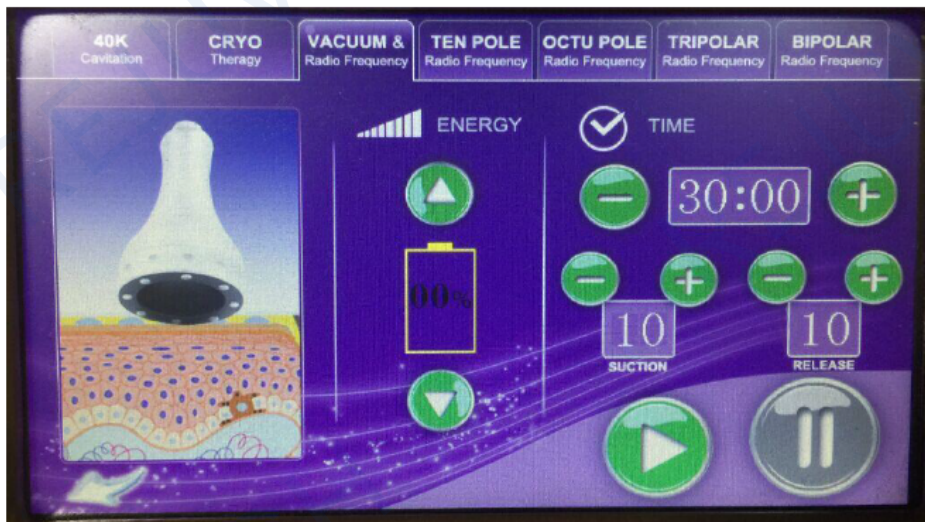

Select the appropriate treatment and install the corresponding handle.

Handle: Vacuum RF Head


Treatment: VACUUM & Radio Frequency

Proceed with the treatment.

1. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability).
2. Set the TIME. Press "START" key to begin the treatment.
3. Apply the slimming cream on the area being treated.
4. After 30 to 40 minutes, use the Vacuum RF Head to massage the specific part of skin circularly. This would quicken the blood circulation in the part of skin to be treated and has the effect of decomposing fat.
5. You can adjust the energy level according to customer's acceptable level.
6. Use the Vacuum RF Head for 15-20 minutes with mainly the spiral movement on body

V. TREATMENT SELECTION

V.4 10 Polar RF

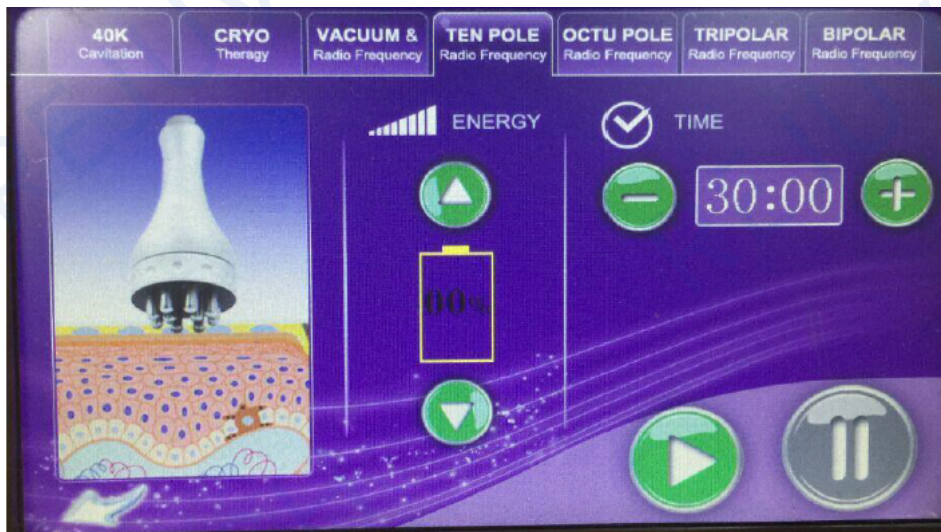

Select the appropriate treatment and install the corresponding handle.

Handle: 10 Polar RF Head


Treatment: TEN POLE Radio Frequency

Proceed with the treatment.

1. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability). We suggest to try for the proper energy level on the back of clients' hands before using them on the body.
2. Set the TIME. Press "START" key to begin the treatment.
3. Apply gel to the negative pressure vacuum RF head, point it to the position for losing weight and operate in pulling technique (drag fat at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.
4. Spread gel or essence on the treated positions evenly.
5. You can adjust the energy level according to customer's acceptable level.
6. Use the TEN POLE Radio Frequency Head for 15-20 minutes with mainly the spiral movement on body.

V. TREATMENT SELECTION

V.5 8 Polar RF

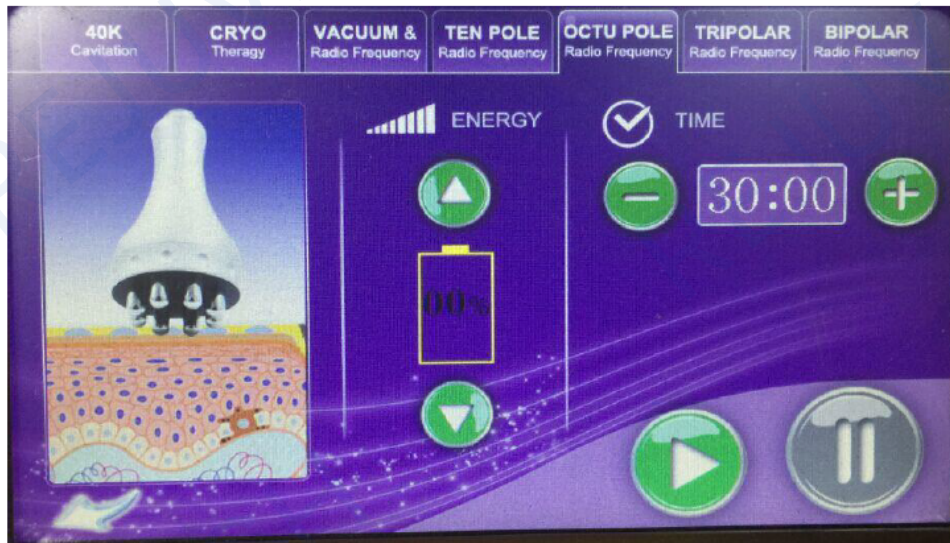

Select the appropriate treatment and install the corresponding handle.

Handle: 8 Polar RF Head


Treatment: OCTU POLE Radio Frequency

Proceed with the treatment.

1. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability). We suggest to try for the proper energy level on the back of clients' hands before using them on the body.
2. Set the TIME. Press "START" key to begin the treatment.
3. Apply gel to the negative pressure vacuum RF head, point it to the position for losing weight and operate in pulling technique (drag fat at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.
4. Spread gel or essence on the treated positions evenly.
5. You can adjust the energy level according to customer's acceptable level.
6. Use the 8 Polar RF Head for 15-20 minutes with mainly the spiral movement on body.

V. TREATMENT SELECTION

V.6 Tripolar RF

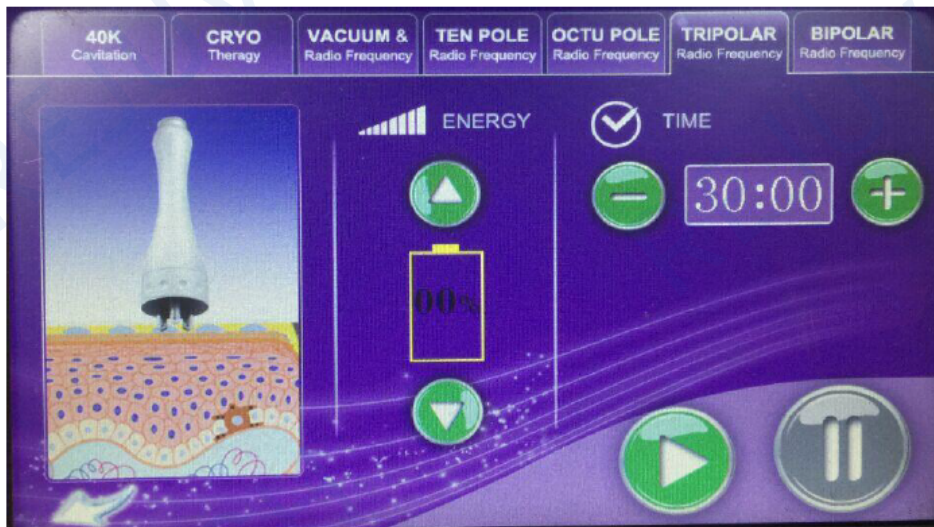

Select the appropriate treatment and install the corresponding handle.

Handle: Tri Polar RF for Face


Treatment: TRIPOLAR Radio Frequency

Proceed with the treatment.

1. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability). We suggest to try for the proper energy level on the back of clients' hands before using them on the body.
2. Set the TIME. Press "START" key to begin the treatment.
3. Apply gel to the negative pressure vacuum RF head, point it to the position for losing weight and operate in pulling technique (drag fat at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.
4. Spread gel or essence on the treated positions evenly.
5. You can adjust the energy level according to customer's acceptable level.
6. Use the Tri Polar RF for Face Head for 15-20 minutes with mainly the spiral movement on body.

V. TREATMENT SELECTION

V.7 Bipolar RF for Face and Around the Eye

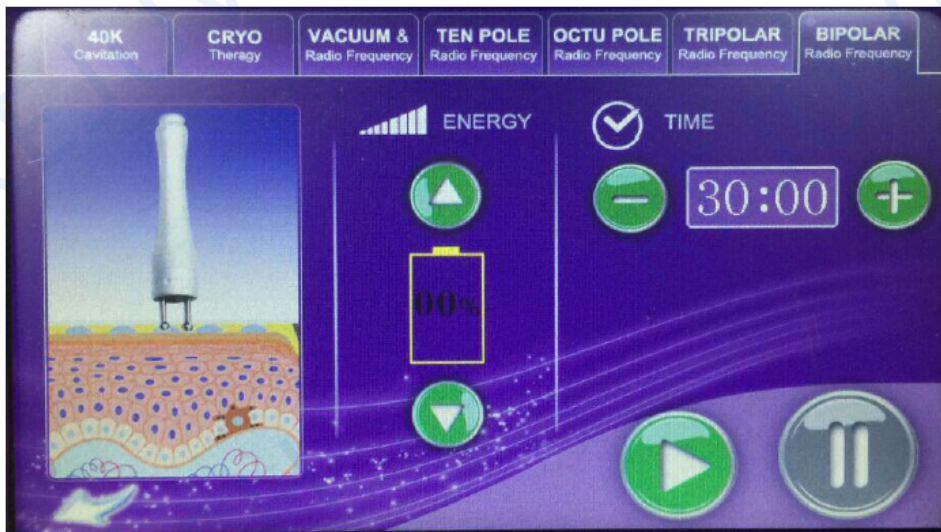

Select the appropriate treatment and install the corresponding handle.

Handle: Bipolar RF for Face and Around the Eye Head


Treatment: BIPOLAR Radio Frequency

Proceed with the treatment.

1. Select the suitable ENERGY LEVEL (start at 10% and adjust according to customer's tolerability). We suggest to try for the proper energy level on the back of clients' hands before using them on the body.
2. Set the TIME. Press "START" key to begin the treatment.
3. Spread gel or wrinkle smoothing essence on the treatment area (generally 20-30G each time), use the Bipolar RF for Face and Around the Eye head for 15-20 minutes with mainly the spiral movement on face and eye around.
4. You can adjust the energy level according to customer's acceptable level.

V. TREATMENT SELECTION

V.8 Laser Lipolysis

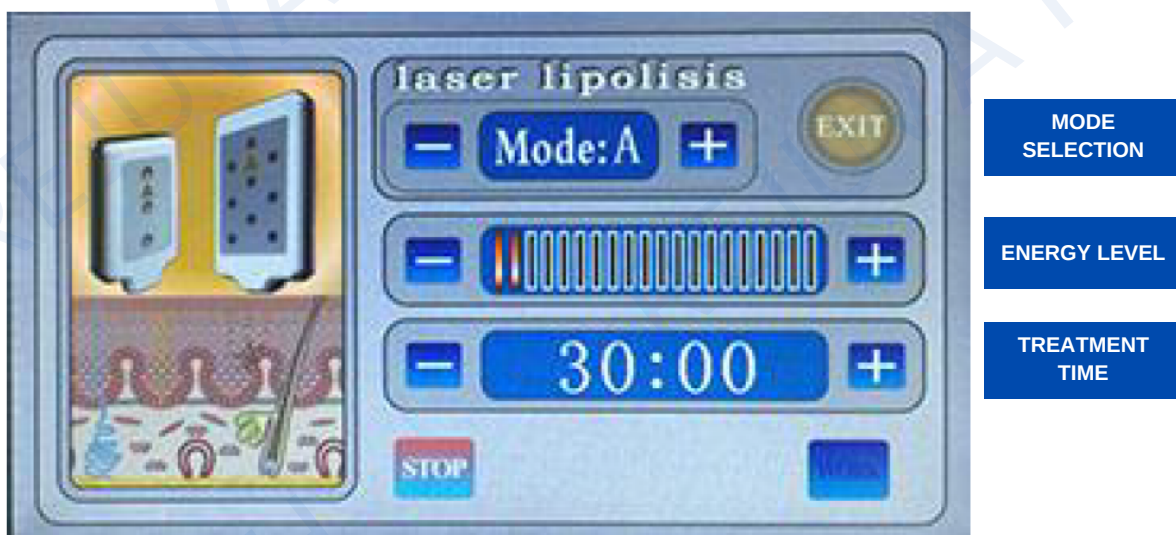

Select the appropriate treatment and install the corresponding handle.

Handle: Laser Positioning Plates

Treatment: Laser Lipolysis

Proceed with the treatment.

1. The Laser Positioning Plates can adopt light to penetrate skin deep level to stimulate metabolism to achieve slimming effects.
2. Simply attach the laser positioning plates using the band on the treatment area.
3. The suggested treatment time is 20 minutes.

There is 4 MODE of A, B, C and D as described as follows:

"A" is continuous working for 3 sec and discontinuous for 3 sec.


"B" is continuous working for 2 sec and discontinuous for 2 sec.

"C" is continuous working for 1 sec and discontinuous for 1 sec.


"D" is continuous working for 0.5 sec and discontinuous for 0.5 sec.

VI. RECOMMENDED TREATMENT POSITION

1. Treatment position for 40 KHz Cavitation Head.
Massage the fat of thigh in circles.

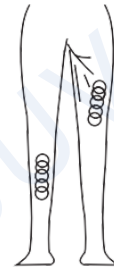

2. Treatment position for RF Head
Waist and abdomen area (25 minutes, 2-3 days/time)

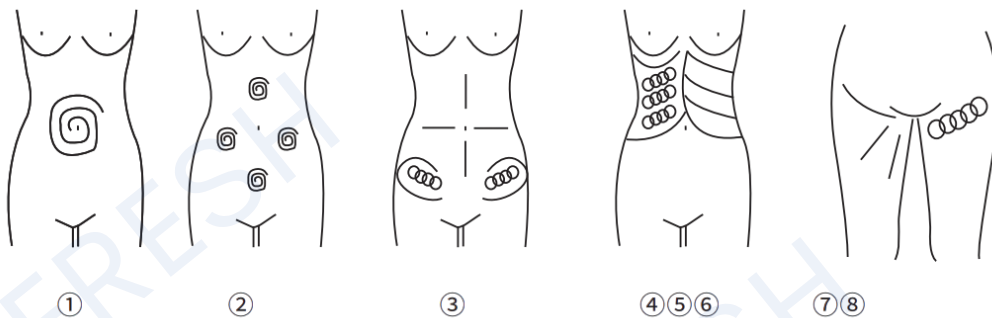

- ①. Massage to navel as center and gradually expanding the range
- ②. Massage chest nest, both sides of former abdomen and below the navel (in diamond formation) from inside to outside in circles.
- ③. Concentrated on the fat from 4 directions the navel as center
- ④. Massage from inner rib to outer edge in circles.
- ⑤. Massage from end of pelvis to groin in circles.
- ⑥. Massage back and forth from inner rib to outer edge in straight line.
- ⑦. Lift the Hip and the lower part of the body care (30minutes, 2-3 days/time) use the 50mm probe
- ⑧. Concentrated from hip to groin in circles or straight line. Repeat massaging from inner to outer edge. Repeat the motion.

VII. RECOMMENDED TREATMENT POSITION

3. Treatment position for Three Polar RF Head for Face and Eyes.

1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw lines.
3. Massage the triangle zone of both sides of the face in circles.



4. Massage from jaw to angulus oris to ear in lines.





REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
1-800-249-1310
rejuvafresh.com