

Beauty Disruptor Microneedling Machine 3 in 1

USER MANUAL



Welcome

to your **Beauty Disruptor Microneedling Machine 3 in 1**



REJUVA
FRESH

TABLE OF CONTENTS

PREFACE

I. PRODUCT INFORMATION & SPECIFICATION	2-5
II. PRODUCT APPEARANCE	6-7
III. TREATMENT SETTINGS	
a. Acne Removal Handpiece	8
b. Golden RF Microneedle Handpiece	9-11
c. Cold Hammer Handpiece	12
IV. SOFTWARE INTERFACE	
a. Acne Removal	13
b. Golden RF Microneedle	14-15
c. Cold Hammer	16

This manual is a property of Rejuva Fresh.
It is not allowed to be copied, reproduced, distributed, published or modified in any way nor be distributed any part of this content over any network.



PREFACE

Thank you for choosing **REJUVA FRESH** Beauty Disruptor Microneedling Machine 3 in 1. A state-of-the-art machine that uses a powerful combination of 3 technologies, which work in harmony to provide a truly comprehensive skin anti-aging solution.

Enjoy the instant luxurious benefits of this top-tier treatment. The non-invasive procedure effectively reduces all sorts of scars and stretch marks, as well as achieving long-term skin tightening and wrinkle reduction.

The treatment with our machine is safe and highly-effective, helping your clients achieve the youthful, smooth skin they deserve.

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com.

Thank you again, and please enjoy your Disruptor Microneedling machine 3 in 1!

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

This machine combines three technologies:

Gold RF Fractional Microneedling:

This highly effective skin care treatment uses a unique blend of 2 clinically proven technologies - gold-tipped microneedling + fractional radio frequency (RF) - to effectively stimulate collagen production and improve skin texture.

Gold-plated microneedles are far finer than human hair, penetrating the skin to create tiny channels that trigger the body's natural repair response. This is then augmented by radiofrequency energy, which heats the deeper tissue layers, encouraging collagen production and skin tightening.

Gold RF Microneedling can be used on all skin types and colors regardless of their individual characteristics. As a result, it is an ideal skin rejuvenation treatment for individuals of all backgrounds.

Cooling Ice Hammer:

The ice hammer treatment is the perfect technique for calming and reducing any skin inflammation or redness that may occur post microneedling. Its cold temperature helps constrict tissue and reduce blood flow to the affected area.

The hand held ice hammer probe is easily moved up and down on the skin. The cold energy instantly restricts blood flow to the area, creating a youthful effect that can last up to 8 hours.

Acne Removal Pen:

The mono-polar RF energy delivered by the Acne Removal Pen reduces oil production that causes acne. Sebaceous glands are targeted and destroyed, leaving your skin feeling clean and clear. Enjoy the long-term benefits of a clearer complexion without any harsh chemicals.

The needles in the Acne Removal Pen are thermally insulated, keeping the skin safe and ensuring scar-free, burn-free treatments.

I. PRODUCT INFORMATION & SPECIFICATION

I.2 Product Specification





Power	250 W
Voltage	100 V/ 110 V/ 220 V Voltage selected according to your region
Screen Size	12 in
Cold Hammer Temperature	0-5 °C
Radio Frequency (RF)	2 MHz
RF Output Power	10-200 W
Needle Cartridges	10 pin, 25 pin, 64 pin and nano needle
Package Dimension	122 x 68 x 60 cm

NOTE: This machine is not advisable to be used if:

- a. pregnant,**
- b. with artificial heart pacemakers,**
- c. metal built-in heads.**

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Description	Quantity	Image
Body	1 set	
Golden RF microneedle handpiece	1	
Cold hammer handpiece	1	
Acne removal handpiece	1	


I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Description	Quantity	Image
Needle Cartridges (Replacement Cartridges are purchased separately.)	4 one each of 10 Pin, 25 Pin, 64 Pin, Nano Pin	
Negative Ion Plate	1	
Foot Switch	1	
Power Cord	1	

II. PRODUCT APPEARANCE

II.1 Front view

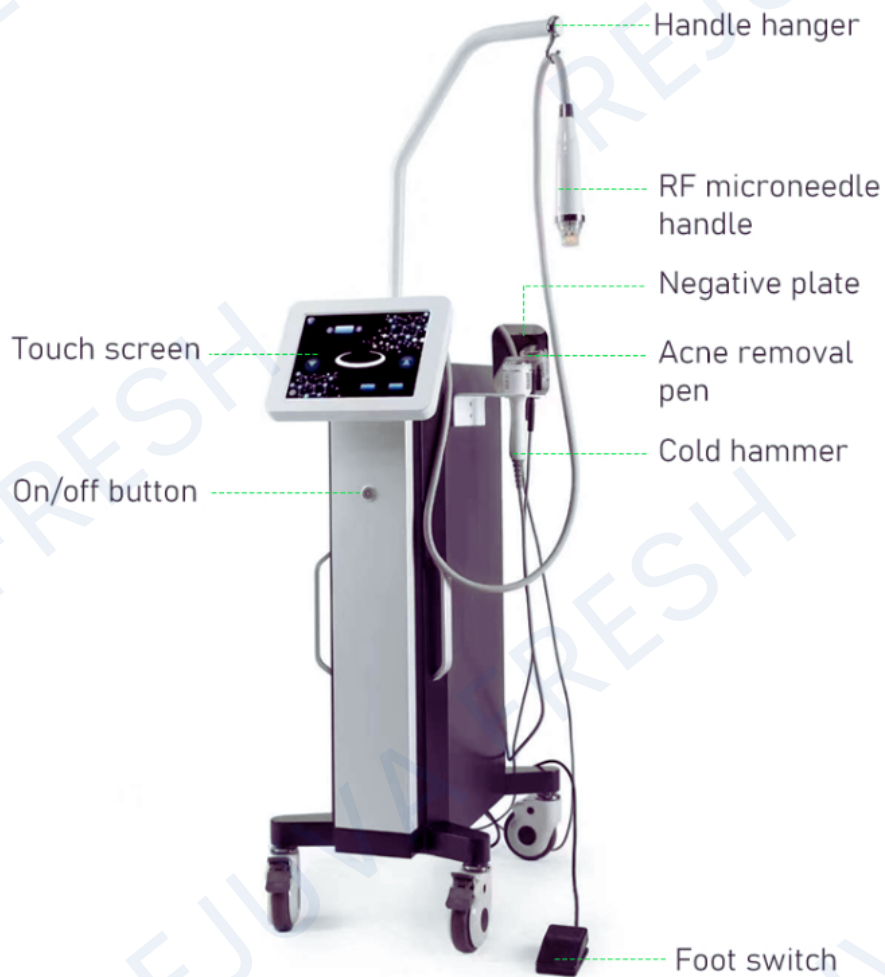

II. PRODUCT APPEARANCE

II.2 Side view




III. TREATMENT SETTINGS

a. Acne Removal Handpiece

III.1 How does this work?

a. Acne Removal Handpiece

When the sebaceous gland under the skin becomes clogged, it results in a common skin condition called acne. In order to prevent acne from re-occurring, it is best to remove the sebaceous glands. The Acne Removal Handpiece use tiny insulated RF needles to prevent burns on the epidermis and only transmit high frequency energy to the sebaceous glands. It is a safe and effective treatment. The figure below shows hoe this handpiece remove the sebaceous glands.

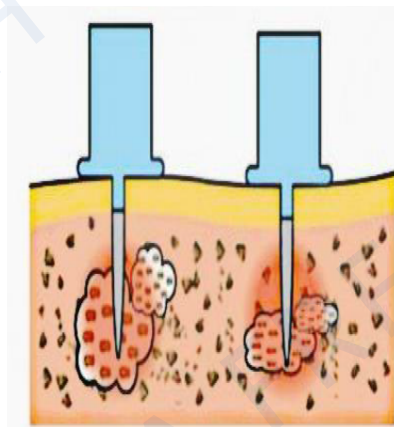


Skin Condition	Energy	Speed	No. of Treatment
Acne	90~110 W	0.4 s	3-6 Times, once every 2 weeks
Syringoma	90~110 W	0.4 s	2-3 Times, once every 2 weeks
Blackhead/ Large pores	90~110 W	0.4 s	3-5 Times, once every 2 weeks

III. TREATMENT SETTINGS

b. Golden RF Microneedle Handpiece

b. Golden RF Microneedle Handpiece

Micro Needle RF system is an ideal technology for skin rejuvenation by applying precisely controlled RF energy directly into a certain depth of the layer of the skin with minimally invasive micro-needles. Such an ideal combination of micro-needling plus RF energy reduces treatment time and recovery time significantly differentiating from fractional laser-based treatment. The exact precision and energy delivery enables tailored treatments for various skin concerns on all skin types- even dark skin. The secret is in the gold-plated micro-needles!

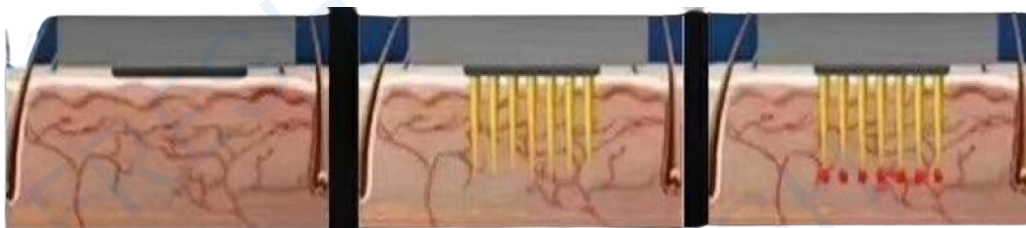

Golden RF
Microneedle skin
contact.

Golden RF
Microneedle
penetrates the skin.

Golden RF
Microneedle launches
high RF.

III. TREATMENT SETTINGS

b. Golden RF Microneedle Handpiece

Settings per treatment area:

Treatment Area	Depth	Duration	Suction Level	RF
Forehead	0.5 - 1.0 mm	0.2 - 0.3 s	1 - 2	60 - 110 W
Face	1.0 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Area around eye	0.5 - 1.0 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Chin	0.5 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Nose	0.5 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Neck	1.0 - 1.5 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Pox Pit	1.0 - 2.0 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Stretch Marks	1.0 - 2.0 mm	0.2 - 0.3 s	1 - 2	60 - 120 W
Belly	1.0 - 3.0 mm	0.2 - 0.3 s	0 - 1	60- 120 W

III. TREATMENT SETTINGS

b. Golden RF Microneedle Handpiece

Needle Cartridges:



NOTE: The needle cartridges are disposable. Additional cartridges are purchased separately. Disinfect with saline before use.

Microneedling target areas and function:

1. Wrinkles: crow's feet, head lines, smiles, neck lines.
2. Enhance face surface.
3. Stretch marks, swelling pattern.
4. Large pores.
5. Dull skin.
6. Red blood circulation.
7. Skin whitening.
8. Acne scars.
9. Sagging scars.
10. Regular skin care routine.

III. TREATMENT SETTINGS

c. Cold Hammer Handpiece

c. Cold Hammer Handpiece

The use of ice directly on the wound to reduce the pain and swelling can be seen in history as this was used by the Egyptians and was often called the method of "cryotherapy". There are many ways to achieve this method, such as the use of dry ice, ice pack, liquid air, and so on. On this machine, this is done by the cold hammer handpiece. It releases ice power that results in tightening and improving the size of the pores. This also results in reduced skin redness, swelling, and pain especially for allergic skin.

IV. SOFTWARE INTERFACE

a. Acne Removal



1. Clean the are of the skin before the start of treatment.
2. Select "I "for Acne Removal.
3. **Remove all metal or any jewelry from the customer.**
4. Put the negative plate in contact with the skin preferably on the back.
5. Set the Operation Time. This is different according to area being treated but generally takes 20-25 minutes. Treatment "Times" in one area is 1-2 times.

NOTE: After the microneedle is completely inserted into the skin, step on the foot switch (to avoid burns)


6. For speed and energy, refer to the table in the Treatment Settings on page 8.
7. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.

POST TREATMENT MEASURES:

1. DO NOT touch the area affected by the treatment to prevent infection.
2. Apply cream on the affected area.
3. After Treatment, there will be a slight swelling and burning sensation, which is due to the use of high-frequency waves in the sebaceous gland part, which is a normal phenomenon. It will be relieved within two days. It will gradually disappear with time.

IV. SOFTWARE INTERFACE

b. Golden RF Microneedle

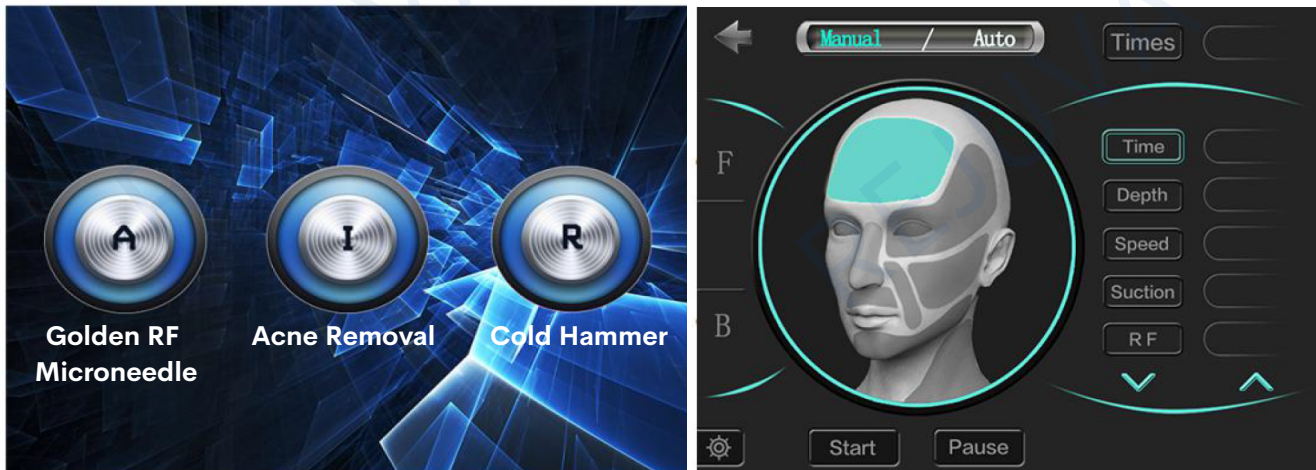

1. Clean the are of the skin before the start of treatment.
2. Apply anesthesia on the area being treated. Wrap with plastic wrap to improve the effect of the anesthesia.
3. Select "A "for Golden RF Microneedle.
4. Select "F" for Face and "B" for body, depending on the area being treated.
5. This has two modes: Manual and Auto. Select "Auto" to release the needle automatically.
6. Set the Operation Time. This is different according to area being treated but generally takes 20-25 minutes. Treatment "Times" in one area is 1-2 times.
7. For Depth, Speed, Suction, and RF , refer to the table in the Treatment Settings on page 9.
8. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.

IV. SOFTWARE INTERFACE

b. Golden RF Microneedle

POST TREATMENT MEASURES:

1. Aftercare, in order to repair the skin, put moisturizing and repair products twice daily.
Recommended to be done in the morning and evening.
2. Let the wound scabs naturally fall. DO NOT peel the wound scabs.
3. Avoid direct sunlight on the treated area, apply sunscreen.
4. Avoid intense exercise, swimming and sauna while healing is taking place.
5. Avoid alcohol intake for a short period of time while healing is taking place.
6. Treated area must be kept dry for 24 hours.
7. May encounter fever or redness on the treated area a day after the treatment.
8. After 3-4 days, scabs will naturally fall off.

IV. SOFTWARE INTERFACE

c. Cold Hammer



1. Select "R "for Golden RF Microneedle.
2. Set the Operation Time. This is different according to area being treated but generally takes 20-25 minutes. Treatment "Times" in one area is 1-2 times.
3. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.



REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
1-800-249-1310
rejuvafresh.com