

Eris Lipo Cavitation Machine 9 in 1 Professional

USER MANUAL



Welcome

to your **Eris Lipo Cavitation Machine 9 in 1 Professional**



REJUVA FRESH
REJUVA FRESH
REJUVA FRESH
REJUVA FRESH
REJUVA FRESH
REJUVA FRESH

TABLE OF CONTENTS

PREFACE	
I. PRODUCT INFORMATION & SPECIFICATION	2-5
II. PRODUCT APPEARANCE	6
III. WARNINGS	7-8
IV. RECOMMENDATIONS	9
V. TREATMENT SELECTION	
V.1 CAVI	10-11
V.2 VAC RF	12-13
V.3 EYE RF	14-15
V.4 FACE RF	16-17
V.5 BODY RF	18-19
V.6 LASER	20-21
V.7 COLD	22
V.8 EMS	23-24
V.9 BIO&LED	25-26
VI. RECOMMENDED TREATMENT POSITION	27-28

PREFACE

Thank you for choosing REJUVA FRESH Eris Lipo Cavitation Machine 9 in 1 Professional. The Eris Lipo Cavitation System is an advanced beauty instrument designed to deliver remarkable effects in weight loss, fat dissolution, and skin tightening.

The strong 20KHz ultrasonic waves blast medium and thicker fat while the negative pressure radio frequency boosts metabolism and lymphatic drainage. Powerful 160mW lipo laser infrared tech helps dissolve even the most stubborn fat.

The use of this machine is safe, minimally-invasive procedure, and easy to operate. There are no bleedings, no pain, and side effects.

Rejuva Fresh is the developer of the groundbreaking 20K lipo laser treatment, and our brand is one of the most popular and best selling in the global spa industry!

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com.

Thank you again, and please enjoy your cavitation equipment!



This spa machine includes a personal training session with an expert instructor. We also offer complimentary certification (certificate that you can hang on the wall).

Contact us after you receive your delivery to schedule your training session by Zoom or FaceTime.

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

- **Powerful fat reduction and slimming effects** - the machine uses the power of concentrated sound waves to liquify fat cells and facilitate effective fat removal. Your body's own natural process is harnessed to blast and bump adipose cells, resulting in visible and lasting fat loss.
- **Lymphatic detoxification** - advanced vacuum radio frequency technology activates adipocyte tissue, which results in increased sweat gland, liver, intestinal and lymph circulation. This leads to the body's natural elimination of excess fat and toxins.
- **Skin tightening & improved skin elasticity** - the radio frequency technology effectively heat the tissues below the skin's surface, stimulating collagen and elastin production for reduced wrinkles and skin laxity. Target temperatures for dermal remodeling are delivered for efficient and long-lasting results.
- **Cellulite reduction** - advanced physical electron technology generates biological waves at targeted acupoints, helping improve orange peel tissue and slim the body shape.

I.2 Product Specification

Machine Power	400 W
Cavitation Frequency	20KHz (also available in 30K and 80K)
Cavitation Power	160 W
Input Voltage	100 V/ 110 V/ 220 V Voltage selected according to your region
Radio Frequency(RF)	Body: 1 MHz Facial: 5 MHz
Lipo Laser Wavelength	635nm~650nm
Lipo Laser Power	160mW per diode
Vacuum Pressure	-20KPA
Cold Hammer Temperature	0-5 °C

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Description	Qty.	Image
Main Unit	1 set	
6 Polar RF Head	1	
Ultrasonic Cavitation Head	1	
Vacuum Bipolar RF Head	1	

I. PRODUCT INFORMATION & SPECIFICATION

Description	Qty.	Image
3 Polar RF Head for Face and Eyes	1	
4 Polar RF Head for Face and Body	1	
Ice Hammer	1	
Laser Positioning Plates	6 Large 2 Small	

I. PRODUCT INFORMATION & SPECIFICATION

Description	Qty.	Image
Microcurrent Photon Therapy	1	 A handheld, silver-colored device with a circular array of small electrodes on the front and a black cable extending from the back.
EMS Pads	1 set	 Two black, rectangular adhesive pads connected by a grey cable with red and black connectors.
Power Cord	1	 A black power cord with a standard three-prong electrical plug on one end and a smaller connector on the other.

II. PRODUCT APPEARANCE

II.1 Front view



III. WARNINGS

III.1 Please follow these precautions before/during/after use



DO NOT use if pregnant,
menstruating, conceiving, or
breastfeeding



DO NOT use if epileptic



DO NOT use if there are open wounds
or any skin infections such as eczema or dermatology issues



DO NOT use if there are heart conditions (such as with
arrhythmias) especially those with pacemakers

III. WARNINGS



DO NOT use if with kidney disease



DO NOT use if have metal on any part of the body



Before using the instrument,
remove any metal jewelry you are wearing

IV. RECOMMENDATIONS

1. Be sure to use the ultrasonic special gel.

2. Handle the heads with care, avoid knocks.

3. Use and prepare enough gel to avoid overheating the heads.

4. Don't linger too long on a single region and avoid the bones.

5. Don't use disinfectants to disinfect the heads, use wet cotton or dry towel instead.

6. Make sure the power supply is connected and plugged well.

7. If the machine is not for use for a long period of time, switch it off and unplug the power supply.

8. The machine should be rested every hour of continuous operation for about 10 minutes.

V. TREATMENT SELECTION

V.1 CAVI

Strong sound wave explosion fat head - with collective strong sound wave head, strong sound wave of 20KHZ may be emitted to human body for impacting fat cells fiercely and causing friction motion between fat cells. This may cause effective consumption of calories and moisture in fat cells and reduce the size of fat cell. What's more, sound wave vibration may cause fierce impact of fat cells to make them be exploded instantaneously, reduce the amount of fat cells and thereby achieve the effects of removing fat.



Select the appropriate treatment and install the corresponding handle.

Handle: 20 KHz Cavitation Head

Treatment: CAVI

Press "-" "+" to adjust the TIME of operation.

Select the CAVI MODE among M1,M2,M3,M4 modes.

Press "-" to decrease the CAVI POWER or click "+" to increase it.

Press "START" to start the strong sound wave explosion fat head.

There is 4 MODE of M1, M2, M3 and M4 as described as follows:

"M1" is continuous working for 3 sec and discontinuous for 3 sec.

"M2" is continuous working for 2 sec and discontinuous for 2 sec.

"M3" is continuous working for 1 sec and discontinuous for 1 sec.

"M4" is continuous working for 0.5 sec and discontinuous for 0.5 sec.

Operate with the 20KHz cavitation head for 15-20 minutes with mainly the spiral movement on body

V. TREATMENT SELECTION

V.1 CAVI

Method : Use the probe to massage the specific part of skin to be treated circularly after applying the slimming cream for about 15 to 20 minutes. It could quicken the blood circulation in the treated part of skin and has the effect of decomposing fat.


Matters needing attention:

1. Apply slimming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Do not keep the applicator at the same place
5. Move the applicator slowly but constantly.
6. Avoid the treatment over bone and do not aim directly to the joint
7. Pay attention to the temperature of the cavitation handle when using it, and stop using it for ten minutes after half an hour of use.
8. Cavitation is forbidden to work without gel. If it doesn't work and you need to re-adjust the frequency, please contact the supplier.

V. TREATMENT SELECTION

V.2 VAC RF

Dissolve fat, enhance lymphatic drainage, firm skin and improve skin elasticity. This treatment has an anti-geocentric attraction effect on the skin that is attracting and outspreading the fibrillar connective tissue of different cortexes through the rhythmic massage and effectively break down subcutaneous fat and reduce cellular tissue accumulation of fat. Coupled with the use of the patents methodology of massaging deeply with roller over different parts of the body to bring up fat layer in dermis curium and subcutaneous, it releases micro-vascular lymphatic vessels at the same time promoting metabolism and converts fat cells in a passive movement to fatty acids. Toxins within the cells are also eliminated from lymphatic system. In vitro the detoxification enhances the function as many as four times. The skin will become more elastic and luster. The perfect "healthy body sculpture management" expert.

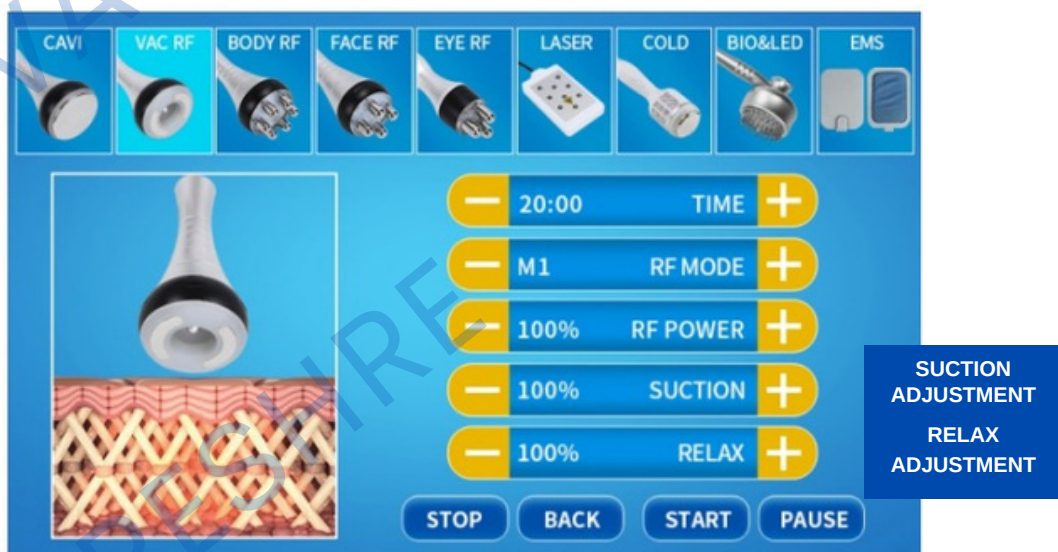

Select the appropriate treatment and install the corresponding handle.

Handle: Vacuum Bipolar RF Head

Treatment: VAC RF

Set the RF MODE : Adjust the RF MODE among Auto,M1,M2,M3 modes.

AUTO always output, **M1** output for 0.5 seconds, **M2** output for 1 second, **M3** output for 1.5 seconds.

Select the RF POWER (adjust according to the customer's acceptable level, 20%-60%is recommended).

Adjust the appropriate SUCTION(Adjust the knob on the back).

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.2 VAC RF

Method: After applying oil on the skin area to be treated, perform circulatory massage on specific areas of the skin for about 30 to 40 minutes, which can speed up blood circulation, It has the function of decomposing fat.


Matters needing attention:

1. Apply sufficient amount of oil.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Do not keep the applicator at the same place
5. Move the applicator slowly, but constantly.
6. Avoid the treatment over bone and do not aim directly to the joint.
7. Always make a skin fold in order to get better access to the fat tissue.
8. In case of resting during treatment, please "PAUSE" and then "STOP" to avoid over-output of the machine.
9. Wipe off the oil inside with alcohol/paper towel after use, then put it on the shelf.

V. TREATMENT SELECTION

V.3 EYE RF

Radio frequency is one kind of high frequency electromagnetic wave that can penetrate the dermis and subcutaneous tissue to heat collagen in the dermis layer. Collagen in the dermis layer will shrink immediately after the temperature rise and proliferation of new collagen, so as to achieve skin rejuvenation and tightening, pigment improvement etc. gradually eliminate wrinkles , restoring skin elasticity and facial improvement, prevent skin aging, and restoring skin elasticity .

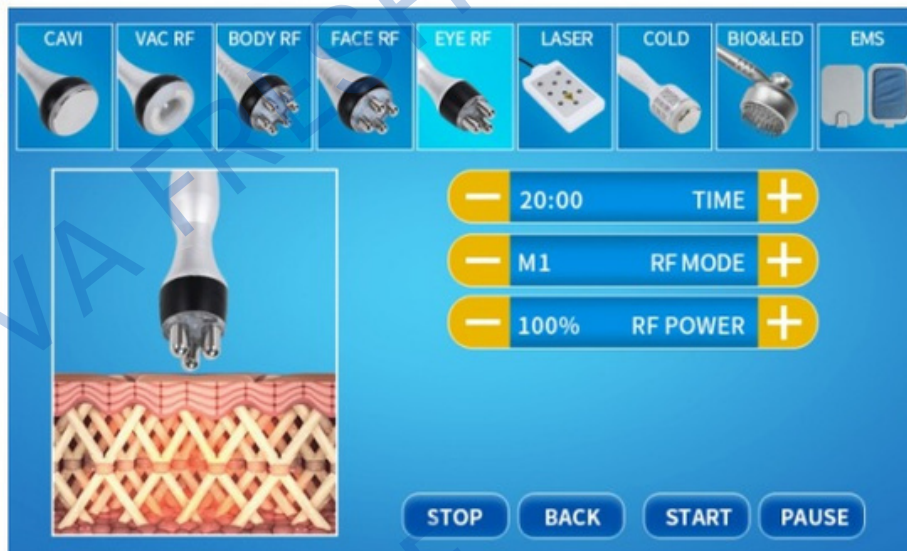

Select the appropriate treatment and install the corresponding handle.

Handle: Three Polar RF Head for Face and Eyes

Treatment: EYE RF

Set the RF MODE : Adjust the RF MODE among Auto, M1, M2, M3 modes.

AUTO always output, **M1** output for 0.5 seconds, **M2** output for 1 second, **M3** output for 1.5 seconds.

select the RF POWER (adjust according to the customer's acceptable level, 20%-60% is recommended).

Spread gel or essence on the treated area evenly.

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.3 EYE RF

Matters needing attention:

1. Spread gel or essence on the treated area evenly.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Do not keep the applicator at the same place.
5. Move the applicator slowly, but constantly.
6. Skin around eyes and forehead is very sensitive, the RF output intensity should be kept relatively low in such areas.
7. In case of resting during treatment, please "PAUSE" and then "STOP" to avoid over-output of the machine.

V. TREATMENT SELECTION

V.4 FACE RF

A capacitive coupling electrode is used for transmitting radio-wave energy and generating an electric field for entering into subcutaneous tissue through the skin surface. In the high frequency to the rapid change in electrode, directions of electrical particles in the skin are also changed. By this time, natural electric resistance in the subcutaneous tissue moves and generates heat energy. As the papillary dermis collagen may immediately shrink when the temperature is within the range from 60 to 70 Celsius degree. After wrinkle treatment, client may immediately sense the skin tightening effects as it is being lifted and firmed. When collagen is produced continuously, thickness and density of the skin papillary dermis may be increased to remove wrinkles, eliminate scars, restore skin elasticity and gloss and make it be blonde and smooth. While collagen is increased, fresh skin is generated in the position of treatment and wrinkles are removed in large amount. In addition, when cortex without elasticity or that with thick layer in the area with wrinkles is separated, surrounding skin will also be renewed.



Select the appropriate treatment and install the corresponding handle.

Handle: 4 Polar RF Head

Treatment: FACE RF

Set the RF MODE : Adjust the RF MODE among Auto,M1,M2,M3 modes.

AUTO always output, **M1** output for 0.5 seconds, **M2** output for 1 second, **M3** output for 1.5 seconds.

select the RF POWER (adjust according to the customer's acceptable level, 20%-60%is recommended).

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.4 FACE RF

Method: Operate with the facial RF head for 15-20 minutes with mainly spiral movement on face and around the eyes.

Matters needing attention:

1. Spread gel or essence on the treated area evenly.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Do not keep the applicator at the same place
5. Move the applicator slowly, but constantly.
6. Skin around eyes and forehead is very sensitive, the RF output intensity should be kept relatively low in such areas.
7. In case of resting during treatment, please "PAUSE" and then "STOP" to avoid over-output of the machine.

V. TREATMENT SELECTION

V.5 BODY RF

Being integrated with the advanced RF technology, the instrument can directly reach the deep-seated fat layer with the incisive positioning ability of RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enterohepatic circulation and lymph. Ultimately achieve the effect of dissolving fat. With controlled depth technology, The effects are very obvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areolar tissue, remove stubborn fat .



Select the appropriate treatment and install the corresponding handle.

Handle: 6 Polar RF Head

Treatment: BODY RF

Set the RF MODE : Adjust the RF MODE among Auto,M1,M2,M3 modes.

AUTO always output, **M1** output for 0.5 seconds, **M2** output for 1 second, **M3** output for 1.5 seconds.

select the RF POWER (adjust according to the customer's acceptable level, 20%-60%is recommended).

Spread gel on the treated area evenly.

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.5 BODY RF

Method: Drag fat areas at the position for losing weight to the position where it is closer to the lymph gland for about 20 minutes.

Matters needing attention:

1. Spread gel on the treated area evenly.
2. Start with low-energy. We suggest to test for the proper energy level to use on the back of clients before using on the to be treated areas.
3. Do not use essential oil as the medium.
4. In operation, start pulling only after negative pressure head is attached to skin.
5. Move the applicator slowly, but constantly.
6. In case of resting during treatment, please "PAUSE" and then "STOP" to avoid over-output of the machine.

V. TREATMENT SELECTION

V.6 LASER

Diode Lipo Laser emits low levels of laser energy which creates a chemical signal in the fat cell, breaking down the stored triglycerides into free fatty acids and glycerol and releasing them through channels in the cell membranes. The fatty acids and glycerol are then transported around the body to the tissues that will use them during metabolism to create energy. This process of fatty acid release is a natural response of the body when the body needs to use stored energy reserves, thus it will not create any unnatural reaction in the body nor does it cause any damage to surrounding structures such as skin, blood vessels and peripheral nerves. A period of exercise post treatment will ensure the complete metabolism and thus eliminate from the body the freed fatty acids. This 5mw lipo laser system can adopt light to penetrate deep level of skin to stimulate metabolism for effective slimming treatment. Tied laser board band at the site needing treatment and use laser for 20 minutes.



Select the appropriate treatment and install the corresponding handle.

Handle: Laser Positioning Plates

Treatment: LASER

Set the LASER MODE : Adjust the LASER MODE among M1,M2,M3,M4 modes.

"M1" works continuously, "M2" works continuously for 0.5 seconds, and pauses for 0.5 seconds. "M3" works continuously for 0.2 second and pauses for 0.2 second. "M4" works continuously for 0.1 seconds and pauses for 0.1 seconds.

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.6 LASER

Method: Tied laser board band at the site needing treatment and use laser for 20 minutes. It could quicken the blood circulation in the treated part of skin and has the effect of decomposing fat.

Matters needing attention:

1. Please do not radiate eyeball directly with colored light.

V. TREATMENT SELECTION

V.7 COLD

1. Help skin firmness, maintain elasticity, prevent allergies and other problems;
2. Shrink pores, lock water and reduce wrinkles;



Select the appropriate treatment and install the corresponding handle.

Handle: Ice Hammer Head

Treatment: COLD

Set the COLD mode : Adjust the TIME.

Adjust COLD POWER, depending on user's acceptable level.

Press "START" key to begin operation.

The indicator light will light up and start working.

V. TREATMENT SELECTION

V.8 EMS

It is a non-invasive slimming treatment technology that can burn fat while exercising. This is just like completing 30,000 sit-ups or squats in just 30 minutes, every 2 to 3 weeks, during 4 to 6 treatments, average fat decreases and muscle mass increases. The treatment area includes the abdomen, buttocks, arms and thighs. This is a safe and comfortable treatment.



The wiring is not divided into two levels, plus or minus. It must be inserted to the end, otherwise the connection will be poor.

Select the appropriate treatment and install the corresponding handle.

Handle: EMS Pads

Treatment: EMS

Set the EMS MODE : Adjust the TIME

Adjust the LASER MODE among M1,M2,M3,M4 modes.

"M1" works continuously, "M2" works continuously for 0.5 seconds, and pauses for 0.5 seconds. "M3" works continuously for 0.2 second and pauses for 0.2 second. "M4" works continuously for 0.1 seconds and pauses for 0.1 seconds.

Select the EMS POWER (adjust according to the customer's acceptable level, 20%-60% is recommended).

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.8 EMS

Method: Connect the electrode patch (regardless of positive and negative), tie the electrotherapy patch to the treatment area with a band to adjust the output energy and mode.

Matters needing attention:

1. People who have heart disease and have undergone heart surgery are prohibited from using EMS electrotherapy.
2. It is forbidden to stick electrotherapy patches around the heart.
3. When the skin is very dry and the conductivity is poor, you can apply some water on the skin. Do not exceed 30 minutes each time.
4. If you feel any uncomfortable feeling after using the product Please stop using it immediately and contact your local retailer/distributor.

V. TREATMENT SELECTION

V.9 BIO & LED

Microcurrent-safe and painless facial helps erase fine lines and wrinkles, while firming your skin and defining your features



Select the appropriate treatment and install the corresponding handle.

Handle: Photo Rejuvenation Head

Treatment: BIO&LED

Set the BIO MODE : Adjust the TIME

Adjust the BIO MODE among M1,M2,M3,M4 modes.

"M1" works continuously, "M2" works continuously for 0.5 seconds, and pauses for 0.5 seconds. "M3" works continuously for 0.2 second and pauses for 0.2 second. "M4" works continuously for 0.1 seconds and pauses for 0.1 seconds.

Select the BIO POWER (adjust according to the customer's acceptable level, 20%-60% is recommended).

The handle can adjust 7 photon modes. First, turn on the switch on the machine screen to ensure that the power is greater than 0, then adjust the LED COLOUR. (LED COLOUR meaning on the next page).

Press "START" key to begin operation.

V. TREATMENT SELECTION

V.9 BIO & LED


Method: : Use the probe to massage the specific part of skin to be treated circularly after applying the essence adjust the output energy and mode.

Matters needing attention:


1. **Red:** Vitalize be suitable for any skin, promote blood circulation, and regenerate collagen protein in order to smooth skin and heal xerodermia, exuviation, degenerate losing hair and folliculosis.
2. **Green:** Balance Normalize sebaceous glands, educe black heads and sores, balance moisture.
3. **Blue:** Calm be suitable for sensitive, oil, and comedo skin-diminish and eliminate acne bacilus to remove acne, whelk, promote to compound protein with ossein, activate skin, tighten slack skin and gravidity winkle.
4. **Purple:** Drain Renew skin, reduce wrinkles.
5. **Orange:** Brighten whiten decompose pigment, and smooth scares.
6. **Cyan:** Defense strengthen the ability to protect skin, calm down, release stress.
7. **White:** Guide guide skin to absorb nutrients and anti-aging.
8. Loop

VI. RECOMMENDED TREATMENT POSITION

1. Treatment position for 40 KHz Cavitation Head. Massage the fat of thigh in circles.

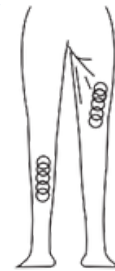

2. Treatment position for RF Head
Waist and abdomen area (25 minutes, 2-3 days/time)

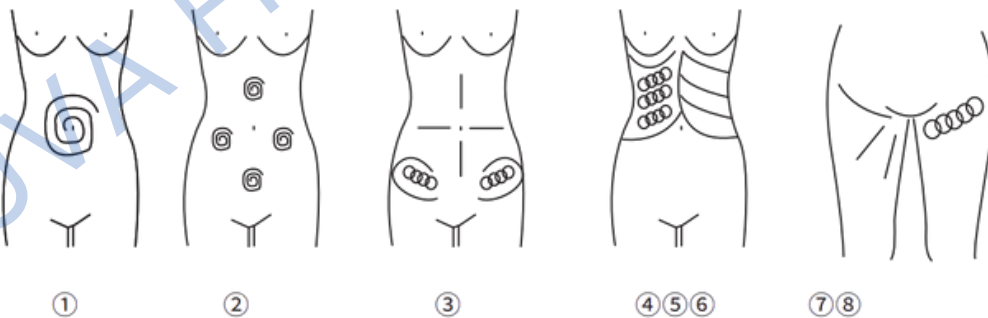

- ①. Massage to navel as center and gradually expanding the range
- ②. Massage chest nest, both sides of former abdomen and below the navel (in diamond formation) from inside to outside in circles.
- ③. Concentrated on the fat from 4 directions the navel as center
- ④. Massage from inner rib to outer edge in circles.
- ⑤. Massage from end of pelvis to groin in circles.
- ⑥. Massage back and forth from inner rib to outer edge in straight line.
- ⑦. Lift the Hip and the lower part of the body care (30minutes, 2-3 days/time) use the 50mm probe
- ⑧. Concentrated from hip to groin in circles or straight line. Repeat massaging from inner to outer edge. Repeat the motion.

VII. RECOMMENDED TREATMENT POSITION

3. Treatment position for Three Polar RF Head for Face and Eyes.

1. Massage the mandible center in circles to produce deep heat.
2. Massage lower jaw lines.
3. Massage the triangle zone of both sides of the face in circles.



4. Massage from jaw to angulus oris to ear in lines.





REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
1-800-249-1310
rejuvafresh.com