

Luxury Portable Diode Laser Hair Removal Machine (A & B)

USER MANUAL



Welcome

to your **Luxury Portable Diode Laser Hair Removal
Machine**



TABLE OF CONTENTS

PREFACE_____	1
I. PRODUCT INFORMATION & SPECIFICATION_____	2-3
II. PRODUCT APPEARANCE_____	4
III. WARNINGS_____	5-6
IV. RECOMMENDATIONS_____	7
V. MACHINE OPERATION	
V.1 Machine Assembly_____	8-9
V.2 Software Operation_____	10-11
VI. MACHINE PARAMETER SETTING_____	12
VII. PRE AND POST TREATMENT CARE_____	13
VIII. FREQUENTLY ASKED QUESTIONS_____	14-15
IX. MAINTENANCE_____	16

PREFACE

Thank you for choosing REJUVA FRESH Luxury Portable Diode Laser Hair Removal Machine. This triple wavelength diode laser machine works flawlessly on all skin and hair colors. The high power handle lets you achieve fast results on all areas of the body, including face, arms, armpits, chest, back, bikini, legs and etc.

REJUVA FRESH Luxury Portable Diode Laser Hair Removal Machine is relatively comfortable and guarantees a painless treatment. This high power machine and handle gives you superior treatment effect. Equipped with a 10.4 in. touch screen, an interface with 8 language options, and 20 millions shots life that ensures long time usage.

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

As the laser penetrates the skin to the hair follicle, the hair follicle stem cells break down and this coagulation necrosis results in hair falling off naturally. Thus, achieving the effect of permanent hair removal.

This machine uses three wavelengths to increase effectiveness. These wavelengths are 755nm, 1064nm, and 808nm. The treatment range for 755nm and 1064nm are: hair in various parts of the body such as: the hairline of the face, lip hair, beard and body hair, chest hair, limb hair, bikini hair, etc.

808nm wavelength causes temperature to rise sharply, destroying the hair follicle and its surrounding tissues, thereby destroying the hair growth center on which the hair grows.

I.2 Product Specification

Maximum Power Output	2000 W
Input Voltage	110 V/ 220 V Voltage selected according to your region
Adjustable Light Outlet	10 mm x 10 mm / 15 mm x 15 mm 20 mm x 20 mm
Skin Cooling Temperature	-10°C ~ 10°C (by cooled Sapphire)
Repetition Frequency	1 ~ 10 Hz
Energy	1 ~ 120 J/cm ²
Transport Package	Air Box
Language Selection	English, Spanish, Russian, German Portuguese Arabic, French, Chinese

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories

Item	Quantity	Image
Body	1	 or
Key	1	
Water Pipe	1	
Water Funnel	1	
Operator Protective Glasses	1	
Power Connection Cable	1	

II. PRODUCT APPEARANCE



MACHINE A



MACHINE B

III. WARNINGS



The user should add water after receiving the machine. Make sure the tank is filled with **DISTILLED WATER**. **MINERAL OR TAP WATER IS PROHIBITED.** The water inside the machine must be changed weekly.



It is not recommended to use if there is a history of keloid, allergies, and scarring. It is also not recommended if suffering from hepatitis, syphilis, skin infection, open wounds, infected wounds, skin diseases, and epilepsy.



DO NOT point directly into the eye the laser light or look at it directly as it is dangerous.



During treatment, the operator must wear protective goggles to protect the operator from direct exposure to the eye.



Only the people who are professionally trained can use the device.

III. WARNINGS




DO NOT use if with infectious disease.



DO NOT use if there are heart conditions (such as with arrhythmias) especially those with pacemakers.



DO NOT use if pregnant, menstruating, or conceiving.

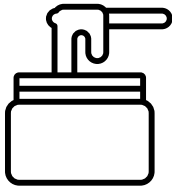

Not recommended for those who had used other methods (such as wax removal) for the past 6 weeks.



Not recommended for those who have recently received sun exposure such as sun bathing.

IV. RECOMMENDATIONS

1. During treatment, if it is painful, stop and check whether the patients' skin has changed. Adjust to the energy that a patient can tolerate.

2. Skin color is very important. For white to light color, the power can be adjusted to 80~100% to achieve good results. For light color to brown, the power can be lowered (an example is 30~60%).


3. Before treatment, the skin hair must be shaved off. Frequency can be raised to protect the skin from burns. For dark skin, extend the treatment time to expose enough energy into the skin and hair follicles.

V. MACHINE OPERATION


V.1 Machine Assembly

Machine Assembly

1. Assemble the handle onto the machine. (a) Align the joints first by the wire column. (b) Press the red buttons on both sides to insert the connector. (c) Finally, buckle the upper and lower locks.

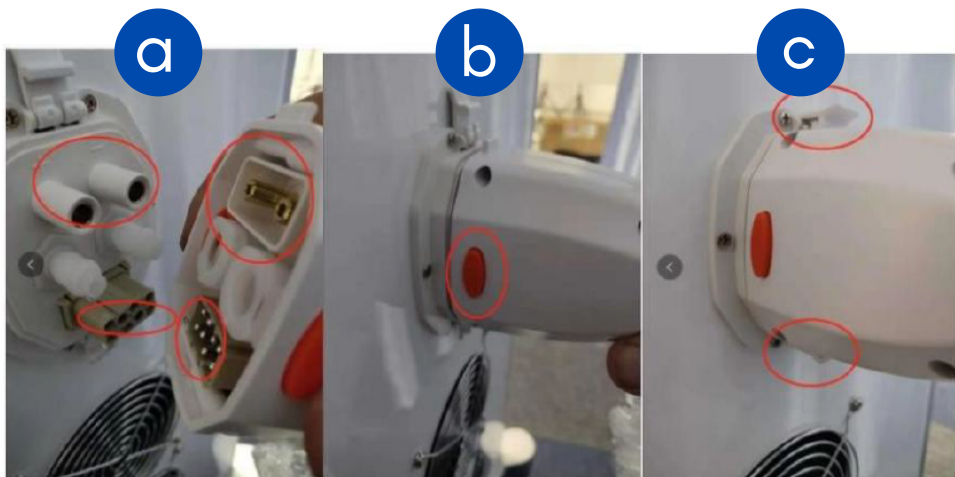

2. Put distilled water from the **injection port** until there is water flowing out from the **overflow port**. The room temperature where the machine will be used is controlled at 5~28 degrees C to prevent water from freezing.

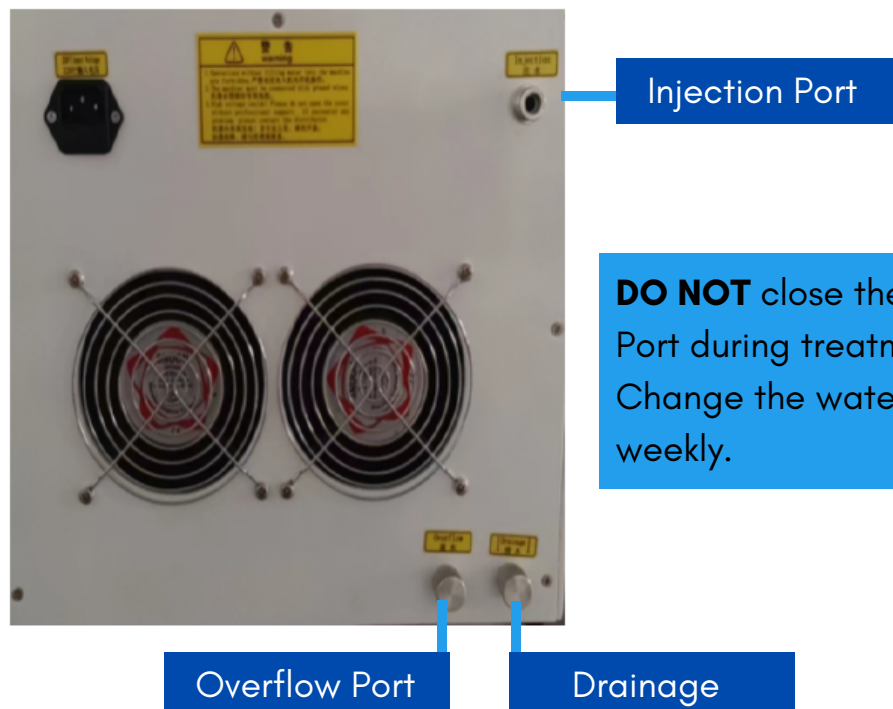

V. MACHINE OPERATION

V.1 Machine Assembly

3. Insert the power cord with machine power port. Connect to power supply.



4. Rotate the Emergency Stop switch clockwise to making it bounce up.




5. Rotate the key clockwise and the animation will automatically start on the screen.

V. MACHINE OPERATION

V.2 Software Operation

Software Operation

1. After the starter animation on the interface, it will automatically jump to input password. Click "**Start**" key to enter the operation interface.

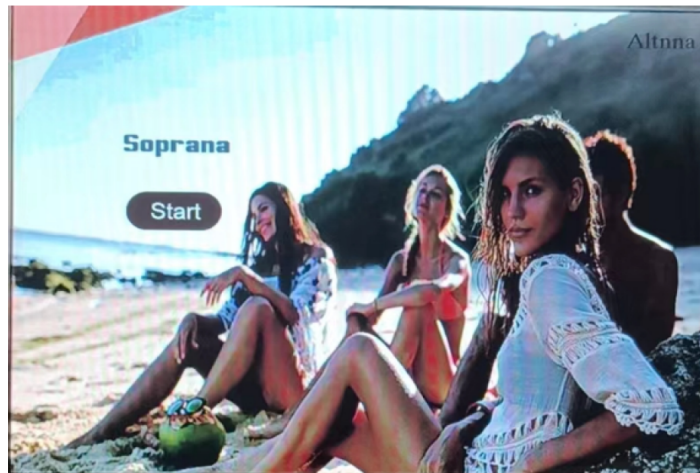

2. Select according to male and female avatars.

Select the size of the area being treated: **S, M, L & XL**

Select the shade of the skin color being treated: **I, II, III, IV, V, & VI.**



V. MACHINE OPERATION

V.2 Software Operation

3. Adjust "**Energy**" density. The general hair removal energy density is above 50 J/cm². The usual setting is kept between 60~80 J/cm² and is adjusted according to sensitivity of the skin.

Adjust "**Frequency**" to determine the energy and the pulse width. If the skin is white, it is recommended to increase the frequency and reduce pulse width. On the contrary, increase pulse width and reduce frequency if the skin is dark.

Click "**READY**" after adjusting the parameters. This key will turn **red** after 3 seconds and the handle will start cooling.



4. Press on the handle button or step on the light to perform hair removal on target area. To pause, press "**STANDBY**" to stop working and enter standby mode.

VI. MACHINE PARAMETER SETTING

Parameter Setting Reference		
Part	Intensity	Frequency
Upper Lip	20~30	1~2
Underarm	20~30	1~2
Limbs	30~45	2
Bikini	20~30	1~2
Hairline	18~25	1~2
Beard	18~25	1~2

NOTE:

The setting will also depend on the skin color.

VII. PRE AND POST TREATMENT CARE

Pre- Treatment

1. Clean the treatment site. Take photos before surgery. Shave the hair if it is long but leave a little hair root, DO NOT completely shave it.
2. According to the treatment area, select appropriate parameters according to the color and skin condition of the customer.
3. Select the corresponding skin color on the software interface. Then select the energy parameter according to the customer's pain and hair thickness. Apply a gel at the treatment site and operate with a sliding method;
4. Check the final outcome: smell if the hair is burnt, check if the the hair is rolled-up, or part of the hair roots are reddish.
5. After the treatment is completed, continue to **Post-Treatment Care**.

Post-Treatment Care

1. After hair removal, do not use body scrub, alcohol, and anti-perspirant. Use warm water when taking a bath. Do not use hot water on the area where hair is removed.
2. DO NOT go to sauna or swim in the swimming pool or beach within one week after hair removal.
3. DO NOT use cosmetics and skin care products containing fruit acid after depilation.
4. The depilated area may produce slight redness and swelling, sensitive skin, hot or itchy sensation. Ice compress can be used to alleviate the pain.
5. Avoid sunshine as much as possible and use sunscreen when you go out.
6. Pay attention to the cleanliness and dryness of the depilation area. Do NOT wear clothes that are not clean or too tight to ensure that they are dry and breathable.
7. Pay attention to diet, prohibit smoking and alcohol, and do not eat spicy stimulating foods. Increase intake of Vitamin C.

VIII. FREQUENTLY ASKED QUESTIONS

1. Is it effective if I use it only once?

Laser hair removal usually takes three to six times to get rid of hair. The reason for this are:

- The hair follicles are mostly bundled together in three groups and open to the same pore. One hair in the pore is the hair growing from one of the three follicles under it. Therefore, one treatment can only destroy one hair follicle in one group.
- Hair growth needs to go through growth period, regression period, and rest period. Laser hair removal is effective in 75% of the growing hair, 25% of the regression period, and the rest period is almost ineffective.

2. Does it affect perspiration after depilation?

Perspiration is mainly caused by small sweat glands and these are not in the hair follicles. At the same time, laser hair removal is to remove hair follicles and not harm sweat glands, so it does not affect human metabolism and perspiration.

3. Is the hair removal effect permanent?

The current laser or glare therapy device has the function of permanent hair removal. Many people will misunderstand that the hair is not long after treatment, in fact, this permanent is not truly permanent. The US Food and Drug Administration's understanding of permanent hair removal is that the hair does not grow during a hair growth cycle after laser or intense light treatment. In general, hair clearance after treatment with multiple lasers or intense light can reach 90%. Its efficacy is affected by many factors.

4. Is there any damage to the skin caused by depilation?

The skin of the human body is a relatively light-transmissive structure. The clinical and beauty experts found that when exposed to a powerful laser, the skin is simply a transparent cellophane and laser can penetrate the skin smoothly and through the hair follicles. The melanin can preferentially absorb a large amount of laser energy and eventually convert it into heat energy that results to the temperature of the hair follicle to rise, thereby destroying the hair follicle. In this process, since the skin does not absorb laser energy relatively, or absorbs a small amount of laser energy, the skin itself will not be damaged.

VIII. FREQUENTLY ASKED QUESTIONS

5. Will the hair get blacker and darker?

The hair remaining after treatment with laser or intense light becomes finer and the color becomes lighter. Shaving, plucking, hair removal cream, wax paper hair removal, etc., people who use these methods for a long time will gradually make the hair thicker, harder, and darker.

6. Is the hair removal effect the same for everyone and all parts?

The number of hair removals depends on the location, the color and thickness of the hair, and the type of skin. Generally, the whiter the skin color, the darker and stronger the hair, the better the reaction, and the better the hair removal effect.

7. Can you sunbathe immediately after hair removal?

After the hair removal, the skin surface protection function will be reduced, and it is easy to be sensitive. Avoid further irritating the skin, so it is not advisable to sunbathe immediately.

IX. MAINTENANCE

XI.1 Daily Inspection

- a. Check water level. Change the water weekly.
- b. The temperature where the machine must be operated should be lower than 28 degrees C but not lower than 0 degree C, otherwise, ice will form and destroy the water pipe and handle of the instrument.
- c. Clean the cold gel on the handle to ensure that there is no dust on the light spot of the handle.
- d. DO NOT bend the protective sleeve of the handle by force, otherwise, it may lead to wrong flow switch.
- e. DO NOT insert and pull out the handle frequently. This may lead to leakage or poor power connection.
- f. DO NOT shoot at will, or you may catch fire.

XI.2 Cleaning

- a. Perform regular cleaning to ensure sanitation and proper function
- b. The outside body of the machine can be cleaned using a soft damp cloth
- c. Clean only when the machine is unplugged or disconnected from the power source.



REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
rejuvafresh.com