

Pro-Secret Icy Gold RF Microneedling Machine

USER MANUAL



Welcome

to your Pro-Secret Icy Gold RF Microneedling Machine



TABLE OF CONTENTS

PREFACE	
I. PRODUCT INFORMATION & SPECIFICATION	2-3
II. PRODUCT APPEARANCE	4
III. FREQUENTLY ASKED QUESTIONS	5-6
IV. NEEDLE CARTRIDGES	7
V. MACHINE OPERATION	
V.1 Face Treatment	8
V.2 Belly Treatment	9
V.3 Cold hammer	10
V.4 Treatment Settings	11
VI. PRECAUTION	12

This manual is a property of Rejuva Fresh.
It is not allowed to be copied, reproduced, distributed, published or modified in any
way nor be distributed any part of this content over any network.



PREFACE

Thank you for choosing **REJUVA FRESH** Pro-Secret Icy Gold RF Microneedling Machine. This machine uses the harmonizing combination of golden RF fractional microneedling plus ice hammer to achieve powerful visible skin anti-aging effects.

Treatment with this machine is not only one of the best ways to reduce scars and stretch marks, but it is also ideal for safely providing long-term skin tightening and wrinkle reduction.

Please read this User Manual carefully in order to use the machine properly and to protect the machine from unnecessary damages.

Should you encounter any issues during operation of the machine or any related questions, you can contact us at our website: rejuvafresh.com or email us at hello@rejuvafresh.com.

Thank you again, and please enjoy your Pro-Secret Icy Gold RF Microneedling Machine!

I. PRODUCT INFORMATION & SPECIFICATION

I.1 Product Information

Microneedling activates the skin's natural healing process, boosting metabolism and the production of collagen and elastin, which in turn keep the skin smooth & glowing. Radio frequency (RF) therapy not only tightens skin and reduces wrinkles, it also helps break down and remove fat cells. So there is a firming, contouring effect. Finally, the cold hammer soothes the skin and reduces any redness or puffiness following the other treatments.

I.2 Product Specification

Power	200 W
Voltage	100 V/ 110 V/ 220 V Voltage selected according to your region
Screen Size	12 in
Radio Frequency (RF)	2-4 MHz
RF Output Power	10-120 W
Needle Cartridges	10 pin, 25 pin, 64 pin and nano needle
Needle Depth	0.2-3.5 mm
Treatment Area	1 cm x 1 cm
Package Dimension	53 x 45 x 36 cm

I. PRODUCT INFORMATION & SPECIFICATION

I.3 List of Accessories


Description	Quantity	Image
Body	1 set	
RF microneedle handpiece	1	
Cold hammer handpiece	1	
Needle Cartridges (Replacement Cartridges are purchased separately.)	4 one each of 10 Pin, 25 Pin, 64 Pin, Nano Pin	
Power Cord	1	

II. PRODUCT APPEARANCE

II.1 Front view



II.2 Back view

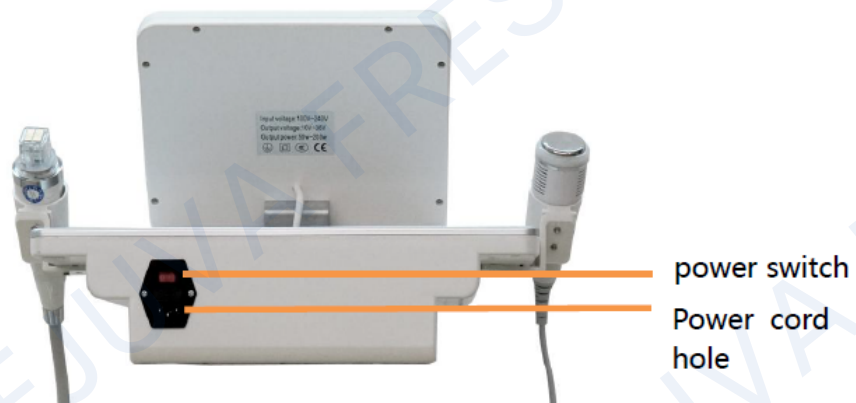

NOTE:


Disinfection of handpiece:

- 1. Place the handpiece in a small bowl.**
- 2. Put alcohol on the bowl. The alcohol must not fully submerge the handpiece.**
- 3. Check if there are remaining liquid on the handpiece before using for treatment to prevent damage.**

III. FREQUENTLY ASKED QUESTIONS

III.1 How does this work?

Microneedling machine has a special design of many fractional needles point array. A high speed digital motor controls its movement through the epidermis in an accurate depth of 0.2–3.5 mm. At the end of the needles, RF is released thereby stimulating collagen production and tissue elasticity. This is done on the safe layer of the skin, which is the epidermal layer. This treatment is recommended to improve the appearance of scars and for long term tightening of the skin and wrinkles.

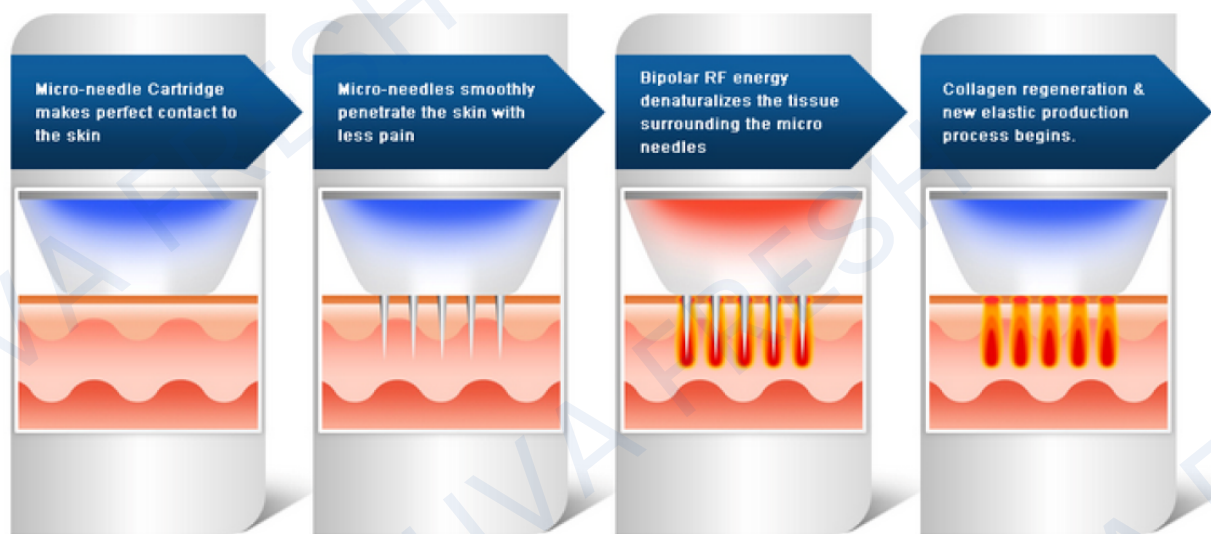

III.2 What is the advantage of using this machine?

1. The treatment uses vacuum making the customer feel comfortable.
2. The needles are gold plated. Customers with metal allergy can use it without concerning contact dermatitis.

III. FREQUENTLY ASKED QUESTIONS

4. The machine uses a stepper motor to provide precise control on the depth of the needles so the needles inserts to the skin smoothly with no shock, no bleeding, and no pain after the treatment.
5. The needle used is a sterilized disposable needle making it safe to use.

IV. NEEDLE CARTRIDGES




NOTE: The needle cartridges are disposable. Additional cartridges are purchased separately.

Microneedling target areas and function:

1. Wrinkles: crow's feet, head lines, smiles, neck lines.
2. Enhance face surface.
3. Stretch marks, swelling pattern.
4. Large pores.
5. Dull skin.
6. Red blood circulation.
7. Skin whitening.
8. Acne scars.
9. Sagging scars.
10. Regular skin care routine.

V. MACHINE OPERATION

V.1 Face Treatment

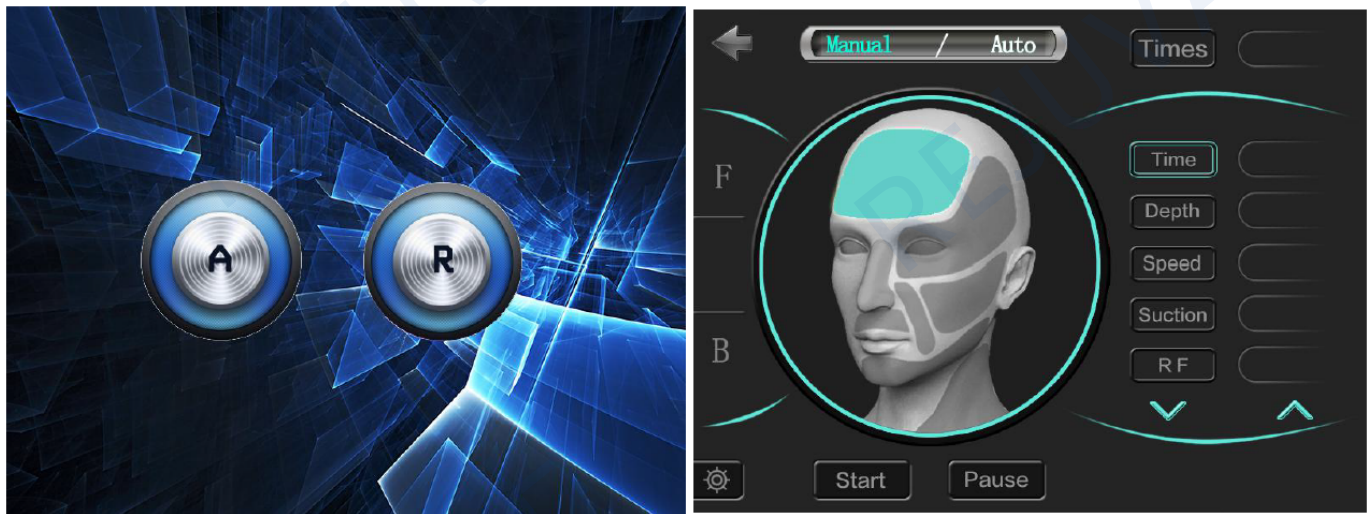


1. Select "A" for Microneedling function.
2. Clean the are of the skin before the start of treatment.
3. Select "F "for Face Treatment.
4. Set the Operation Time. This is different according to area being treated but generally takes 20-25 minutes. Treatment "Times" in one area is 1-2 times.

NOTE: Treatment in one area cannot be repeated several times as it will affect the recovery time.

3. Treatment "Depth", "Speed", "Suction" and "RF" settings are identified on the page for "Treatment Settings"
4. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.

V. MACHINE OPERATION

V.2 Belly Treatment

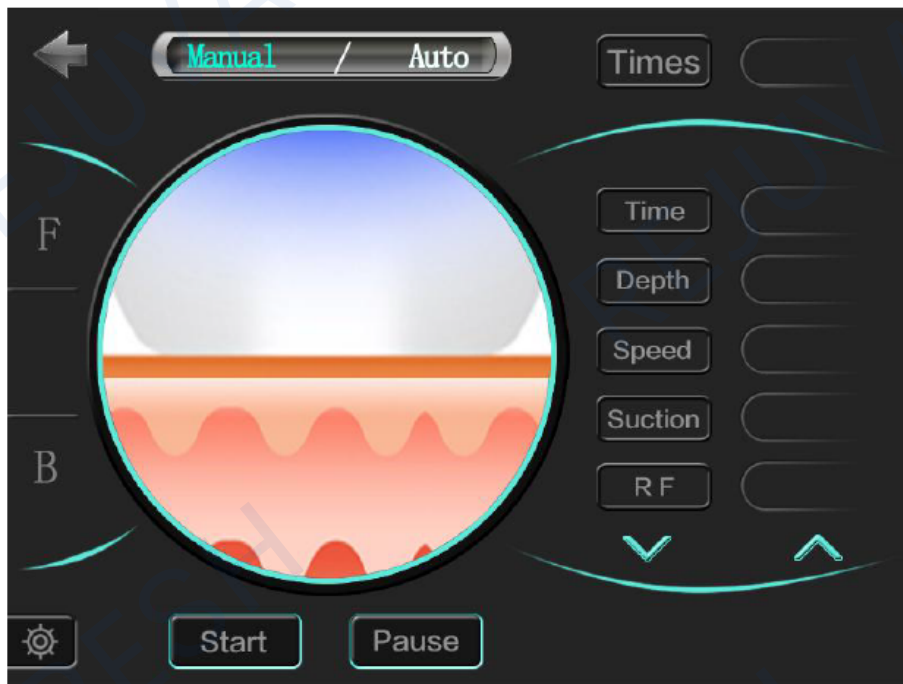


1. Clean the are of the skin before the start of treatment.
2. Select "B" for Belly Treatment.
3. Set the Operation Time. This is different according to area being treated but generally takes 20-25 minutes. Treatment "Times" in one area is 1-2 times.

NOTE: Treatment in one area cannot be repeated several times as it will affect the recovery time.

3. Treatment "Depth", "Speed", "Suction" and "RF" settings are identified on the page for "Treatment Settings"
4. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.

V. MACHINE OPERATION

V.3 Cold Hammer

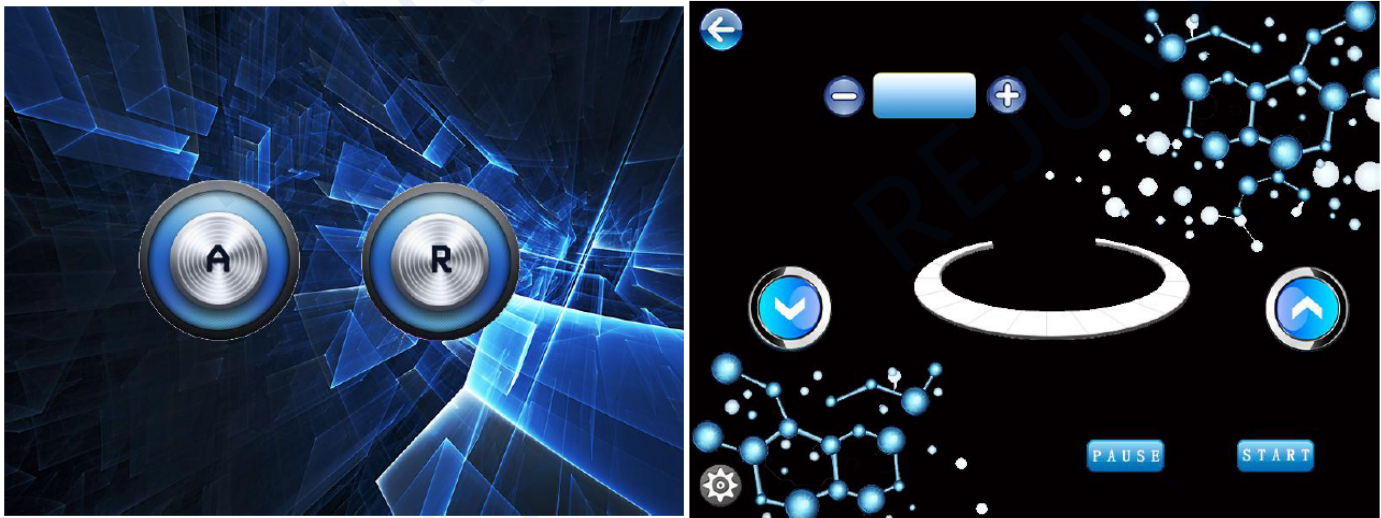

1. Select "R" for Cold Hammer. It's best to use cold hammer for 5-10mins after each session of microneedling. This is for the redness and swelling to be reduced and skin can recover faster. The beauty benefits are maximized.
2. Set the Operation Time. This is different according to area being treated but generally takes 5-10 minutes. Treatment "Times" in one area is 1-2 times.
3. Press "Start" to start the treatment and "Pause" if the treatment needs to be stopped or paused.

V. MACHINE OPERATION

V.4 Treatment Settings

Area	Depth	Duration	Suction Level	RF
Forehead	0.5 - 1.0 mm	0.2 - 0.3 s	1 - 2	60 - 110 W
Face	1.0 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Area around eye	0.5 - 1.0 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Chin	0.5 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Nose	0.5 - 1.2 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Neck	1.0 - 1.5 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Pox Pit	1.0 - 2.0 mm	0.2 - 0.3 s	1 - 2	60 - 100 W
Stretch Marks	1.0 - 2.0 mm	0.2 - 0.3 s	1 - 2	60 - 120 W
Belly	1.0 - 3.0 mm	0.2 - 0.3 s	0 - 1	60 - 120 W

VI. PRECAUTION

1. Aftercare, in order to repair the skin, put moisturizing and repair products twice daily.
Recommended to be done in the morning and evening.
2. Let the wound scabs naturally fall. DO NOT peel the wound scabs.
3. Avoid direct sunlight on the treated area, apply sunscreen.
4. Avoid intense exercise, swimming and sauna while healing is taking place.
5. Avoid alcohol intake for a short period of time while healing is taking place.
6. Treated area must be kept dry for 24 hours.
7. May encounter fever or redness on the treated area a day after the treatment.
8. After 3-4 days, scabs will naturally fall off.



REJUVA FRESH, LLC
551 Red Bridge Road
Ellsworth,
Maine 04605
United States
1-800-249-1310
rejuvafresh.com