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# ※ATTENTION

## DEAR USERS:

Thank you for choosing the instrument designed and manufactured by our company. In order to better play the effectiveness of the product, we sincerely suggest that:

- ◆ Before using the instrument, please read this manual carefully and keep it properly. Well managed for future reference and inquiry.
  - ◆ Please install, use and operate correctly according to the manual.
  - ◆ Do not disassemble or refit any parts of the machine.
  - ◆ Except for the authorized personnel of the company, it is strictly forbidden to open and open the machine cover host.
  - ◆ It is strictly forbidden for non professional personnel to carry out maintenance and transformation without authorization.
  - ◆ If there is any fault during operation, please contact us.
- 

# ※SUMMARY

South Korea 808nm semiconductor laser hair removal instrument is a high-end hair removal instrument, which adopts the most advanced laser hair removal technology in the world, and is the current beauty market.

## ※ PRODUCT ADVANTAGES

- ◆ **Laser: using the world's top stacked laser.**
    - Super long service life: 10 million times of continuous light emission.
    - Ultra high peak power: single target power up to 50W / bar.
  - ◆ **Efficient and stable refrigeration system.**
    - The laser refrigeration system adopts semiconductor refrigeration, which can ensure the continuous operation of the whole system for 7 \* 24 hours. Air cooling + water cooling, which is widely used in the market, can only work for 2-3 hours.
    - Using contact sapphire cooling system, high efficiency of cooling conduction, make skin more comfortable in the process.
    - Intelligent water temperature control system: avoid high water temperature and damage handle.
    - Unique shape design and color matching make the product more beautiful and fashionable.
- 

## ※ FUNCTIONAL PRINCIPLE

- ◆ 808nm semiconductor laser hair removal system, using its unique long pulse width laser, penetrates the skin to the hair follicle, according to the selective wide laser, penetrates the skin to the hair follicle. According to the selective light absorption principle, the laser energy is preferentially absorbed by the melanin in the hair, and then makes the hair lose the regeneration ability. At the same time, in the operation process, the system realizes the unique dynamic cooling of sapphire. The technology can cool the skin in a short time, ensure that the skin will not be burned, and achieve the painless, fast and long-term hair removal effect.

## ※ SCOPE OF APPLICATION

- ◆ Depilation: effectively remove the redundant hair of various parts of the human body, such as armpit hair, beard, lip hair, bikini line and body hair of limbs.
- ◆ Tender skin: shrink pores and whiten skin.

Arm/Hands

Thigh/Leg

Face/Lips

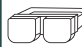
Body/Privates

## ※ TECHNICAL PARAMETERS

Home – use	measur- ement	parameter	note
Peak power of laser	W	50	
Peak power of the whole machine	W	250	
Input working current	A	1.8	
Input working voltage	AC/V	220	
Illuminance Area	mm	10*5mm	
Working conditions	/	DC 20%, max 400ms	
Energy density	J/cm <sup>2</sup>	8-40	
Working temperature	°C	15~35°C	
Refrigeration mode	/	semiconductor	
Working frequency	Hz	0.6-4	
Cooling gear	/	0-3	Stop when 0
Alarm temperature	°C	37°C	Stopping working

# ※ LIST OF ATTACHMENTS


Part name	 Most	 Handle	 Water injection funnel	 power cord	 instructions	 Protective glasses
Number	1	1	1	1	1	1

# ※ INSTALLATION OPERATION


## ◆ Unpacking and inspection

- Carefully take out the instrument and accessories from the packing box and keep the packing materials ready for future transportation or storage. Check the accessories according to the attachment list.
- Check for any mechanical damage.

## ◆ Instrument water injection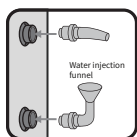


1. Take out the water injection funnel and connect it to the water injection hole on the back of the instrument.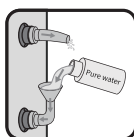


2. Inject pure water or distilled water until there is water flowing out of the overflow port.

**Note:** Be sure to use pure water or distilled water. Do not use tap water or other water.

## ◆ Power on operation

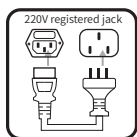
1. Connect the power line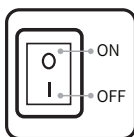

2. Open the power line


# ※ INTERFACE INTRODUCTION

## ◆ Boot interface: boot interface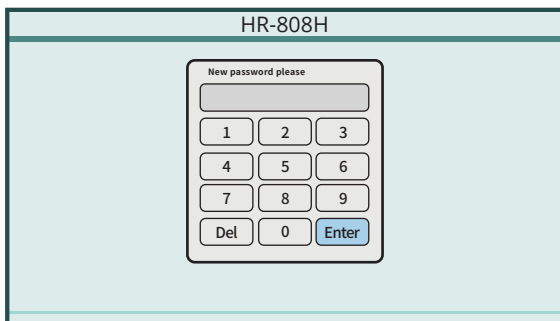

- Click to enter the password and press enter to enter the next corresponding interface.

## ◆ Enter the corresponding operation interface, as shown in the figure:

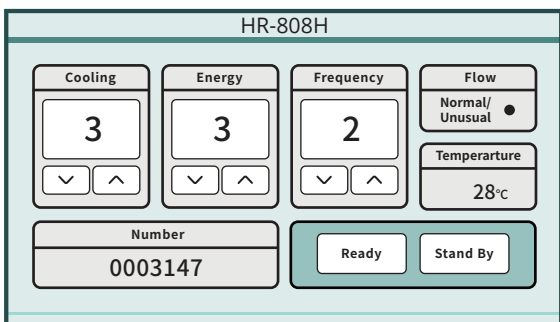- Refrigeration: adjust the cooling intensity to ensure that the skin will not be burned.
- Energy: adjust the laser energy intensity, unique long pulse width laser.
- Frequency: adjust the frequency, stable and safe output energy.
- Flow state: the water circulation state of the handle.
- Water temperature state: the water circulation temperature state of the handle.
- The number of times of light: how many times does the energy give out.

# ※ OPERATION STEPS

## 1 Shaving

Before operation, a shaving process should be carried out, and the hair should be kept 1-2mm to facilitate the absorption of light by the hair.


## 2 Apply cold gel

Apply a little cold gel on the operation area. The thickness of the coin should not be too thick, otherwise the effect will be affected.

## 3 Adjust operating parameters

.According to the guest's skin color, hair thickness and depilation position, select the corresponding operation parameters, and adjust them according to the customer's feeling.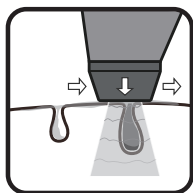


## 4 The operation process

Fit the skin, avoid light leakage, and slowly slide the handle probe for operation.


Energy regulation: starting from energy 4, if the operator thinks that the heat is too large, the energy will be reduced; if there is a slight thermal expansion, it means that melanin is absorbed by the laser with the wavelength of 808nm, and the light energy is converted into heat energy to destroy the melanin of hair follicle, and the effect is the best.

Operation time: about 15 minutes for legs, 10 minutes for arms, 7 minutes for armpit and 5 minutes for lip hair (for reference only, the specific time should be determined according to the changes of hair and the size of hair removal part). The clinical end point is that the hair follicles are red and protruding, the hair is slightly curled and the hair follicles are hot.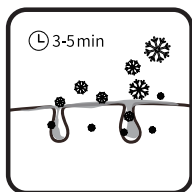

## 5 Skin cooling

After operation, cool the operating part for 3-5 minutes.

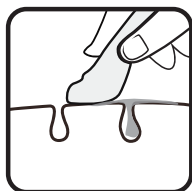## 6 Clean the skin

Remove gel, apply moisturizing products or body lotion.

### ◆ Operation precautions

- Eyes must not look directly at the strong light coming from the exit of the front end of the laser handle.
- During operation, the handle should slide slowly, and it should not stay in one place for too many times to avoid customer discomfort.
- Treatment interval: 15days.

### ◆ Precautions after depilation

- Avoid hot water bath within 7 days after depilation, suitable for warm water.
- Avoid spicy, tobacco, alcohol, stimulant food.
- Avoid exposure to the sun, do a good job in sun protection, wipe sunscreen, isolation cream.
- Avoid manual shaving after depilation, do a good course of depilation.

## ※ PRECAUTIONS FOR INSTRUMENT MAINTENANCE

- It is recommended to replace purified water or distilled water every 1 month.
  - The instrument should be placed in a clean and ventilated room with a temperature of 5-40 °C and a humidity of no more than 80%.
  - During use, make sure that the air vent of the instrument is not blocked, and there is at least 5cm gap around the instrument.
  - The instrument should not be moved in a long distance; before transportation, the water in the instrument must be drained completely, and the shock absorption packaging provided by the original factory must be used during transportation.
- 

## ※ TABOO CROWD

- Patients with heart disease and hypertension.
- Pregnant women, epileptics
- Patients with hypertension and diabetes.
- Photoallergic customers.
- People who have been working under strong light for a long time, and their skin has been sunburnt recently.
- Short term skin change customers.
- Customers taking antioxidants or hormones.

## ※ QUALITY ASSURANCE

The instrument is guaranteed for 12 months from the date of purchase (with the bill), and the fault (non-human phenomenon) within the warranty period is free of charge. Beyond the warranty period can also be responsible for maintenance, according to the company's provisions to charge the cost of spare parts and consumables (transportation costs to be borne by the customer). If you need to supplement or purchase, please contact our company.

### ◆ No free maintenance and warranty service is provided for the following user's personal reasons:

- Failure caused by unauthorized disassembly and modification of the product.
  - Failure caused by using hand tools not distributed by the company.
  - Failure caused by careless use, falling and falling.
  - Failure due to lack of reasonable maintenance.
  - Failure caused by failure to follow the instructions correctly.
- 

## ※ PRODUCT WARRANTY CARD

Machine name		Customer name	
Machine model		Telephone	
Purchase date		Address	
Maintenance date		Maintenance content	
Cause of failure		Repairman	

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