Manufactured for
Respironics Respiratory Drug Delivery (UK) Ltd
Chichester Business Park, City Fields Way
Tangmere, Chichester, PO20 2FT UK

+1 724 387 4000

www.philips.com/respironics

© 2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
(Royal Philips) or their respective owners.

1122649 R01

CE 0123

www.philips.com/respironics

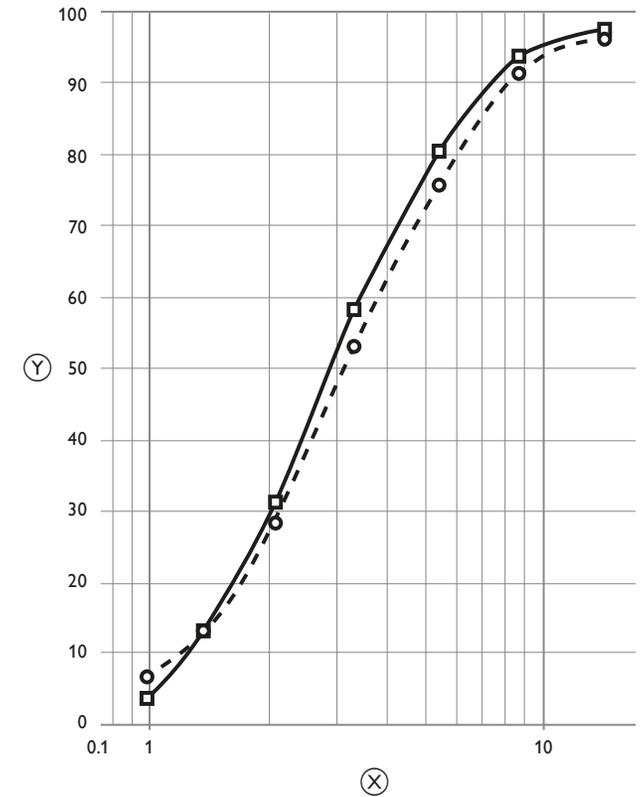
SideStream

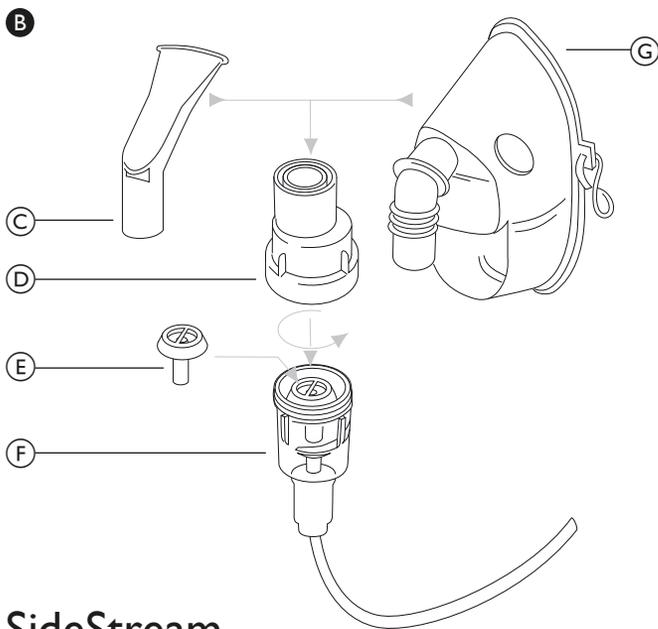
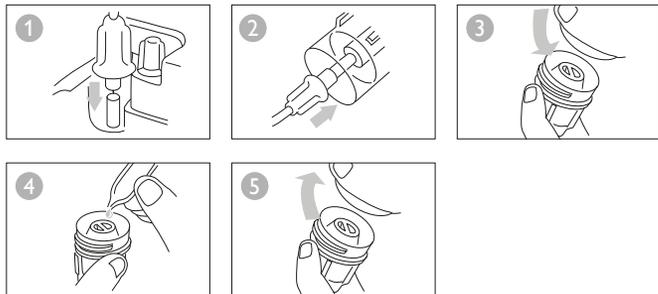
| | |
|---------|---|
| English | 3 |
| Deutsch | 7 |

PHILIPS
RESPIRONICS

A

SideStream

2

SideStream

1 reusable nebulizer, angled mouthpiece, masks and tubing.
For single patient use.

B Parts of SideStream (see page 2)

- | | | | |
|---|------------|---|--------------|
| C | Mouthpiece | F | Medicine cup |
| D | Top | G | Mask |
| E | Baffle | | |

Material information

- Polypropylene - nebulizer parts (excluding jet and base), mouthpiece, mask connector
- PVC - Adult and child masks, tubing
- Thermoplastic polyurethane - tubing (ends)
- Polycarbonate - nebulizer jet and base
- Polyester - mask elastic straps

Important

SideStream is a nebulizer designed to convert liquid medication into aerosol for inhalation by the patient in the home or hospital environment, for treatment of respiratory diseases.

SideStream cannot be used to deliver oxygen or oxygen mixtures.

Please carefully read and understand these instructions before using this product. If you do not understand any portion of these instructions, contact your healthcare provider or call Customer Service at 1-800-345-6443 or 1-724-387-4000.

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12

Warning

- This product may contain phthalates which may cause adverse health effects. For children, or if you are pregnant or breast feeding, contact your health care professional before use.
- Symbol with DEHP contains di-(2-ethylhexyl) phthalate.
- Clogged jets or improper assembly may prevent proper delivery of medication.
- To reduce the risk of infection, clean the nebulizer between treatments as directed.
- Do not immerse tubing in water.

Caution

- Do not try to remove the red jet from the medicine cup. This is permanently attached.
- Do not towel dry the unit and never use a cleaning brush or put sharp objects into the jet holes or baffles as this will damage the nebulizer.
- SideStream reusable is not suitable for use in an anaesthetic breathing or lung ventilator system.
- Do not remove top when connected to gas supply.
- Use of 22mm male connectors may reduce output rate.
- Only use prescribed medication with Sidestream.
- Children and infirm individuals should be supervised during use.

A Cumulative drug distribution – Salbutamol (see page 1)

Results of cascade impactor measurements for particle size.

(X) Upper aerodynamic diameter (μm)

 6L/min

(Y) Drug deposition (%)

 8L/min

Measure according to EN13544-1:2007 +A1:2009.

Performance information provided may not apply to drugs supplied in suspension or high viscosity form. In such cases, please refer to drug supplier.

Getting ready

Upon receipt of your SideStream

We recommend that you boil your reusable SideStream (except for mask and tubing) in water (minimum 100° C) with 2–3 drops of dishwashing liquid for 10 minutes. This ensures maximum performance.

Instructions for use

You should wash your hands before handling medication or the nebulizer:

- 1 Connect one end of the tubing to the compressor air outlet.
- 2 Attach the other end of the tubing into the bottom of the medicine cup.
- 3 Unscrew the top from the medicine cup.
- 4 Pour your prescribed medication into the medicine cup, 2-10ml fill.
- 5 Screw the top back onto the medicine cup and connect the mouthpiece/mask.
Do not lay the SideStream down on its side, because the medication will spill out.

It is recommended that you replace your SideStream every 12 months. Dispose of according to the local regulations in force.

Taking a treatment

- To take the treatment, sit in an upright position and relax.
- Switch on the compressor; supply flow to be 6-8l/min.
- Make sure mist is visibly coming from the mouthpiece/mask.
- Take the mouthpiece between your teeth, surround it with your lips and breathe in and out slowly through your mouth or place the mask in position over nose and mouth and breathe slowly in and out through your mouth.

When the SideStream begins to work intermittently and 'sputters', then the treatment is finished. There is no need to nebulize every drop of your medication, there will always be a small amount left in the medicine cup.

Cleaning

After every use

- 6 Disconnect the SideStream from the tubing. Unscrew the top from the medicine cup, rinse out the nebulizer and refill with clean water. Reassemble top and medicine cup, reconnect the tubing and run SideStream again for a few seconds to flush out any drug trapped in the nebulizer jets.
- 7 Take apart the nebulizer.
- 8 Disconnect tubing from the medicine cup.
- 9 Wash all items (except mask and tubing) in either hot soapy water or contained within a basket on the top rack of your dishwasher.
- 10 Rinse all parts in clean water for 2 minutes.
- 11 Shake off excess water and allow to air dry. Do not towel dry.
- 12 Wipe mask with a clean, damp cloth and allow to air dry.
- 13 Reassemble and store the SideStream in a clean place when not in use.

Once a week

- 14 Disassemble the nebulizer.
- 15 Boil the SideStream (except mask and tubing) in water with 2–3 drops of dishwashing liquid for 10 minutes.
- 16 Rinse all parts in clean water for 2 minutes.
- 17 Shake off excess water and allow to air dry. Do not towel dry.
- 18 Reassemble and store SideStream in a clean place when not in use.

| | Minimum flow 6 LPM @ 69 kPa | Maximum flow 8 LPM @ 124 kPa | Recommended flow 6 LPM @ 69 kPa |
|--------------------------|------------------------------------|------------------------------------|------------------------------------|
| Particle size (MMAD) | 2.9 | 3.1 | 2.9 |
| Aerosol output | 0.49 mL (2.5 mL/salbutamol) | 0.42 mL (2.5 mL/salbutamol) | 0.49 mL (2.5 mL/salbutamol) |
| Aerosol output rate | 0.09 mL/min (2.5 mL/salbutamol) | 0.15 mL/min (2.5 mL/salbutamol) | 0.09 mL/min (2.5 mL/salbutamol) |
| Medicine cup capacity | Minimum 2 mL – maximum 10 mL | Minimum 2 mL – maximum 10 mL | Minimum 2 mL – maximum 10 mL |

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