SuperPeg II

Properties / Eigenschaften:

SuperPeg II- the ultimate Ceramic Firing Paste- Peg Putty

- Conveniently stabilizes ceramic restorations on firing pegs.
- Zero clean-up is required- just a blast of compressed air is enough.
- Eliminates divesting by way of scraping out and sand blasting.
- No drying time is required.

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SuperPeg II

Apply a small amount from the syringe directly into the crown/coping or on the back of the veneer or inlay. Please make sure that material does not touch the unbaked ceramic.

Fix and adjust your object on the holding pins of firing tray or place it directly on the tray (please fill coping with a little excess material to do so). No drying time required. Finally start your firing process as usually, SuperPeg II does not ask for any adjustments of the firing cycle. After Firing simply clean off SuperPeg II with a stroke of compressed air. Please do not reuse SuperPeg II once fired, the material contracts a bit and a perfect fit during further firing cycles cannot be guaranteed.

SuperPeg II contains small fibres that might clog the syringe tip- please simply use an instrument to push in and unclog the tip again.

The sealing cap of the syringe needs to be put on immediately after usage to prevent dryout of the material!

Cool or humid storage of SuperPeg II is not mandatory.

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