


# Zirtooth® MultiLuster

## Aesthetic & Strength


- Natural gradation of shades
- Excellent strength to withstand margin chipping and fracture
- Gradation block with the same strength from cervical to incisal

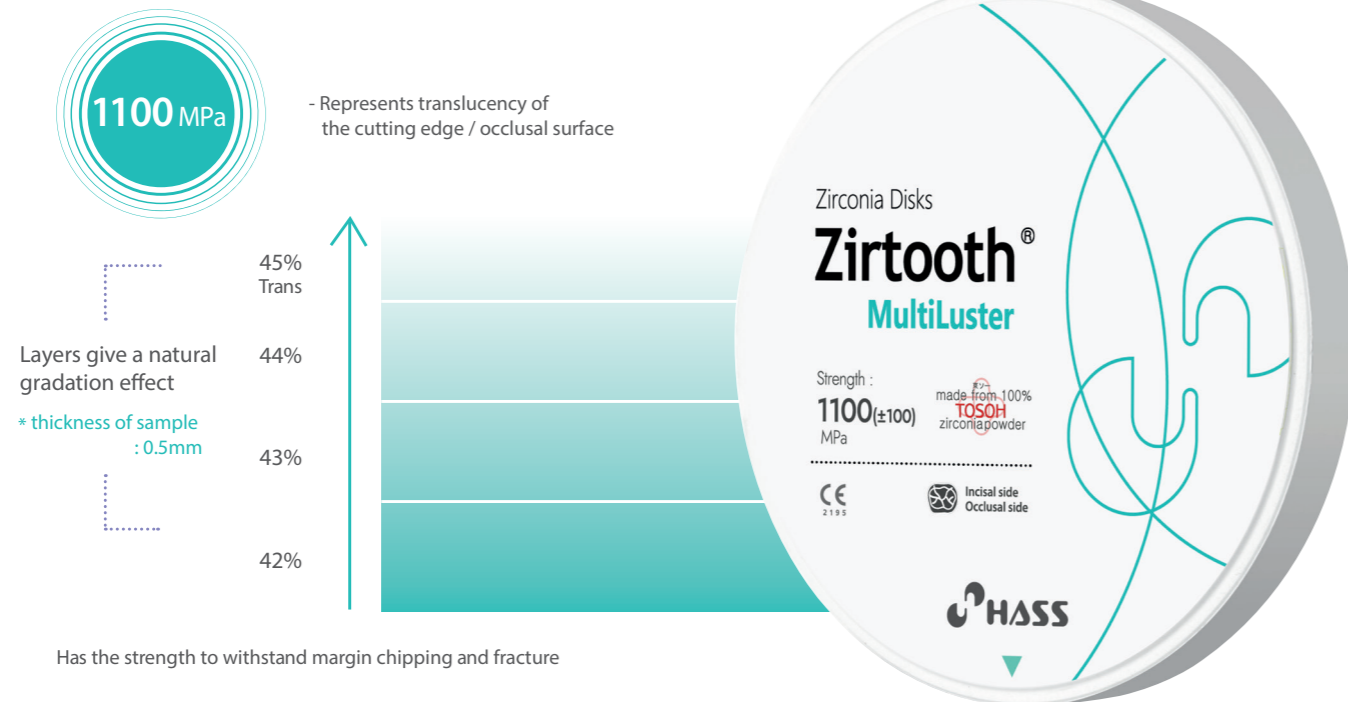

Has the strength to withstand margin chipping and fracture

## MultiLuster

- Translucency ●●●○○
- Strength ●●●●●
- Aesthetic ●●●●○

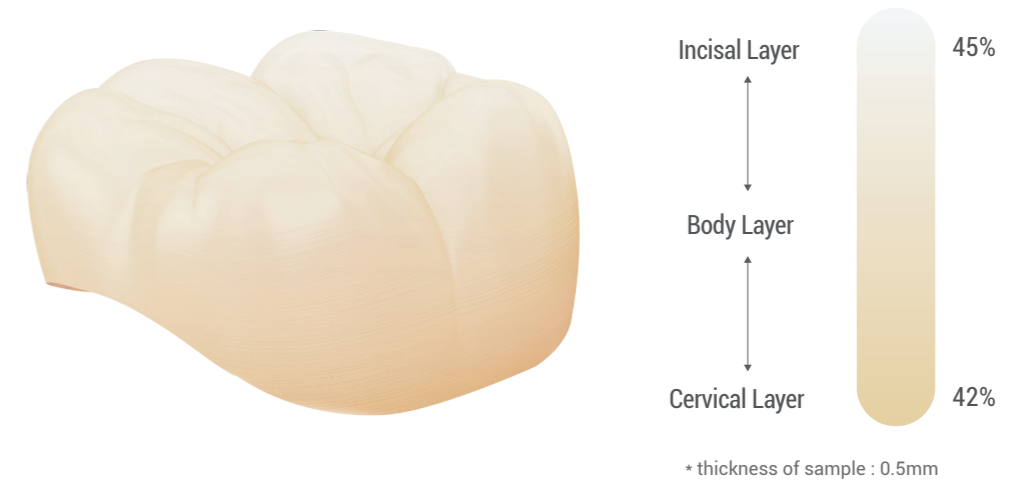

## Indication

- Anterior Crown
- Posterior Crown
- 3-Unit Bridge
- Long-span Bridge



## Sintering Schedule

| Step         | 1                         | 2              | 3                         |
|--------------|---------------------------|----------------|---------------------------|
| Temperature  | Room temperature ~ 1000°C | 1000°C~ 1530°C | 1530°C ~ Room temperature |
| Heating Rate | Max 9°C                   | Max 3.5°C      | Max 8°C                   |
| Holding Time | 10 min                    | 2 hours        | -                         |

## Size

| Model No. | Dimensions | Model No. * Zirkozahn | Dimensions |
|-----------|------------|-----------------------|------------|
| O-9812    | ø98 × 12T  | Z-95(12)              | ø95 × 12T  |
| O-9814    | ø98 × 14T  | Z-95(14)              | ø95 × 14T  |
| O-9816    | ø98 × 16T  | Z-95(16)              | ø95 × 16T  |
| O-9818    | ø98 × 18T  | Z-95(18)              | ø95 × 18T  |
| O-9820    | ø98 × 20T  | Z-95(20)              | ø95 × 20T  |
| O-9822    | ø98 × 22T  | Z-95(22)              | ø95 × 22T  |
| O-9825    | ø98 × 25T  | Z-95(25)              | ø95 × 25T  |

## Available Shades

- A1, A2, A3, A3.5, A4
- B1, B2, B3, B4
- C1, C2, C3, C4
- D2, D3, D4

## Clinical Case



Zirtooth MultiLuster A2 - Shade A2(24-25), A3(26), A3.5(27)  
\* after color characterization