


Zirtooth® UltraLuster

High Strength

- Excellent strength and transparency (**1100MPa**)
- Suitable for various types of prosthesis

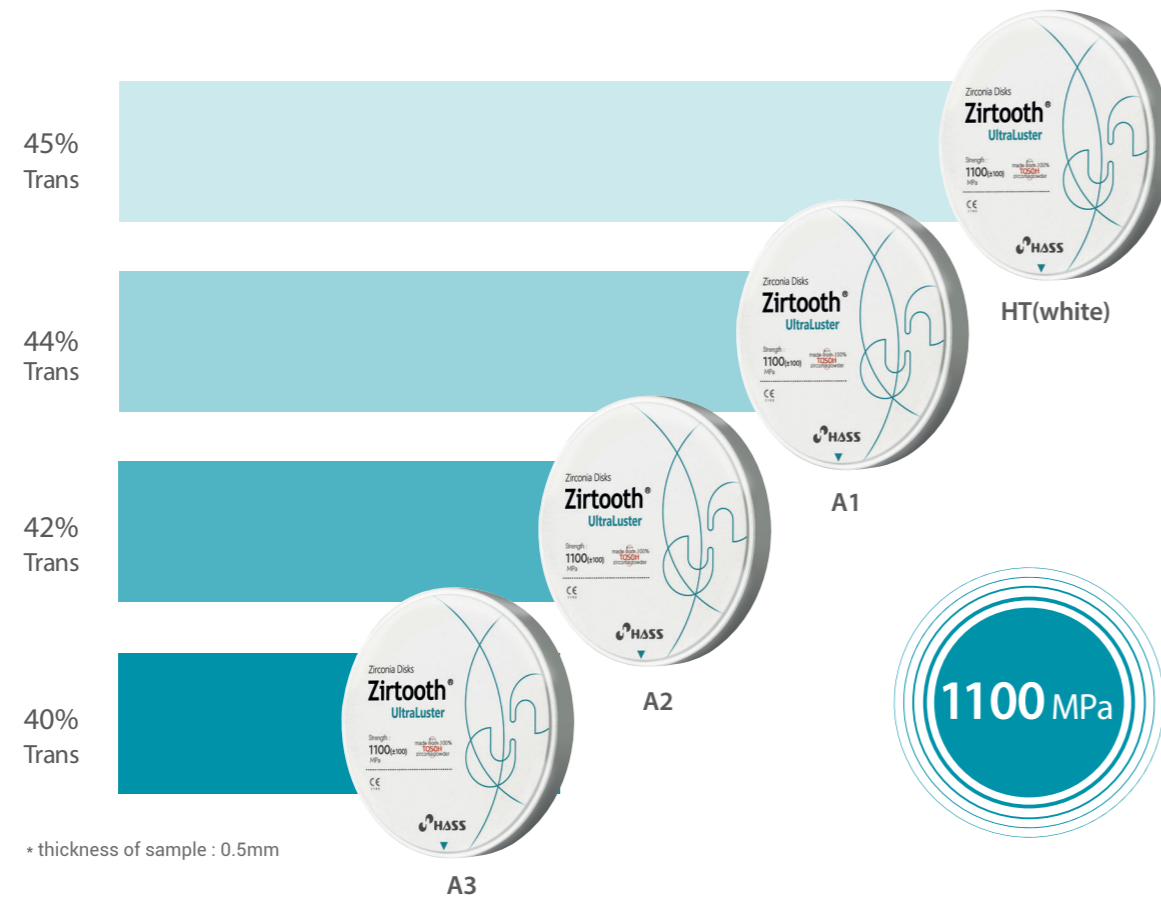

Clinical Case



Zirtooth UltraLuster A1 - Shade A2(15), A3(16), A3.5(17)
* after color characterization

UltraLuster

- Translucency ●●●○○
- Strength ●●●●●
- Aesthetic ●●●○○



Indication

- Anterior Crown
- Posterior Crown
- 3-Unit Bridge
- Long-span Bridge



Sintering Schedule

Step	1	2	3
Temperature	Room temperature ~ 1000°C	1000°C~ 1530°C	1530°C ~ Room temperature
Heating Rate	Max 9°C	Max 3.5°C	Max 8°C
Holding Time	10 min	2 hours	-

Size

Model No.	Dimensions	Model No. * Zirkozahn	Dimensions
O-9810	ø98 × 10T	Z-95(10)	ø95 × 10T
O-9812	ø98 × 12T	Z-95(12)	ø95 × 12T
O-9814	ø98 × 14T	Z-95(14)	ø95 × 14T
O-9816	ø98 × 16T	Z-95(16)	ø95 × 16T
O-9818	ø98 × 18T	Z-95(18)	ø95 × 18T
O-9820	ø98 × 20T	Z-95(20)	ø95 × 20T
O-9822	ø98 × 22T	Z-95(22)	ø95 × 22T
O-9825	ø98 × 25T	Z-95(25)	ø95 × 25T

Available Shades

- HT (White)
- A1
- A2
- A3