VenenWalker® pro

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DE

VeinWalker® pro

Instruction manual (Page 22)

GB

VeinWalker® pro

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FR

VenenWalker® pro

Handleiding (Pagina 62)

NL

by **GlobalMIND®**

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Please be sure to read the instruction manual!

Should you have questions about the product, you can contact us from Mon-Fri by calling the following service number: +49 (0) 38851 314337 (subject to charges).

Contents

The VeinWalker® by GlobalMIND	24
The history of AIC	27
The principle of AIC	27
Advantages of AIC	28
Using the VeinWalker®	29
Contra-indications	30
Important tips	30
Operating your VeinWalker®	31
Adjusting your VeinWalker®	32
Using your VeinWalker®	33
Cleaning and looking after your VeinWalker®	33
Troubleshooting	34
Hotline	34
Additional information about the VeinWalker®	35
Technical data	38
Explanation of symbols used	40

The VeinWalker[®] pro by GlobalMIND

The comprehensive massage equipment for effectively treating vein problems at a very reasonable price.

Sitting or standing for a long time and eating the wrong kind of food: it is generally estimated that 20 % of all Europeans have problems with their legs resulting from disorders of the veins – in more than half of the cases the disorder is at an advanced stage. Like other international diseases, this disorder is caused by modern life in a civilized society. It begins harmlessly enough with a tingling sensation in your legs. Your legs then begin to feel tired and heavy. You discover thread veins and varicose veins. Your whole circulatory system is endangered – you run the risk of getting an ulcerated leg, thrombosis, pulmonary embolism and you could even become unfit for work.

What happens exactly when you have diseased vein(s)? Veins gather used blood in the tissue and transport it back to the heart. The muscles in your legs and valves in your veins push the blood in the arteries forward like a pump by becoming tense and then limp. If, however, the blood becomes congested in the veins as a result of standing or sitting for too long or being overweight, the blood can no longer flow properly and the walls of the vein are overstretched. This causes your leg(s) to swell up, leading to an oedema in the swollen leg because fluid and protein seep in the body tissues through the weakened wall of the vein. If this venous congestion becomes permanent, then it is high time to consult a doctor. Medication can neither prevent congestion in the veins, nor remedy the dilatation of the veins. Elastic stockings are a tried-and-tested method of supporting the weakened tissue externally and preventing your legs from swelling up. They have to be worn though all the time. Vein walking, a special kind of running popular in the USA, improves the blood circulation especially in the veins and trains all the important muscles – however, this only brings results if you do the sport on a longterm basis. Special massage equipment using alternating compression has proved to be very effective and time-saving not only in the therapy of diseased veins, but also in the cosmetic treatment of lipedema. The VeinWalker[®] pro is the first really effective piece of massage equipment based on recognized methods which have been tried and tested in the medical world – and that at an amazingly reasonable price.

The VeinWalker® pro consists of a control unit and 2 two-layer cuffs for wrapping around the legs, feet and soles. The legs are gently massaged by the VeinWalker® pro by means of an airbag that goes up and down simulating the pump-like effect that the muscles in your leg have when you exercise. The blood in the veins can flow back better, the fluid that has accumulated is set in motion and can be flushed out.

The VeinWalker® pro was developed in close co-operation with specialists (phlebologists). It is both easy and safe to use and (if used properly) has no unwanted side effects. The VeinWalker® pro consists of massage equipment with two leg cuffs and includes an extensive manual that is easy to understand.

The specialist's opinion

Frank Sömmer MD, general practitioner and phlebologist (specialist for veins) has many years of experience in treating and preventing vein disorders and is the medical director of the Technical Institute for Compression Therapy in Lübeck. Dr. Sömmer has this to say about GlobalMIND's VeinWalker pro:

"We have carried out tests on the Vein Walker in the institute and compared it with equipment used not only in clinics and doctors' surgeries, but also at home. The results of the numerous tests show clearly that the VeinWalker is in no way inferior!

On the contrary, the VeinWalker is unique in so far as the patient is in a position to pump the leg cuffs empty. The arduous task of deflating the cuffs is now child's play. If used as prescribed, the VeinWalker is absolutely safe.

The cuffs were specially designed to fit not only the leg but also the foot perfectly.

Moreover, the surface of the cuffs is particularly easy to clean. When you also take into consideration that the VeinWalker is easy to use and is reasonably priced for such a sophisticated product, then it can be thoroughly recommended for use by doctors and private individuals. At last, reasonably priced, professional equipment has been developed for effective treatment at home without the risk of any side effects whatsoever."

Further comments on AIC

Taken from: Brunner, U., Schrey, A. Die intermittierende Kompression (Intermitting Compression). Symposium on 28th May 1982.

“In summary it can be said that an apparative massage (intermitting compression) brings about not only a significant reduction in the volume of a swollen extremity through loss of water but also an improvement and an acceleration in the flowing of lymph. By reducing the oedema, the blood circulation is improved.” (p 63)

...

“A pressure bandage or intermitting compression of an extremity makes the vein capacity calculable by eliminating the elasticity of the veins. This gives rise to a network of communicating tubes consisting of arteries and veins. This network obeys the rules of physics firstly inasmuch as the remaining pressure in the arteries and the suction from the diaphragm can now contribute a great deal to the circulation and the flowing back of the blood to the veins and secondly inasmuch as this cannot occur in the non-compressed leg where the blood can clot in the basin formed in the overstretched vein. A resting circulation at an extremely high level results, acting as a platform for a now incomparably more efficient performance by the pump-like mechanism of the muscles and the joints.” (p 76)

...

“Intermitting compression is a valuable asset in the prevention of thrombosis including orthopaedic surgery.” (p 108)

...

“The effects of intermitting compression are not limited to merely filling and emptying the system of veins in the lower leg. In addition, certain biochemical changes take place, for example there is an increase in the fibrinolysin and an influence is exerted on the mechanisms that control the circulation of blood in the leg.” (p 123)

Taken from: Weber, B., Vanscheidt, W. Intermittierende Kompression als adjuvante Therapie. Angiologie: „Haut“. (Intermitting Compression as adjuvant Therapy. Angiology: “Skin”.) Offprint, edition 4/1992

“If the circumstances warrant it, then intermitting apparative compression is an effective method that has hardly any side effects and makes a sensible and inexpensive contribution to the basic therapy.”

Taken from: “Venen” (Veins). Offprint of the *Ärztezeitung* (medical journal), 1996.

“During treatment with intermitting compression, waves of pressure are produced at set intervals with the help of cuffs filled with air. This simulates the pump-like effect that the muscles in your leg have when you exercise. The blood in the veins can flow back better, the extracellular fluid that has accumulated is set in motion and can be flushed out. Prof. Altenkämper: ‘The patients obtain relief from their pain and the feeling of heaviness in their legs.’”

The history of AIC

The principle of apparative intermitting compression has its origins in the mercury massage. As early as in 1899 there were reports on the special treatment of lymph oedema in the upper extremities (arms). It is reported that the extremities were submerged in a cylinder filled with mercury. As early as in 1917, stagnating blood was driven out of the veins by increasing the pressure in a pneumatic chamber. In 1929, air cushions that rhythmically filled and then emptied themselves were used to treat bed-ridden patients suffering from trophic ulcers. At the university hospital of Leipzig in the early thirties, chronic circulatory disturbances coupled with indurations and ulcers were treated by rhythmically changing the flow of blood in the arteries.

At about the same time in the USA, there were also attempts to treat blockage of the blood vessels with intermitting compression. With the increase in the use of AIC in hospitals, there followed a technical improvement in the quality of the equipment culminating in today's multi-chamber system for treating severe cases of lymph oedema.

The principle of apparative intermitting compression (AIC)

Intermitting (alternating) compression that is produced by massaging with apparatus and not by hand, is known in the medical field as apparative intermitting compression (in short, AIC). This is an application of the pneumatic alternating pressure procedure. It is prescribed by doctors for the treatment of vein or lymph disorders. Apparative intermitting compression has none of the risks that many other types of medication have and is therefore used in the prevention of thrombosis and the treatment of sport injuries.

AIC simply copies the natural muscle pumps in the human body. Air is pumped into two-layer cuffs in such a way that the pressure changes. The pressure is increased and lowered at fixed intervals. The level of pressure should be adjusted to suit the patient's wishes and should never exceed 100 mmHg. How long and how often the treatment should take place depends on the patient's own wishes as regards the cosmetic requirements, and on the doctor's recommendations as regards the medical requirements. One session should however last at least 20 minutes. The treatment should of course be supervised by a doctor.

A lot of attention should be attached to the shape and kind of cuff used. It has been proved how important it is to empty the network of blood vessels in the foot. For this reason, the cuff should not only be wrapped around the whole leg, but also the foot. The single-chamber cuffs are just as effective as the multi-chamber cuffs. The only exception to this is in the treatment of lymph oedema (swelling) where multi-chamber cuffs have proved to be more effective.

By rhythmically increasing and lowering the pressure on the tissue, water and protein is pumped out of the layer of cells surrounding the blood vessels. If the medical grounds warrant it, a pressure bandage or an elastic stocking should be worn after treatment with AIC has taken place. This is because the protein still remaining after treatment with AIC causes blood serum to flow back into the tissue. This step prevents the flowing back. You can forgo these steps if you simply remain lying after each session. For this reason, treatment should preferably take place just before you go to bed.

Advantages of apparative intermitting compression (AIC)

- Improvement in the venous and lymphatic macro-circulation by passively activating the natural muscle pump
- Improvement in the micro-circulation of skin, connective tissue and muscular system
- Improvement in the capacity of the blood to flow
- Positive effects on the flow mechanism
- Increase in the anticoagulant protein C (inhibits clotting of the blood)
- Relief from pain
- Injuries heal more quickly
- Quicker relief from pain caused by accumulation of fluid in the body tissues (edema) due to surgery – shorter stay in hospital
- Healing effect on an ulcerated leg
- Positive influence on the lymph fistulas
- Less risk than with other methods
- Pain-free, convenient, easy to use and no negative side effects

Using the VeinWalker[®] pro*

In the cosmetic field

- Swollen legs caused by lack of movement when, for example: a profession mainly performed sitting or standing or long distance travel
- In the case of lipedema
- Swelling in the legs as a result of confinement to bed or **pregnancy
- In case of varicose veins
- In case of aching muscles, especially as a result of sport

In the medical field

Swollen legs diagnosed by the doctor as being the result of:

- Diseased veins, for example varicose veins (also acts as a prevention)
- Varicose veins during a pregnancy (also acts as a prevention)
- Prevention of embolism as a result of confinement to bed
- Swelling in the lymphatic system (lymph oedema)
- Congestion caused by an injured joint or overexertion, such as a haemorrhage following a sport injury
- Swelling resulting from a pulled muscle or a sprain
- An ulcerated leg caused by circulatory disturbances
- Surgical wounds heal more quickly

** If the patient has difficulty in breathing or if the pulse is rising while using the VeinWalker[®]pro, the treatment should be interrupted. In such cases, the patient should consult a doctor.

Contra-indications *

AIC should not be used, or at least not before a doctor has been consulted, in cases of:

- Decompensated cardiac insufficiency
- Extensive thrombophlebitis
- Erysipelas
- Severe, not regulated hypertonia
- Acute soft tissue trauma of the extremities
- Neuropathy
- Occlusive processes within the area of lymphatic down gradient

* These indications such as contraindications were taken from the medical guidelines of lipedema and apprative intermitting compression (AIC) published by the German Association of Phlebology. Please refer to these guidelines for further information.

Important tips

How to get started

Position the control unit to your left or right (see picture 1) in such a way that the hoses are neither twisted nor bent. You should make sure that your VeinWalker® pro always remains unobstructed. It should never be covered by a blanket or any other object(s).

How to use your VeinWalker® pro

The VeinWalker® pro operates on 230 V/50 Hz alternating current. Simply put the plug in the safety socket (Schuko type). Adapters are suitable for use abroad. But pay particular attention to the voltage and the frequency of the country you are staying in.

Operating your VeinWalker® pro

Connect the tubes to the valves on the leg cuffs and plug the other end of the tubes into the control device. Before use, make sure that the tubes are securely attached to the valves and connected correctly to the control device. Now put on the leg cuffs of your VenenWalker®. Make sure that the tubes do not get bent during use. Only use one cuff. Plug the second opening of the control unit with the stopper. Now turn on your VeinWalker® pro by pressing the switch ■.

Your legs are massaged by an airbag that goes up and down.
(Please do not open the zips during the massage since they could be damaged!)

The massaging of your legs has the same effect on your veins and muscles as light exercise. You can lie down and position your VeinWalker® pro next to you.

The picture shows how to make optimal use of your VeinWalker® pro.

Picture 1

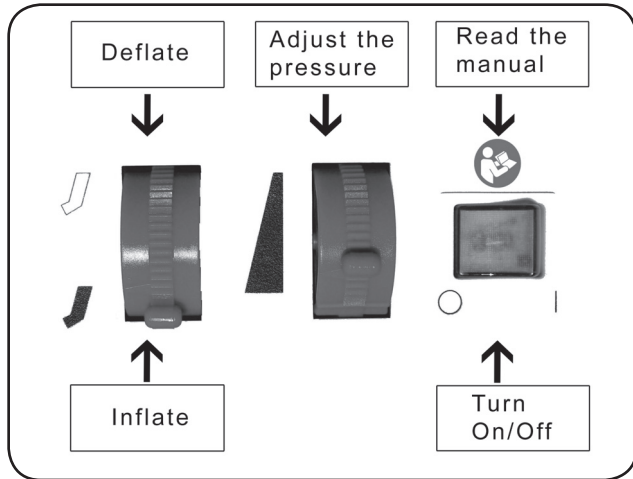
In order to allow the blood to circulate unhindered, the leg around which the cuff is placed must be stretched out during application!

Adjusting your VeinWalker[®] pro

There is a rotary switch on the face of the control unit marked  and .

If you turn the switch downwards in the direction  air flows into the cuffs and inflates them. Once the cuffs are full, your legs are massaged by an airbag that goes up and down. Should the inflating cuffs exert too much pressure on your legs, turn the right-hand regulator up until the pressure is reduced.

If you find that the massage is too rigorous, simply turn the regulator up a little. In this way you can adjust the pressure to suit each individual user. Your legs will, of course, sweat to a certain extent in the cuffs. There is no reason to worry – this is a desired effect that will improve the metabolism in your legs!

Using your VeinWalker[®] pro

If the medical grounds warrant it, please consult your doctor.

When you use your VeinWalker[®] pro, we recommend that each session should last 20 to 30 minutes – depending on how you feel. If you have an unpleasant feeling during the session, reduce the pressure or break off treatment.

After each session is over, first of all turn the left switch in an upward direction . The air will now be sucked out of the cuffs. After approximately one minute you can open the zip and take off the cuffs. Let the pump continue to run until the air is completely pumped out of the cuffs. This means you do not have to go to all the trouble of pressing the air out yourself. Once the cuffs have been pumped empty, turn your VeinWalker[®] pro off . Now you can remove the hoses from the control unit.

If necessary, clean the cuffs and then carefully fold them up. Try to avoid bending the cuffs too sharply – this could ultimately cause the cuffs to split after being used for some time.

Cleaning and looking after your VeinWalker[®] pro

Before cleaning your VeinWalker[®] pro, you should always disconnect the hoses from the control unit and the cuffs. Should you sweat a little under the cuffs, you can clean them with a dry (or slightly moistened) cloth and some cleanser. When the cuffs are applied directly on the skin, you should disinfect them with a commonly available disinfectant so as to avoid cross-contamination if they are to be used by several different individuals. But never use astringent cleansers. In particular be sure that no moisture finds its way into the openings for letting air in and out. You should make a point of plugging the openings with the stoppers when you clean the cuffs.

After your VeinWalker[®] pro has been in use for a long time, the ends of the hoses could begin to widen out with the result that they are no longer airtight. In such a case you can cut a short piece off the ends.

Fold up the cuffs only loosely. If the cuffs are bent too forcefully, they could split. We recommend cleaning the control unit only when it is necessary. Clean the control unit in the same way as you clean the cuffs. The cuffs can be cleaned after each session or at regular intervals.

Always remember to pull the plug out before you clean your VeinWalker[®] pro!

Troubleshooting

Your VeinWalker® pro is easy to use, but in case...

...the cuffs are not pumped up completely

Before use, make sure that the tubes are securely attached to the valves and connected correctly to the control device.

Check whether the directional switch for the pump is set to “inflate”

...Check whether the pressure regulator pump is set to “maximum”.

...the cuffs are not emptied completely or not quickly enough:

Check whether the directional switch for the pump is set to “deflate”.

Check whether the pressure regulator pump is set to “maximum”.

On page 30 “Important tips, How to get started” and page 31 “Operating your VeinWalker® pro”, is an exact description of how you should use your VeinWalker® pro.

We wish you a pleasant and successful treatment!

Hotline

Should you have questions about the product, you can contact us from Mon-Fri by calling the following service number: +49 (0) 38851 314337 (subject to charges).

Additional information about the VeinWalker® pro

Advantages

- Easy, safe way of increasing the effectiveness of pressure bandages in the treatment of congested veins
- Easy to use
- Cuffs covering the leg from the foot up to the groin are rhythmically in- and deflated
- The cuffs work automatically
- The leg is gently massaged
- The pump-like mechanism of the muscles in the leg is copied
- Not only for cosmetic problems and sport injuries
- Also suitable for elderly or overweight patients
- Can be used daily in the comfort of your own home
- Constantly accelerates the flow of blood in the veins for several hours
- Reduces swelling, pain and the feeling of heaviness in the legs

How to operate and use your VeinWalker® pro

- Place the control unit (on a non-slip surface) on the floor!
- Set the switch for changing the direction of the pump (on the left-hand side of the unit) to the desired pump direction by pushing up or down gently.
- Set the pressure regulator (on the right-hand side of the unit) to maximum or minimum.
- The cuffs are inflated or deflated automatically; this may take several minutes
- The cuffs should be carefully folded up after use – do not bend!
- Control unit should not be wet-cleaned!

Use your VeinWalker[®] pro at home for

- Sport injuries
- Aching muscles (especially following sport)
- Bruises
- Sprains
- Haemorrhages
- Lipedema
- Thread veins
- Varicose veins
- Minor congestion in the veins
- Swellings
- Feeling of heaviness
- Minor circulatory disturbances of the arteries

If you suffer from a serious illness, please consult your doctor before!

Intermittent compression has been proven to be effective in cases of:

Disorders of the veins, arteries, lymph and connective tissue

What is achieved?

- Swelling goes down
- Relief from pain resulting from congested veins
- Less pressure in varicose veins/thread veins as their cross-section is
- Reduced valves in the veins close better
- Flushing-out of fluid in body tissues > the firming of body tissues
- Supplement in the treatment of lymph edema

Why do my muscles ache after sport?

There are two theories:

1. There is an accumulation of lactic acid when you exert yourself more than you usually do, and when the blood supply is reduced. Lactic acid irritates the sensitive nerves in your muscles and causes an inflammation which is not caused by bacteria (abacterial inflammation).
2. Numerous microinjuries which cause the abacterial inflammation.

What can I do when my muscles are aching?

Keep the aching limbs cool, move about and massage the affected area with your VeinWalker® pro.

Treatment with the VeinWalker® pro:

Twice a day / 15–20 minutes, until you notice an improvement

What kind of sports injuries are there?

- Swellings and inflammations
- Bruises
- Haemorrhages
- Sprains
- Pulled muscles

What can I do when I injure myself?

Ice pack or ointment for sport injuries in combination with your VeinWalker® pro.

Treatment with the VeinWalker® pro:

Twice a day / 15–20 minutes, until you notice an improvement

What is a “minor congestion in the veins”?

(only a doctor can diagnose and/or confirm it)

- Congestion in the veins without inflammation
- Slight swelling around the ankle in the evening
- There is no feeling of pain and pressure in the lower leg tissue
- The calf feels swollen
- You notice a sensation of pain and pressure in the shin
- The swelling/ feeling of heaviness improves during the night or when you move around

What can I do?

Put your foot/feet up, cool the injury, move around, use your VeinWalker® pro

Treatment with the VeinWalker® pro:

Long term, daily / 20–30 minutes

Technical data

- Pressure: 80 mmHg or 107 millibars
- Power supply: 230 volts
- Frequency: 50 hertz
- Operating current: 0.1 A
- Temperature
 - Use: 15–40 °C
 - Storage and transport: 0–40 °C
- Air humidity
 - Use: 30–75 %
 - Storage and transport: 10–80 %
- Dimensions: 221 mm (L) x 163 mm (W) x 165 mm (H)
- Weight: approximately 2,300 gram

- Fuse: T 0.315 A H
- Degree of protection: type B ⤴
- Protection class: II 
- Alternating current: ~

Please be sure to read the instruction manual!

Power ON

Power OUT

Inflate cuff

Deflate cuff

In line with the Medizin-Produkte-Gesetz (MPG)
(German medical device directive)

The switch jumps into the right position with mild pressure

Please ensure proper disposal at a collection point for electronic waste.

Any repairs or maintenance work including replacing parts should only be carried out by persons authorised by the manufacturer.

Note on emitted interference and interference resistance according IEC 60601-1:

Emitted interference: Group 1; Class B according to EN 55011

Interference resistance: non-life-preserving devices according to IEC 60601-1-2

The VeinWalker[®] pro can cease to function correctly if it experiences interference from portable and mobile transmitting devices.

Explanation of symbols used

Devices bearing this symbol were put into circulation after 13.08.2005 (DIN EN 50419). Devices bearing a later symbol fulfil the designation requirements of all EU member states.

Read and follow the device instructions carefully and keep them in a safe place.

This product may only be used in dry areas.

Symbol for devices of Protection Class II. These devices have a reinforced or double insulation and are not connected to the protective conductor – also called protective insulation. Movable devices of Protection Class II do not have a safety plug.

CE symbol. The CE label allows the manufacturer or importer to confirm the product's compliance with the relevant EU directives and adherence to the "essential requirements" these stipulate. The number 0123 indicates the section (certifier). The two together confirm adherence to the Medizin-Produkte-Gesetz (MPG) (German medical device directive) by the distributor.

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consumer electronics GmbH
06.2017 GlobalMIND GmbH, Ernst-Mantius-Str.11, 21029 Hamburg – Germany