••• VENEN **WALKER**[®] PRO² •••

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Operating instructions

Mode d'emploi

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DE

EN

FR

NL

PL

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Contents

Introduction _____	30
Brief description _____	31
Purpose _____	32
Use _____	32
Safety instructions _____	34
Items supplied _____	37
Control unit at a glance _____	37
Use _____	38
Cleaning, care and storage _____	45
Troubleshooting _____	46
Technical specifications _____	48
Permitted accessories _____	49
Disposal _____	49
Customer service _____	49

IMPORTANT NOTICES! PLEASE KEEP FOR REFERENCE!

Before using the device for the first time, please read through these operating instructions carefully and keep them for future reference and other users. They form an integral part of the device.

The manufacturer and importer do not accept any liability if the information in these operating instructions is not complied with.

Explanation of the symbols

Danger symbols: These symbols indicate hazards. Read through the associated safety instructions carefully and follow them.

Supplementary information

Read operating instructions before use! This symbol appears in blue on the device.

Protection class II

Symbol for devices classified as "Type BF" (level of protection against electric shock)

Manufacturer

Serial number (Date of production / consecutive number)


Circuit symbol for alternating current

Devices bearing this symbol were put into circulation after 13.08.2005 (DIN EN 50419). Devices labelled in this way meet the labelling requirements of the EU member states.


International recycling symbol

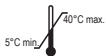Temperature range

Humidity range

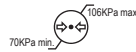Air pressure range

Keep dry

Fragile

EC REP EU Representative

CE 0123

With the CE mark, the manufacturer confirms that the product meets the requirements of the applicable EC directive and complies with the "general requirements" stipulated therein. The number 0123 alludes to the notified body (certifier). Together these certify that the German Medical Devices Act (MPG) has been observed by the distributor.

Explanation of signal words

DANGER warns of serious injuries and danger to life

WARNING warns of **potential** serious injuries and danger to life

CAUTION warns of slight to moderate injuries

NOTICE warns of damage to property

Dear Customer,

We are delighted that you have opted for this **VENENWALKER® PRC²**. This device will help you prevent venous diseases as well as reducing the risk of spider veins and varicose veins. If you have a medical indication, please consult your doctor before use.

The **VENENWALKER® PRC²** is a medical product certified according to DIN EN ISO 13485:2012 and meets the necessary safety and performance requirements according to MDR:2017: (Appendix I)

- DIN EN 60601-1:2013-12; VDE 0750-1:2013-12 - Part 1: IEC 60601-1:2005 + Cor. :2006 +Cor. :2007 + A1:2012; German version EN 60601-1:2006 +Cor. :2010 + A1:2013
- DIN EN 60601-1-2:2016-05; VDE 0750-1-2:2016-05- Part 1-2: IEC 60601-1-2:2014 German version EN 60601-1-2:2015
- DIN EN 60601-1-6:2016-02; VDE 0750-1-6:2016-02- Part 1-6: IEC 60601-1-6:2010 + A1:2013; German version EN 60601-1-6:2010 + A1:2015
- DIN EN 60601-1-11:2016-04; VDE 0750-1-11:2016-04 Part 1-11: IEC 60601-1-11:2015; German version EN 60601-1-11:2015;
- REACH EG 1907/2006; EU 2011/65/EU (RoHS II);
- EN ISO 10993-1

If you have any questions about the device or about spare parts / accessories, please contact the customer service department via our website:

www.venenwalker.com or by phone on +49 (0) 38851 314337*

(Mon – Fri, 8 am – 12 midday)*

All the very best with the **VENENWALKER® PRC²**
Yours sincerely,

GlobalMIND!

The illustrations in these instructions may differ slightly from the actual device.

* Calls to German landlines are subject to your provider's charges.

Introduction

Too much sitting, too much standing, the wrong diet: according to consistent estimates, 20 % of all Europeans suffer from venous leg complaints and more than half are affected by advanced vein problems.

This global lifestyle disease starts off harmlessly with a tingling sensation in the legs. The legs then become heavy, swollen and tired, and spider veins and varicose veins begin to appear.

How does a venous disorder occur? Veins collect the used blood in the body's tissue and transport it back to the heart. Leg muscles and venous valves tense up and relax, thus acting like a pump to transport the blood in the veins upwards to the heart. However, if a person stands or sits for a long time, or if excess strain due to surplus weight or genetic predisposition leads to congestion, the blood cannot flow away properly and the venous walls become overstretched. Swollen legs are the result: these can form an oedema due to the fact that liquid and protein pass through the slackened venous wall and into the tissue. A doctor should be consulted as a matter of urgency when such venous congestion becomes a continual complaint.

Special devices for alternating compression of the extremities have proved highly effective and time-saving in treating venous problems, as well as in combating lipoedema.

The **VENENWALKER® PRC²** is such a device. It has been developed in close collaboration with medical specialists (phlebologists) and is easy to handle.

The legs are lightly massaged by the **VENENWALKER® PRC²** using an inflating and deflating air cushion which simulates the pumping effect of the leg muscles experienced during walking exercises. Venous reflux is improved so that accumulated fluid is mobilised and can be flushed out.

The IPC principle

Intermittent (alternating) compression which is not generated through manual massage but rather by a device, is known in medicine as intermittent pneumatic compression, or IPC. It uses the principle of pneumatic alternating pressures. It is prescribed by doctors for the treatment of venous or lymphatic diseases. When used to prevent thrombosis or for sports injuries, it does not involve the risks found in medication-based treatment.

The IPC simulates the function of the natural muscle pumps found in the human body. Air is pumped with alternating pressure into double-walled leg cuffs for example. The pressure is increased and decreased at defined intervals. It should be possible to adjust the pressure level to suit the user and the pressure must never exceed 120 mmHg. When used for cosmetic purposes, treatment duration and frequency depend on the person's condition; in the case of medical

application, the doctor's recommendation should be followed. The treatment should last for at least 20 minutes, however. Any medical use should, of course, be supervised by a doctor.

The shape and type of cuff used is important. Due to the proven importance of the draining of the vein network in the sole, the cuff should cover both the entire leg and the foot.

The rhythmic increase and decrease in pressure on the tissue leads to water and protein being pumped out of the interstitial tissue.

If the patient has a medical indication, a compression dressing should be applied or a surgical stocking should be worn after using the IPC as the residual protein causes blood serum to flow from the capillaries into the tissue after an IPC treatment session. This measure prevents this flow occurring. Alternatively, it is possible to remain in a lying position so as to prevent this subsequent flow of blood serum. So when using the equipment at home, it makes sense to perform the treatment before bedtime.

Brief description

The **VENENWALKER® PRC²** is a system comprising a control unit and two double-walled leg cuffs with three interior chambers which also cover the feet and soles. Optional cuff extensions can be used in order to enlarge the circumference of the cuffs.

The control unit feeds air into the chambers of the leg cuffs via connection tubes. This enables controlled pressure to be applied so as to pump up the cuffs gently and/or massage the legs.

What you can expect from the VENENWALKER® PRC²

In addition to its function, with the **VENENWALKER® PRC²**, as with all our products, we attach particular importance to user safety. We ensure through design features that the pressure that the cuffs place on the leg cannot exceed 120 mmHg. A higher pressure may have an effect on the arterial blood supply. Even if parts of the device should be defective or not working properly, this pressure cannot be exceeded.

Purpose

The VENEN**WALKER**[®] PRC² may only be used for decongestion of the extremities. If you have a medical indication, please consult your doctor before use!

The VENEN**WALKER**[®] PRC² is designed for private use at home.

Only use it as described in the operating instructions. Any other use is deemed to be improper.

The following are excluded from the warranty: all defects caused by improper handling, damage or unauthorised attempts at repair. The same applies to normal wear and tear.

Use

The VENEN**WALKER**[®] PRC² is designed in such a way that the user does not require any special knowledge or skills.

The VENEN**WALKER**[®] PRC² may be used up to 3x per day for 20 to 40 minutes on each occasion. The system is suitable for long-term use.

The indications and contraindications listed below are taken from the current guideline for apparative intermittent compression (AIC or IPC) issued by the German Society of Phlebology. For more detailed information, please consult the "S1 Guideline 037-007, Intermittent Pneumatic Compression (IPC, AIC)" version 20.06.2017 incl. editorial revision SR 11.01.2018, published by AWMF online – the scientific medical platform.

Indications

Apparative intermittent compression (AIC) is designed for the treatment of the following symptoms:

- Leg swelling caused by lack of movement, for example due to standing for long periods of time in the case of sales personnel, due to sitting for extended periods when doing office work or due to long-haul travel
- Swollen legs due to being bedridden or as a result of pregnancy**
- Swollen legs following a medical diagnosis
- Venous diseases such as varicose veins and spider veins
- Lipoedemas
- Prevention of embolisms due to being bedridden
- Lymphatic system swelling (lymphoedema)
- Lower-leg ulcers caused by a venous circulation disorder
- Congestion due to joint injury or overexertion, e.g. haematoma and sore muscles in the event of sport injuries

** Please consult your doctor.

- Swelling following a pulled muscle or sprain
- To assist the wound healing process after an operation
- Peripheral arterial occlusive disease, with certain indications**

Contraindications

Please do **not** use the VENEN**WALKER**[®] PRC² in the following instances, or only after consultation with your doctor:

- Decompensated cardiac insufficiency
- Extended thrombophlebitis, thrombosis or suspicion of thrombosis
- Acute erysipelas
- Severe, unmanaged hypertension
- Pronounced neuropathy of the extremities
- Acute phlegmons
- Compartment syndrome
- Extended or open soft-tissue trauma of the extremities
- Occluding processes in the the lymph drainage area where AIC has caused congestion in the groin and genital area
- Blistering dermatoses

If you are not sure whether you suffer from one of the above illnesses, consult a doctor before using the system!

Precautionary measures

The VENEN**WALKER**[®] PRC² is to be used with care on patients with the following symptoms or states:

- Peripheral neuropathy, pain or numbness of the limbs
- Undiagnosed, untreated or infected wounds, inflamed skin, transplantations or dermatological conditions which are exacerbated by the cuff
- Extremely deformed limbs which prevent the correct application of the cuff in practice

Please report to the manufacturer all severe incidents that occur in connection with the system.

Safety instructions

- This system is **not** designed to be used by individuals (including **children**) with limited sensory or mental abilities or those without sufficient experience and / or knowledge, unless they are supervised by someone responsible for their safety or have received instructions from them on how to use the system.
- Children must **not** play with the device.
- Cleaning may not be carried out by **children**.
- Children under 8 years of age and animals must be supervised near the system.
- Always check the system for damage and wear before putting it into operation. It may only be used if it is undamaged and does not display any signs of wear and tear.
- Electrical and medical devices can be dangerous if used improperly. The control unit does not have any parts which can be serviced by the user. The casing of the control unit may not be opened. Under no circumstances may the control unit be modified!
- If components of systems are damaged, they must be replaced by the manufacturer, aftersales service or authorised technical personnel or else repaired in order to avoid hazards.
- The socket where the control unit is plugged in must be readily accessible so that it can be quickly disconnected from the mains.
- The control unit is not designed to be operated with an external timer or a separate remote control system.
- The control unit may only be used with accessories approved by the manufacturer, and the accessories may not be used with other compressors. Otherwise it is not possible to guarantee that the system will work correctly, and all guarantee and warranty claims will be invalid.

DANGER – risk of electric shock

- Use and store the system only in closed rooms. Do not operate the system in rooms with high humidity.
- Never immerse the system and its mains cable in water or other liquids and ensure that they cannot fall into water or become wet.
- Should the control unit fall into water, switch off the power supply immediately. Do not attempt to pull the control unit out of the water while it is still connected to the mains power! Have the control unit checked in a specialist workshop before using it again.
- Never touch the control unit or its cable with wet hands when these components are connected to the mains power supply.

WARNING – risk of injury

- Danger of suffocation! Keep plastic film and bags well away from children and animals.
- **Risk of strangulation!** Make sure that the mains cable and the connection tubes are always kept out of the reach of small children and animals.
- **Danger of tripping!** Take care to ensure that the mains cable and connection tubes do not create a trip hazard.
- Sleeping or unconscious people may not be treated with the system.
- Treatment should be interrupted immediately if pain, a tingling sensation or numbness occurs in the limbs during or after treatment.
- In the event of a power cut or disruption in which the cuff remains inflated, disconnect the connection tubes so as to let the air out of the cuff(s) and remove the cuff(s) from the limbs.

WARNING – danger of fire

- Do not use the system in rooms containing easily ignitable or explosive substances.
- Do not operate the control unit in the vicinity of combustible material. Do not place any combustible materials (e. g. cardboard, paper or plastic) on the control unit.
- Do not cover the control unit during operation in order to prevent it from catching fire. Do not insert anything into the ventilation openings of the control unit and make sure that these do not become clogged.

NOTICE – risk of damage to material and property

- Keep the cuffs and connection tubes away from sharp objects. Do not insert safety pins or other pins or sharp objects into the cuffs.
- Only plug the control unit into properly installed sockets where the mains voltage corresponds to the voltage specifications on the rating plate of the control unit. Only use appropriate extension cables which comply with the technical specifications of the control unit.
- Unwind the mains cable fully before connecting the control unit.
- Make sure that the mains cable and connection tubes are not crushed, bent or laid over sharp edges and do not come into contact with hot surfaces.
- The control unit must be switched off when inserting the mains plug in the socket or removing it.
- Remove the mains plug if a fault occurs during operation or before a thunderstorm.
- When removing the mains plug from the plug socket, always pull on the mains plug and never the mains cable. Do not pull or carry the device by the mains cable.
- Do not use the control unit if it has malfunctioned, fallen or been dropped in water. Have it checked in a specialist workshop before using it again.
- Always place the control unit on a dry, even and firm surface. Do not place on cushions or other soft surfaces in order to prevent the control unit from overheating.
- Do not place the system on or next to heat sources such as hobs or ovens. Keep the system away from open fire such as candles or cigarettes.
- Do not expose the system to extreme temperatures, persistent moisture, direct sunlight or impacts. Please refer to the chapter entitled “Technical specifications” for details.
- Never place heavy objects on the system.

Electromagnetic Compatibility (EMC)


For details, please refer to the back-up documentation or <https://www.venenwalker.com/downloads/>

Items supplied

- A** Cuffs (2x)
- B** Control unit (1x)
- C** Connection tubes (2x)
- D** Storage bag
 - Operating instructions (1x)

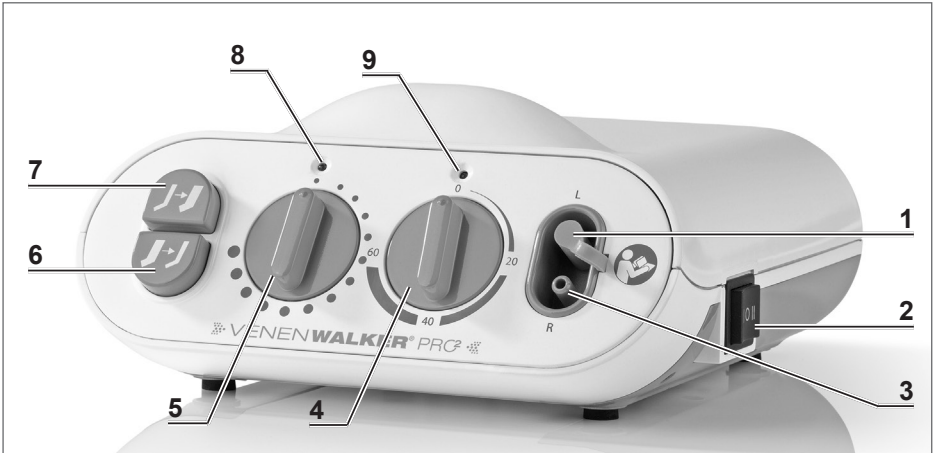Control unit at a glance

- | | |
|--|--|
| <p>1 Seal plug for tube connectors (here: in L tube connector for left leg)</p> <p>2 I O II Main switch</p> <p>3 R Tube connector for right leg</p> <p>4 MINUTES Rotary knob: Timer, time in minutes</p> | <p>5 PRESSURE Rotary knob: Pressure regulator</p> <p>6  Button: Extraction</p> <p>7  Button: Pump</p> <p>8 Green operating lamp</p> <p>9 Blue timer lamp</p> |
|--|--|

Use

PLEASE NOTE!

- In the event of **medical indication**, ask your doctor whether and how often you are allowed to use the VENEN**WALKER**[®] PRC².
- If use of the system is **unpleasant**, reduce the pressure or terminate treatment.
- If an **increased pulse rate or breathing difficulties** occur in pregnant women, use of the system must be interrupted. If this happens, please contact your doctor.
- **Do not walk or stand** while wearing the cuffs!
- Take care to ensure that the **cuffs and connecting tubes are not bent or twisted** so as to avoid impairing the system and damaging the air chambers!
- **Do not open the cuffs while using the system** in order to avoid damaging the cuffs.
- Take care to ensure that the **control unit is free-standing** and not covered with a blanket or other objects.
- Make sure the system is clean and dry prior to storage or use.

Mode of operation

The VENEN**WALKER**[®] PRC² works based on apparatusive intermittent compression (AIC). Your legs are massaged by the rhythmic, alternating inflation and deflation of the air cushions. The cuffs exert a regularly interrupted pressure from outside on the capillary pump in the calf muscles and feet. This allows venous blood to be pumped towards the heart. Furthermore, water inside the tissue and the metabolic particles dissolved in it can be flushed out and eliminated from the body.

Notices

- Your legs should be horizontal or raised when using the system so as to allow the blood to circulate unhindered.
- We recommend using the VENEN**WALKER**[®] PRC² every day for approx. 20 to 30 minutes in the morning and evening.
- Never use the VENEN**WALKER**[®] PRC² for long periods without interruption since excessive massage can be strenuous. When you use the system for the first time, limit yourself to 20 minutes. If you are comfortable with a period of 20 minutes, you can extend subsequent treatments to approx. 30 minutes.
- In the event of medical indication, apply a compression bandage or compression stocking immediately after using the system or remain lying down

for a while in order to prevent blood serum from subsequently flowing from the capillaries into the tissue as far as possible.

- To make it easier to put on compression stockings, it can be easier to use the **VENENWALKER® PRC** beforehand.
- In the event of overload, the zips on the cuffs open so as to prevent a blockage of the arteries.
- If the circumference of your thighs is greater than approx. 70 cm or that of your lower leg is greater than approx. 40 cm, you will require cuff extensions. These are available at www.venenwalker.com.

A) Setting up the control unit

1. Choose a place where you can sit comfortably while putting up and stretching your legs, e. g. on a sofa, bed or armchair with a footrest.
2. In order to be able to operate the control unit sitting down, place it next to you on a side table or chair.
3. Insert the mains plug of the control unit into an easily accessible socket. If there isn't one nearby, use an extension cable.

B) Applying cuff extensions

A cuff extension is 10 cm wide at the widest point (thigh). If this is not enough, you can connect several cuff extensions to one another.

Connecting cuff extensions to one another: Join the zip halves at the narrow ends of the cuff extensions and close the zip.

Attaching cuff extensions to the cuff:

- Open the zip of the cuff.
- Join the zip halves of the cuff and the cuff extension and close the zip.

C) Connect the cuffs to the control unit and apply

The VENENWALKER® PRC² can also be used by applying one cuff to one leg.

If you are wearing trousers, take them off before applying the cuffs so as to ensure the treatment is effective and not unpleasant. **If you are wearing tights or compression stockings, you can keep these on.**

1. Spread out the cuffs at the point where the treatment is to be carried out.

2. Pick up the connection tubes. Attach the tube ends to the tube contacts on the cuffs.

3. Open the zips of the cuffs.
4. Sit down and place your legs in the cuffs.
5. Close the zips of the cuffs. If you have problems, read the tips below about how to put on the cuffs.

6. Put the right-hand connection tube on the control unit tube connector marked **R** and put the left-hand connection tube on the control unit tube connector marked **L**. Make sure the tubes are firmly attached to the tube connectors.


If you are only using one cuff, close the unused control unit tube connector with the seal plug.

Tip for putting on the cuffs


If the cuffs are already tight around your legs before they are pumped up be sure to use cuff extensions. Otherwise the cuffs and zips might be damaged. Several cuff extensions can be used at the same time in order to achieve the necessary circumference. For details, read Section B) of this chapter: “Applying cuff extensions”.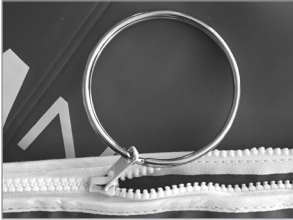

If you find it difficult or impossible to hold the zip slider we recommend attaching a (key) ring (see picture).

If you cannot lean forward enough to close the cuffs, try the following:

- Ask a second person to help. Since the cuffs can be fully opened and unfolded, a second person can easily position them under your legs and close the zips.
- Before applying the cuffs, close the zips up to approx. the middle of the cuffs (or as far as necessary). Then sit down, slip into the cuffs and close the zips completely.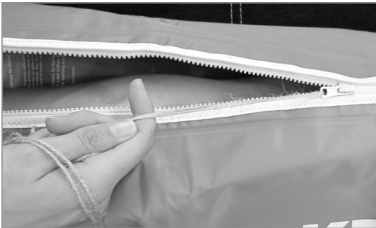

- If it is still too difficult, you can attach a piece of string to the zip slider and close the zip by pulling the string.

D) Switching on the control unit to start massaging


The VENENWALKER® PRC² massage should be terminated after approx. 20 – 30 minutes. You can either switch off the control unit manually or activate the time function prior to switching on. In the latter case, the control unit switches off automatically after the set period of time.

We recommend using the timer function. This means you do not have to watch the time and can relax completely during the treatment.

1a. Switching on **WITHOUT** timer:

Press the main switch to position **I** – the green operating lamp goes on.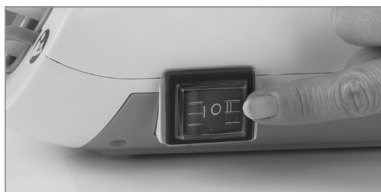

1b. Switching on **WITH** timer:

Press the main switch to position **II** – the blue timer lamp goes on.

Use the **MINUTES** rotary knob of the timer to set the required period (between 0 and 60 minutes).

2. Turn the **PRESSURE regulator** clockwise to the **highest level** so that the pumping process does not last longer than 10 minutes.

3. Press the **J-J** button to pump up the cuffs. The control unit begins to fill the cuffs alternately with air. The legs are then massaged by the inflating and deflating air cushions.

Please note that the cuffs are not activated simultaneously but consecutively. The pressure is built up in one cuff first and then in the other.

- Stretch your legs and take up a comfortable position.

In order to ensure the blood can circulate unhindered, it is essential to ensure that the leg in the cuff is stretched out during treatment!


- If the pressure exerted on your legs is too powerful or unpleasant, simply turn down the **PRESSURE** regulator until the pressure is reduced. The ideal pressure on your legs should be powerful but not unpleasant.

It is natural for your legs to sweat in the cuffs. This is not a cause for concern: in fact it is an intended effect which stimulates metabolism in the legs.

E) Finishing the treatment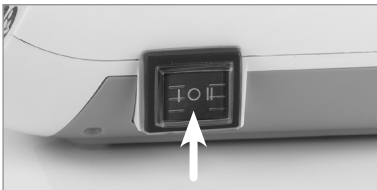

- Switch the control unit off **after 20 to 30 minutes**. For this purpose, press the main switch to the central position **O**.

If the timer has been activated, the control unit switches off automatically after the set period. After this, press the main switch to the central position **O**.

- Wait until the cuffs have loosened. You can then open the zips and remove your legs from the cuffs.

OR: Press the main switch to position **I** after treatment and press the **J** extraction switch. This starts air extraction from the cuffs. For details, read the following section "After treatment".

F) After treatment

You can leave the VENENWALKER® PRC² where it is for the next treatment. Be sure to remove the mains plug from the socket, however.

If you have been sweating, clean the cuffs (see the “Cleaning, care and storage” chapter).


If you wish to place the VENENWALKER® PRC² in the storage bag ...

1. ... press the main switch to position **I**.

2. Press the **J** extraction switch and turn the **PRESSURE** regulator to the highest level.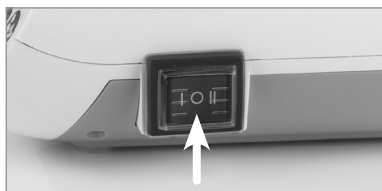

3. When the cuffs are completely deflated, press the main switch to the central position **O** to switch off the control unit.
4. Disconnect the control unit from the mains.

5. Remove the connection tubes from the control unit and the cuffs. Loosely roll up the connection tubes.

6. Carefully fold the cuffs. **Avoid sharp bending, otherwise breakage points may appear in the cuffs after frequent use.**

7. Place all parts in the storage bag.

Cleaning, care and storage

PLEASE NOTE!

- Before cleaning, always remove the connection tubes from the cuffs and the control unit.
- Do not use any caustic or abrasive cleaning agents for cleaning. This might damage the surfaces.
- It is imperative to ensure that no moisture enters the tube connectors on the cuffs and the control unit.
- Always remove the mains plug from the socket before cleaning the control unit.
- Allow all parts to dry completely before they are stored away. Allow to dry in the air only or wipe with a cloth. Never use a hair drier or similar for drying purposes!

1. Clean the cuffs with a clean, damp cloth and if necessary a mild household cleaning agent.
2. If the cuffs are used directly against the skin by more than one person, they should be cleaned using a commonly available disinfectant after use so as to avoid cross-contamination.
3. The control unit and connection tubes should only be wiped with a clean, damp cloth if necessary. You can also use a mild household cleaning agent.
4. Allow all parts to dry in the air after cleaning or use a cloth.
5. Store the **VENENWALKER**[®] PRC² in the storage bag if you will not be using it for a lengthier period of time. Keep the storage bag in a cool, dry place which is protected from direct sunlight and frost; it must be kept out of reach of children and animals.

Do not place anything on top of the storage bag so that it doesn't get crushed and the **VENENWALKER**[®] PRC² is not damaged!

After frequent use, the connection tubes may widen at the ends and no longer fit tightly. In this case we recommend cutting off the ends.

Troubleshooting

If the system stops working properly, check whether you are able to correct the problem yourself based on the tips below. If this is not possible, contact the customer service department.

Do not attempt to repair a electrical device yourself!

Problem	Possible cause	Fix
The cuffs do not pump up within 15 minutes.	Air escapes between the connection tubes and tube connectors.	<ul style="list-style-type: none"> • Push the ends of the connection tubes onto the tube connectors of the cuffs and control unit as far as they will go. • Check to see if the ends of the connection tubes are firmly attached to the tube connectors or whether they have widened. If they have widened, cut off the widened ends.
	The pump is not activated.	Press the  button to pump up the cuffs. Set the PRESSURE regulator to the highest level.
	The air input is obstructed.	Make sure the connection tubes and cuffs are not bent or twisted.
The cuffs do not deflate completely.	The air extraction function is not activated.	Press the  button to extract air from the cuffs. Set the PRESSURE regulator to the highest level.
	The air flow is obstructed.	Make sure the connection tubes and cuffs are not bent or twisted.

Problem	Possible cause	Fix
<p>The control unit cannot be switched on.</p>	<p>The timer function is activated but no period of operation is set.</p>	<p>Check the position of the main switch. If it is in position II, the timer function is activated. In order to start the control unit you have to set the MINUTES rotary knob of the timer to the required period of operation.</p> <p>If you do not wish to use the timer function, press the main switch to position I.</p>
	<p>The control unit is not connected to the mains.</p>	<p>Insert the mains plug into the socket.</p>
	<p>There is no mains voltage.</p>	<ul style="list-style-type: none"> • Try another mains socket. • Check the fuse of your mains connection.
	<p>The fuse in the control unit has been tripped due to overvoltage (e. g. lightning strike).</p>	<p>Contact the customer service department.</p>
<p>The cuffs open by themselves during use.</p>	<p>In the event of overload, the zips on the cuffs open so as to prevent a blockade of the arteries.</p>	<p>Use cuff extensions. These are available at www.venenwalker.com.</p>

Technical specifications

Product name:	VenenWalker PRO2
Model number:	001-1011-00
Reference article no. for customer service:	04318
Mains voltage:	230 V ~
Mains frequency:	50 Hz
Current consumption:	0.1 A
Power:	25 W
Cuff pressure:	max. 120 mmHg
Weight of control unit:	1700 g
Fuse:	T3.15 A H250 V (5 x 20 mm)
Electric shock protection class:	Type BF 
Fluid ingress protection class:	IP21
Protection class:	II
Temperature range:	Operation: +5 – +40 °C Transportation: -25 – +70 °C
Air humidity:	Operation: 30 – 75 % Storage and transportation: max. 90 %
Air pressure:	Operation: height up to 2000 m, 70 – 106 KPa Storage and transportation: 70 – 106 KPa
Dimensions:	Control unit: 230 mm (L) x 225 mm (W) x 90 mm (H) Cuffs: 85 cm (L), thigh circumference approx. 70 cm, calf circumference approx. 42 cm Connection tubes: 160 cm
Cuff material:	Nylon (outside and inside)
Service life of the system:	5 years
Serial number:	201903/123456 (201903 – date of production in March 2019 / 123456 – consecutive number)
ID operating instructions:	Z 04318 M GM V3 0520 (05 – month; 20 – year)

Permitted accessories

- VenenWalker PRO leg cuffs, standard pair / red; article no.: 003-1008-32
- VenenWalker PRO leg cuffs, short pair / red; article no.: 003-1008-27
- VenenWalker PRO cuff extension, standard pair / red; article no.: 003-1008-22
- VenenWalker PRO cuff extension, short pair / red; article no.: 003-1008-23
- VenenWalker PRO arm cuffs, pair / red; article no.: 003-1008-14
- VenenWalker PRO leg cuffs, standard pair / turquoise; article no.: 002-1008-11
- VenenWalker PRO cuff extension, pair / turquoise; article no.: 002-1008-10

Disposal

Dispose of the control unit in an environmentally friendly manner at a recycling centre for used electrical and electronic devices.

Customer service

If you have any questions on the **VENENWALKER®** PRC², contact us via our website www.venenwalker.com or by telephone on +49 (0) 38851 314337* (Mon – Fri, 8 am – 12 midday)

* Calls to German landlines are subject to your provider's charges.

Manufacturer within the meaning of the Medical Device Directive (MDD)

Zero-Plus International Limited
 Room 1004, 10/F, Join-In Hang Sing Centre
 71-75 Container Port Road
 Kwai Chung. N.T.
 Hong Kong

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GlobalMIND GmbH
DS Produkte GmbH