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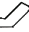
Please note the following points on handling the VeinWalker® pro

Basic points on setting up and connecting your VeinWalker® pro

- The leg cuffs are designed for a maximum thigh circumference of 72 cm. Damage to the cuff caused by application to larger circumferences does not fall under the warranty!
- Always place the device on the floor!
- The supplied tubes are connected directly to the valves on the leg cuffs and to the control device. Before use, make sure that the tubes are securely attached to the valves and connected correctly to the control device.
- Make sure that the tubes do not get bent during use.
- If using one cuff only, the open outlet – on the rear of the device – must be closed!
- Please note that the VenenWalker® inflates the cuffs in intervals alternately. Depending on the leg circumference, this may take around 5 to 8 minutes.

Switch for changing the direction of the pump

- set the switch for changing the direction of the pump (on the left-hand side of the unit) to the desired pump direction by pushing up or down gently.
- The cuffs are pumped in the  position!
- The cuffs are deflated automatically in the  position!

The regulator

- The right-hand regulator serves to regulate/set pressure!
This regulator allows you to adjust the pressure to continuously variable levels.
This regulator moves easily and may **not** be forced beyond tangible resistance.

The cuffs

- The zips must be **completely** closed prior to pumping!
- Do **not** open the zips when the cuffs are inflated!
- Fold the cuffs carefully after use – do not crease sharply!

For detailed information see the manual!

Hotline

Should you have questions about the product, you can contact us from Mon-Fri by calling the following service number: +49 (0) 38851 314337 (subject to charges).

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DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Allemagne
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland