



As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

**1** Remove the Smart-Tip<sup>™</sup> probe from test unit.

**2** Sample suspected substance.

Plant Matter	Hashish
Touch the Smart-Tip <sup>™</sup> probe to the sample. Make sure the tip is completely covered.	Firmly press Smart-Tip <sup>™</sup> probe on the substance while scrubbing in a circular motion until the tip is completely covered with substance.

Test Preparation

**3** Lock Smart-Tip<sup>™</sup> probe into test unit. Press firmly.

Flex Pins

**STOP**  
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Identification

**4** With thumb and forefinger, squeeze chamber 1 until capsule breaks.

**5** Gently shake test unit for 1 minute.

Identification

**6** With thumb and forefinger, squeeze chamber 2 until capsule breaks.

**7** Gently shake test unit for 10 seconds.

Wait up to 3 minutes for the color to fully develop. The color may develop sooner.

Results

If a blue-violet or purple color develops within three minutes, Marijuana is suspected. Proceed to step 8 to confirm the results.

If no blue-violet or purple color develops, Marijuana and Hashish are not present. Proceed to Disposal.

Confirmation

**8** With thumb and forefinger, squeeze chamber 3 until capsule breaks.

**9** Vigorously shake test unit for 20 seconds.

Hold test unit steady and keep upright, until 2 layers are visibly separated. This may take up to 2 minutes. The process may occur sooner.

Confirmation Results

The appearance of a blue-violet or purple color (even if the color is extremely faint) in the lower layer of liquid within two minutes strongly indicates the presence of Marijuana or Hashish.

\* Color may darken with time.

If the lower layer of liquid does not turn a blue-violet or purple color, Marijuana and Hashish are not present.


Disposal

The test kit is safe to dispose of in the trash.

Important Information

### IMPORTANT INFORMATION

1. IDenta Touch&Know<sup>™</sup> Discreet Drug Test Kits are used to determine the presence of certain drugs in substances such as powders, crystals, granules, flakes, pills, tablets, or liquids.
  2. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional verification.
  3. A color change indicates a positive result for the listed drugs only as indicated in the instructions.
- If there is NO color change as indicated in the instructions, the listed drugs are NOT present in the tested sample.

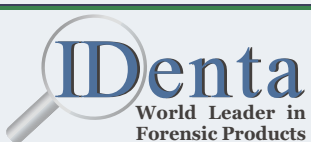

Manufactured by IDenta Ltd  
[www.touch-know.com](http://www.touch-know.com)  
[info@touch-know.com](mailto:info@touch-know.com)