




As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1

Remove the Smart-Tip[™] probe from test unit.

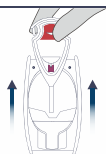


2

Sample suspected substance or surface. (Not for use with plant materials.)

Powder & Trace Material


Swipe the paper of the Smart-Tip[™] probe across the suspect substance or surface.

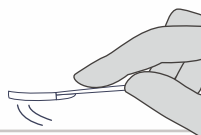

Test Preparation

3

Lock Smart-Tip[™] probe into test unit. Press firmly.

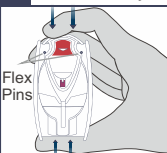


STOP
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Chamber 1 Identification


4

With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

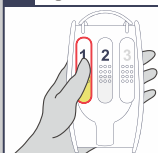

5

Gently shake test unit for 2-3 seconds.

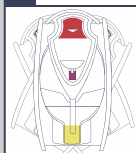

Chamber 1 Results



The appearance of a purple-red color within a few seconds strongly indicates the presence of **Brown Heroin**, if the substance is a brown powder.



The appearance of a green or blue-green color within a few seconds indicates the presence of **Crack (Cocaine Base)**.




If any of the above colors develop, proceed to Disposal. If **none** of the above colors develop, **proceed to Step 6**.

Chamber 2 Identification


6

With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

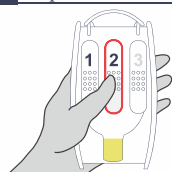

7

Gently shake test unit for 2-3 seconds.

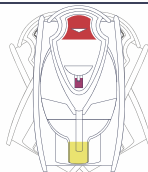

Chamber 2 Results



The appearance of a purple-red color within a few seconds indicates the presence of one of these suspected drugs:

Heroin, Amphetamine, Methamphetamine, Ecstasy/MDMA, Bazedrone, Buphedrone, Carfentanyl, Cathinone, DMT, EAPB, Fentanyl, Ketamine, LSD, mCPP, MDMAI, MDPBP, MDPV, Mephedrone, Methadone, Methcathinone, Methylone, Mexedrone, MPA, PCP, Pentedrone, Pentylone, PMA, PMMA, Psilocybin, 2-aminoindane (2-AI), other synthetic/designer drugs.



The appearance of a green or blue-green color within a few seconds indicates the presence of **Cocaine**.



If **none** of the above colors develop within one minute, the above listed drugs are **not** present.



Disposal

The test kit is safe to dispose of in the trash.

IMPORTANT INFORMATION

1. IDenta Touch&Know[™] Discreet Drug Test Kits are used to determine the presence of certain drugs in substances such as powders, crystals, granules, flakes, pills, tablets, or liquids.
2. **All results are presumptive.** If necessary, further testing by a competent drug testing lab can provide additional verification.
3. **A color change indicates a positive result for the listed drugs only if it occurs within the first minute.**

If there is **NO** color change within the time indicated, the listed drugs are **NOT** present in the tested sample. After a minute, the sampler and the liquid may begin changing color as a normal part of the testing process and this color change does **NOT** indicate a positive result.