# CONNECT WITH US!

 [facebook.com/cambridgemaskco](https://facebook.com/cambridgemaskco)

 [cambridgemask](https://cambridgemask)

 [twitter.com/cambridgemaskco](https://twitter.com/cambridgemaskco)

 [info@cambridgemask.com](mailto:info@cambridgemask.com)

[WWW.CAMBRIDGEMASK.COM](https://WWW.CAMBRIDGEMASK.COM)

CAMBRIDGE MASK CO

**PRO**

---

**USER MANUAL**

# CAMBRIDGE MASK CO

EST. 2015

---

[WWW.CAMBRIDGEMASK.COM](http://WWW.CAMBRIDGEMASK.COM)

 Cambridge Environmental Solutions Ltd, RM 1001-2 10/F Wilson House, 19-27 Wyndham Street Central Hong Kong.

Store in a cool, dry place. Hand wash with warm water and washing soda. Hang dry.      Washing process will only clean the exterior surfaces and will not clean the filter layers.

## CONTENTS

**02** English

**04** Spanish

**06** German

## THANK YOU

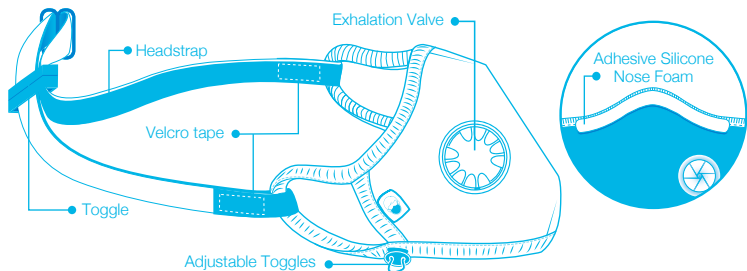
Thank you for choosing Cambridge Mask. We work hard to make sure you and those you care about can get outside and enjoy life without worrying about the air you breathe or the way you look.

If you have any feedback about our masks, please get in touch via [info@cambridgemask.com](mailto:info@cambridgemask.com)

## YOUR CAMBRIDGE MASK

We are proud of the high quality product that we make. As a non-disposable half-face respirator, Cambridge Mask can be worn far longer than most disposable industrial masks. Once the filters are "full" of particles, the mask needs to be changed. Our PRO masks are non-disposable, meaning that you can use them for a maximum of 340 hours or 3-6 months. The length of time they are suitable for depends on how polluted the environment is when you use it. Replace your mask once you feel the breathing resistance increasing significantly from your normal use.

**CONTENTS:** Mask, Instructions, Headstrap, Adhesive Silicone Nose Foam

## FITTING YOUR CAMBRIDGE MASK

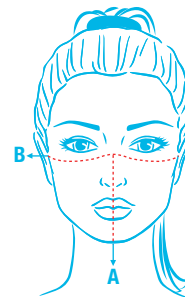
1. The ear straps are designed to fit behind the ears to make the mask comfortable to wear. Please make sure there is a tight seal. Press the mask to your face when you put it on to help form a seal.
2. When you wear the mask be sure to adjust the noseclip and toggles so that there is a tight seal around all of the face. This ensures that the air is flowing through the mask and not around it.
3. Included is a headstrap. If required, affix the headstrap as indicated in the diagram above using the velcro. Then tighten the headstrap as necessary using the toggle at the back.
4. Included is an adhesive silicone nose foam. This is intended for wearers who are finding their glasses are fogging while using the mask. Fix to the inside of the mask, under the noseclip, if required.

## SIZING

Measure your face using the instructions below to select your size.

- A. Measure from the bridge of your nose to 2cm under your chin.
- B. Measure cheekbone to cheekbone for the width of the mask.

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## TESTING

- Particle Filtration Efficiency >99%
- Bacterial Filtration Efficiency >99%
- Breathing resistance <2.4 mbar
- Viral Filtration Efficiency >99%

## STORAGE & GUIDELINES

1. Keep your mask sealed in a cool, dry place with low humidity (-20 C to +30 C).
2. The Cambridge Mask can be gently hand washed with washing soda and warm water to remove dirt from the outer fabric layer.
3. Mask must be fitted properly to filter effectively, Infants and children should be supervised when wearing a mask. Mask does not filter industrial chemicals or oil-based particles such as paint. Do not exceed recommended maximum hours of usage.
4. The mask is intended to provide protection of the wearer's respiratory system from airborne substances and mixtures which are hazardous to health. The mask is intended as a lifestyle product for use by the general public. It is not intended for professional, industrial or other occupations.

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