




## If gross debris is present on the surface:

1. Spray CaviCide1 directly onto surface to be cleaned.



2. Wipe with clean towel to pre-clean surface of all gross debris. Once surface is clean, dispose of towel in accordance with Federal, State, and local regulations for infectious materials disposal.

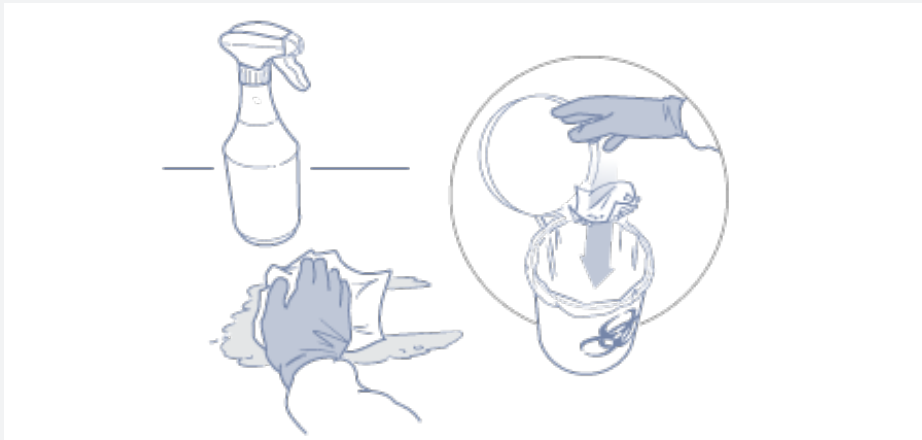

## Disinfect after Cleaning

1. Spray CaviCide1 directly onto surface, thoroughly wetting area. Leave surface wet for 1 minute.



2. Wipe with clean towel. Once complete, dispose of used towel.



## If NO gross debris is present on the surface:

1. Spray CaviCide1 directly onto surface, thoroughly wetting area. Leave surface wet for 1 minute.



2. Wipe with clean towel. Once complete, dispose of used towel in accordance with Federal, State, and local regulations for infectious materials disposal.



## Ordering Information

Part No.	Description
CAVISPRY01	CaviCide1 - 24 ounce spray
CAVIGALLO1	CaviCide1 - 1 gallon
CAVIGALL25	CaviCide1 - 2.5 gallon

