# **Invisalign Stickables**

English	2	Қазақша (Kazakh)	44
العربية (Arabic)	4	한국어 (Korean)	46
Bahasa Indonesian	6	Latviešu (Latvian)	48
Български (Bulgarian)	8	Lietuvių k. (Lithuanian)	50
Català (Catalan)	10	Malti (Maltese)	52
简体中文 (Simplified Chinese)	12	Norsk (Norwegian)	54
繁體中文 (Traditional Chinese)	14	Polski (Polish)	56
Hrvatski (Croatian)	16	Português (Brazilian Portuguese)	58
Čeština (Czech)	18	Português (Portuguese EU)	60
Dansk (Danish)	20	Română (Romanian)	62
Nederlands (Dutch)	22	Русский (Russian)	64
Eesti (Estonian)	24	Slovenský (Slovak)	66
Suomi (Finnish)	26	Slovenščina (Slovene)	68
Français (French)	28	Español (Spanish EU)	70
Deutsch (German)	30	Svenska (Swedish)	72
Ελληνικά (Greek)	32	ภาษาไทย (Thai)	74
תִּירָבֵּעַ (Hebrew)	34	Türkçe (Turkish)	76
Magyar (Hungarian)	36	Українська (Ukrainian)	78
Íslenska (Icelandic)	38	Tiếng Việt (Vietnamese)	80
Italiano (Italian)	40		
日本語 (Japanese)	42		

## Accessory description:

Invisalign Stickables are accessories that can be used to decorate the outside of Invisalign® clear aligners. This product is available exclusively from Align Technology, Inc. Patent is pending.

## Indication for use

Invisalign Stickables are intended to be used in accordance with the Instructions for Use as decorative accessories only with Invisalign clear aligners that are made from SmartTrack® material.

## Precautions:

- Read Instructions for Use before using Invisalign Stickables.
- Use under supervision of parent or guardian.
- Do not use with Vivera® retainers.
- The use of Invisalign Stickables has not been tested on Vivera retainers, aligner material other than SmartTrack aligner material, any third party clear aligners or retainers, any other orthodontic supplies such as brackets, retainers, expanders or others, or any body parts, e.g. teeth, tongue, nails or skin.
- It is not recommended to apply Invisalign Stickables in areas of crowded or heavily overlapped teeth, short clinical crowns, interior of the aligner, incisal/occlusal or the lingual part of the aligner, teeth with attachments, precision cuts, or precision wings, the eruption compensation and pontic area of the aligner and compliance indicators. Application of Invisalign Stickables in any of these areas may result in stickers having folds with sharp edges, stickers hiding attachments or wrapping around to the inside of the aligner and blocking the erupting tooth. It is recommended to place one sticker on the outer aligner surface covering one tooth length on the buccal/labial or the outer area of upper anterior teeth – centrals, laterals or canines. Ensure that sticker fits on the tooth area of the aligner and that it does not hang over the incisal/occlusal or gingival part of the aligner.
- Activities which could alter, damage, loosen, or dislodge the Invisalign Stickables should be avoided. Loosened Invisalign Stickables may be accidentally inhaled or swallowed by patients or could cause other damage to the patient. If stickers are swallowed, it is recommended to allow for these to pass naturally through the digestive system. If the patient is choking and coughing, stay with the person and encourage him/her to cough until the obstruction is cleared. If the patient cannot speak, or is not getting oxygen, seek emergency help immediately. Inform Invisalign doctor if repeated loosening of the Invisalign Stickables is noticed or for any other questions or concerns. If any unusual symptoms are exhibited, consult health care practitioner (or physician) immediately.
- If a reaction to the stickers is exhibited, discontinue use and contact Invisalign doctor.
- Do not ingest Invisalign Stickables.

## Instructions for Use

- Invisalign Stickables should be applied on the outside of clean and dry aligners. Note: It is recommended to wipe aligners with a tissue prior to applying stickers.
- Sticker adhesiveness may be also reduced when aligners are wet or slick due to the presence of oil or liquids from bare hands, saliva or other liquids.
- Clean and dry aligners thoroughly as per the Invisalign Instructions for Use before applying stickers.

- For doctors: Place one sticker on the aligner outer surface covering one tooth length on the buccal/labial area of upper anterior teeth – centrals, laterals or canines. Ensure that sticker fits on the tooth area of the aligner and that it does not hang over the incisal/occlusal or gingival part of the aligner. Note: stickers can be placed on aligner areas with Power Ridge™ features.
- It is recommended that users replace stickers every 3 days. If stickers are beginning to peel or lift off the aligner, please refer to the sticker removal instructions to replace and reapply a new sticker.
- Invisalign clear aligners that are decorated with stickers should be worn and cleaned as per Invisalign treatment Instructions for Use.

## Invisalign Stickables placement instructions (Figure 1):

1. Slowly lift a sticky rectangular applicator (which has sticker underneath it) with a thumb and an index finger on one hand while holding the sheet of stickers with the other hand. Gently take off the applicator with a sticker underneath it from the sheet.
2. Use applicator to line up sticker fully on the buccal/labial or the outer part of the aligner.
3. Make sure that the circle with the sticker design is right side up: the top of the design should be oriented towards the gingival part of the aligner.
4. Make sure that the circle with the sticker design is lined up at the center of the tooth and that there are no loose edges of the circle with the sticker design that are outside of the tooth area.
5. Use an index finger to press firmly and gently, and rub so the sticker fully contacts the aligner.
6. Slowly remove the applicator, rolling it backwards in order to remove the applicator without removing the sticker.
7. Finish pressing the sticker so that it fully contacts the aligner.


## Invisalign Stickables removal instructions

1. Gently use a scrubbing motion with a finger to remove the sticker from an aligner. Toothbrush and water may be used to assist with a removal and cleaning process.
2. Make sure that all adhesive from the sticker is off by wiping aligner surface clean with a tissue.
3. Discard used sticker, clean the area thoroughly with a tissue, and apply a brand new sticker if desired.

## Invisalign Stickables storage instructions

- Store at room temperature. Avoid direct sunlight.
- All information pertaining to product specification can be found in the Instructions for Use and primary packaging. Do not throw away or separate sticker from primary packaging and the Instructions for Use to maintain product information for traceability.

Figure 1: How to apply Invisalign Stickables

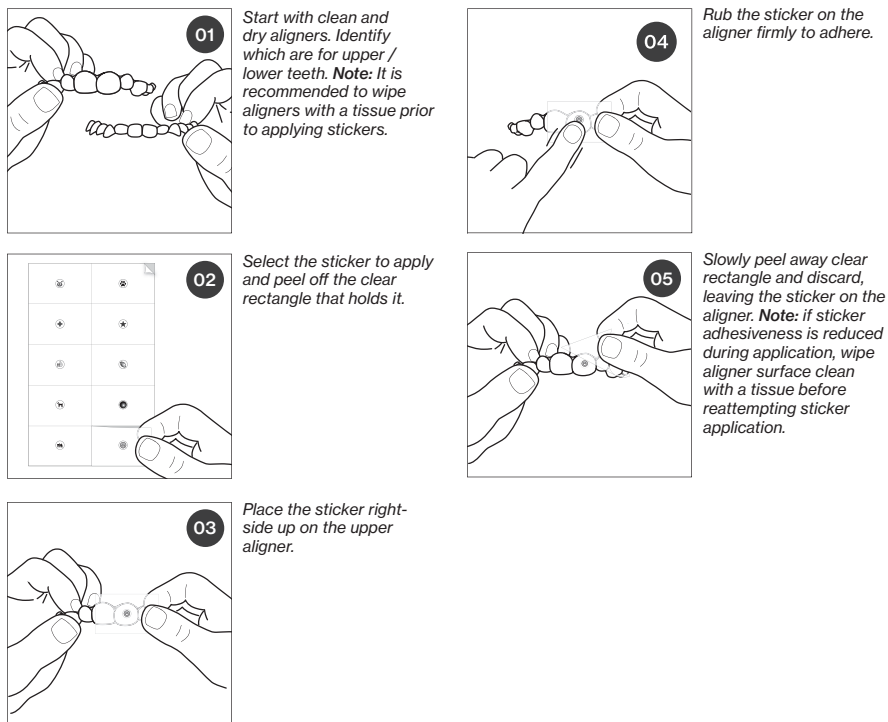## Invisalign Stickables symbols legend

Symbol	Title	Explanatory Text	Standard reference
	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified.	EN ISO 15223-1; 5.1.5
	Catalogue number	Indicates the manufacturer's catalogue number so that the product can be identified.	EN ISO 15223-1; 5.1.6
	Use-by-date	Indicates the date after which the product is not to be used where Use By Date YYYY-MM refers to the last day of the month.	EN ISO 15223-1; 5.1.4
	Manufacturer	Indicates the product manufacturer as defined in EU Directive 93/42/EEC.	EN ISO 15223-1; 5.1.1
	Manufacturer date	Indicates the date when the product was manufactured.	EN ISO 15223-1; 5.1.3
	Instructions for Use	Indicates a reference to enclosed or attached user instruction	EC Regulation 1223/2009

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© 2020-2021 Align Technology, Inc. Align, the Align logo, Invisalign, the Invisalign logo, SmartTrack, and Power Ridge, among others, are trademarks and/or service marks of Align Technology, Inc. or one of its subsidiaries or affiliated companies, and may be registered in the U.S. and/or other countries.
Align Technology, B.V.