

MIRACLE SUCTION INSTRUCTIONS FOR USE

1. Before Use

- Inspect for any damages on parts and components
- Remove protective film from the mirror head
- Sterilize all components before use

2. Instructions for Use

• Using the HVE adaptor

- Locate the gray end of the hose (NOT the blue connector end)
- Insert the HVE adaptor into the gray end of the hose
(See Picture A for reference)
- Insert adaptor into the HVE (High Volume Evacuator) handpiece/valve
- Screw the mirror head into the mirror handle if detached
- Insert the complete mirror handle into the hose with the blue connector
(See Picture B for reference)

• Using without the HVE adaptor

- Locate the gray end of the hose and insert directly into the SE (Standard Evacuator) valve
- Screw the mirror head into the mirror handle if detached
- Insert the complete mirror handle into the hose with the blue connector

• Attaching the Suction Holder

- Attach the Suction Holder wherever it is suitable for ease of use

3. Cleaning and Sterilization

- HVE adaptor, hose, mirror handle, and mirror heads are all sterilizable

• Cleaning of Mirror Head and Handle

- Detach the mirror head from the mirror handle
- Remove any visible soil by rinsing under water
- Use the cleaning brush provided to clean inside of the handle and suction holes and rinse under water
- Do not use any hard wire brushes to clean the mirror head as scratching may occur
- Prepare enzymatic instrument cleaning solution per the manufacturer's recommendation in a soaking container or ultrasonic cleaner
- Place the mirror head and mirror handle in the solution for the time specified by the manufacturer's cleaning instruction
- Rinse the mirror head and mirror handle under running water (drinking quality) to remove any cleaning solution residue
- Allow the mirror head and handle to dry before bagging

• Cleaning of Hose and Adaptor


- Detach the hose and the adaptor
- Remove any visible soil and thoroughly wipe down the entire outer surface of the hose and adaptor with a new, single-use wipe along with an alcohol-based, tuberculocidal, cleaning and disinfecting solution. Continue with additional wipes to ensure all visible soil is removed
- Clean the hose with evacuation cleaner following manufacturer's directions for use
- Allow the adaptor and hose to air dry before using

• Sterilization Method

- Detach all the components (adaptor, hose, mirror handle, and mirror head)
- Place them in a separate steam-sterilization pouch
- Place the bagged components into a steam sterilizer, as per the manufacturer's instructions
- The following minimum sterilization cycles maybe used:
 - Pre-vacuum: Minimum temperature of 121°C/250°F for at least 4 minutes.
Drying time: 30 minutes
 - Gravity cycle: Minimum temperature of 121°C/250°F for at least 30 minutes.
Drying time: 30 minutes

Connect this end to
the HVE handpiece

Picture A

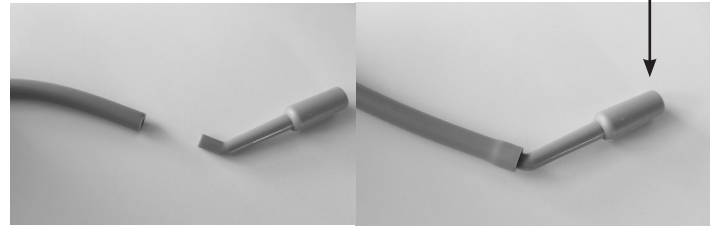

Picture B



DiaDent[®]

DiaDent Group International
www.diadent.com

#11 - 3871 North Fraser Way, Burnaby, BC Canada V5J 5G6
TF: 1-877-342-3368 | T: 604-451-8851 | F: 604-451-8865