



Comfort Arms

Stylish, Sturdy, Safe, Comfortable Armrests for the Toilet



SIMPLE, DO IT YOURSELF, INSTALLATION INSTRUCTIONS

Step 1

Remove seat



Remove the 2 bolts that attach the existing toilet seat to the toilet basin. Remove the toilet seat and set it aside. Clean the top of the toilet basin with an anti-bacterial cleaner.

Step 2

Position base plate



Place the Comfort Arms base plate over the toilet basin, aligning the slotted holes in the base plate with the holes in the toilet basin.

Step 3

Reinstall mounting bolts



Place the toilet seat on top of the Comfort Arms base plate. Reinstall the 2 bolts to secure the toilet seat and base plate. Do not overtighten the bolts.

Step 4

Apply silicon caulk



Place a bead of silicon caulk around the outer edge of the Comfort Arms base plate in the front and the back. This seals the base plate for easier cleaning.

Step 5

Attach arms at proper height



Slide the arms over the base plate upstands. Ask the person who will be using the Comfort Arms to sit on the toilet seat. Position one arm to just under his or her bent elbow for optimal comfort and leverage when transferring. Align the hole in the arm with the closest hole in the base plate upstand and insert the supplied bolt from the inside of the arm. Install washer and nut on the outside of the arm and tighten until snug. Do not overtighten the bolt. Attach the other arm in the same position as the first. Your installation is complete!

About Comfort Arms...

Designed with strength to last, and styled to look like a standard feature of a toilet, Comfort Arms come in 20-inch and 22-inch wide models. Both support 330 lbs. (150 kg) of weight. They fit on any standard toilet and are simple to install and uninstall.

Package contents:

- Zinc-etched, plastic-coated steel base plate.
- Nylon reinforced polymer arm (2).
- Stainless steel bolt (2), nut (2), washer (2).

Tools and supplies needed:

- Pliers or channel locks.
- Phillips-head screwdriver.
- Anti-bacterial bathroom cleaner.
- Bathroom Tub & Tile silicon caulk, white or clear.

Cleaning the Comfort Arms:

Use any good quality non-abrasive bathroom cleaner to keep Comfort Arms clean.

