Frequently Asked Questions 7-)

What is vFit PLUS?

The world's first and only home-use device for women to help improve intimate wellness and sexual function. It is non-hormonal and non-invasive.

How does it work?

It uses a patented combination of red LED lights, gentle heat, and sonic technology to warm the vaginal tissue.

Is vFit PLUS safe?

vFit PLUS has undergone testing to ensure safety when used according to its directions. It has been designated by the FDA as a low-risk general wellness product.

What does it feel like? Does it hurt?

vFit PLUS is comfortable and easy to use. While using vFit PLUS, you will feel a gentle heat which many women find very comfortable.

How much is it?

\$495 for the vFit PLUS device and \$25 for the photonic gel.

How do I use vFit PLUS?

vFit PLUS is used in the privacy of your home. To operate vFit PLUS, you turn on the power, select the session time and then activate the device. Insert the device into the vagina until the light window is completely covered. The device will automatically turn off once the session has ended.



How long are the sessions? How many sessions do I need?

vFit PLUS sessions are up to 12 minutes long. The device is intended to be used every other day, for 6 weeks. Many women start to notice improvement within 30 days. After that, we recommend women use it one to two times per week for maintenance.

How long do the results last?

Once your initial sessions are completed, use vFit PLUS once or twice per week to maintain your results.

Should I use anything with it?

The Photonic Gel is designed to be used with vFit PLUS. It helps with device insertion, performance, and comfort. It has been formulated with hyaluronic acid and aloe to moisturize.

Do I need a consultation with the doctor to buy it?

vFit PLUS is sold exclusively through professional providers. A consultation with the doctor is not required to buy this product. When buying from us, one of our staff will always discuss the product with you to provide guidance and answer any questions. May we get you started with vFit PLUS today?