



100WRP

PORTABLE WRIST STRAPS

IMPORTANT INFO

- Must be used under direct supervision
- Hinders unwanted arm movements.
- Use with EZ-ON Vest only
- Sewn on Wrist straps are available.
- Register this product to be notified for a safety recall.




WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanship under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

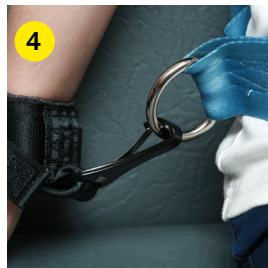

INSTALLATION INSTRUCTIONS

1. Wrap strap around each wrist one at a time.
2. Clip buckle closed one at a time.
3. Adjust the size by pulling on excess webbing.
4. Clip the hooks on the wrist strap to the hip level D-ring on EZ-ON Vest.