



# 503

## LAY DOWN VEST

### IMPORTANT INFO

- Use for body casts, hip spica casts, braces and any other configuration where the passenger must be transported laying down.
- For use by passengers over 12 months old, weighing 22 lbs. – 106lbs.
- A layer of padding no thicker than 2” may be used under the passenger for added comfort.
- Can be secured by using lap type seat belts or lap/shoulder type seat belts. If using lap/shoulder belt, shoulder belt must be in the locked position.
- EZ-ON Products must be worn under winter jackets.
- REGISTER this transportation vest to be notified of a safety recall.
- Sizing: (According to chest measurements) XS 18” – 24” | S 23” – 29” | M 28” -34”

### WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

### WARNING

Death or serious injury can occur if manufacturer’s instructions are not followed correctly. All EZ-ON products are dynamically tested to meet NHTSA standards. Please follow washing instructions on product tag. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement after six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

### PINCH TEST PROCEDURE

#### THE PINCH TEST

Tighten vest so that no pinch in the webbing can be achieved on the chest strap.  
When vest is snug fingers will not be able to grab any webbing.





# 503 LAY DOWN VEST

## INSTALLATION INSTRUCTIONS

### TO SECURE VEST ONTO PASSENGER



1. Place vest on passenger with buckles positioned to the front and the bottom strap over hip bones.
2. Close and secure push buckles. Tighten the buckles by pulling on the loose webbing.
3. Bring crotch straps through passenger's legs and secure.
4. Tighten crotch straps to be snug, but not too tight. Make sure to position the hip strap low and over hip bones.

5. Thread loose end of shoulder strap through black plastic bar slide.

*\*Do not pull too tight, as this will cause vest to pull off hips.*

6. Back thread to lock in place.

*\* Repeat steps 5 & 6 for second shoulder strap.*

### PINCH TEST

*Please refer to procedure on reverse side.*

### TO SECURE PASSENGER TO REAR SEAT

**NOTE:** Rear vehicle seat must be flat. Passenger must be able to fit on rear seat laying lengthwise.



7. Lay passenger on the rear seat with the casted leg against the seat back.

- 8A. For passengers over 30" tall, route the vehicle seatbelt closest to the passenger's head through the chest strap, buckle and tighten. *\*When using a lap/shoulder belt, the shoulder portion of the belt must be in the locked position.*

- 8B. For passengers under 30" tall, route seat belt closest to passenger's head through both the chest and hip strap of vest, buckle and tighten. Skip step 9. *\*When using a lap/shoulder belt, the shoulder portion of the belt must be in the locked position.*

9. For passengers over 30" tall, route the middle seat belt through the hip strap of vest, buckle and tighten.

*\*When using a lap/shoulder belt, the shoulder portion of the belt must be in the locked position.*

10. Secure seat belts as tightly as possible. Make sure all the seat belts are in the locked position.

11. Buckle remaining seat belt closest to passenger's knee preparing for next step.

12. Place extremity belt around both legs and lap portion of the already buckled seatbelt.

13. Buckle and tighten the extremity belt at or above the passenger's knee.