



**EZ-ON
PRODUCTS**

MAXPV MOUNT

INSTALLATION INSTRUCTIONS

IMPORTANT INFO

- Meets FMVSS213 when used with the MAXPV Vest
- Must be used with the MAXPV Vest
- Product must be used on a seat with the factory tether anchor point on the back side of the seat.
- Cannot be used in a sedan or pickup truck.
- For use by passengers weighing 31 - 168lbs
- Install tether hardware kit if passenger weight exceeds the vehicles tether anchor
- Weight determined by the vehicle manufacturer
- Must use factory seat belt



WARNING

Death or serious injury can occur if manufacturer's instructions are not followed correctly. Read below warning information for replacement recommendations.



REGISTER

Register this transportation vest to be notified of a safety recall.



WARRANTY

EZ-ON Products stands behind the products we make. Warranty covers any defects in material or workmanships under normal use during the warranty period. We will repair or replace product defects for 1 year from manufacturer date.

WARNING: All EZ-ON products are dynamically tested to meet NHTSA standards. Replace product if it is damaged, frayed, or has been in a crash. Recommended replacement period is six years. EZ-ON Products are not liable for injury if these recommendations are not followed.

EMERGENCY RELEASE

CUT ALONG DOTTED LINE FOR
EMERGENCY RELEASE



100 SCBC
SAFE CUT BELT CUTTER



605 Commerce Way West
PO Box 660
Jupiter, FL 33458

561.747.6920
cs@ezonpro.com

EZONPRO.COM



EZ-ON PRODUCTS

MAXPV MOUNT

MOUNT FOR PERSONAL VEHICLES

INSTALLATION INSTRUCTIONS

SCAN CODE FOR VIDEO INSTRUCTIONS & MORE



1. Fold seat and locate tether anchor. May have to remove fabric cover to gain access to bite of seat



2. Lay mount over back of seat



3. Connect central tether hook to lower vehicle anchor



4. Disconnect shoulder straps from D-ring and hang straps over the seat



5. Disconnect bottom tether hooks from D-ring



6. Feed bottom tether hooks through bite of seat



7. Fold seat up and pull bottom tether hooks fully through bite of seat



8. Properly seat the passenger



9. Loop bottom tether hooks through hip D-rings on vest



10. Feed bottom tether hook back through bite of seat. Captain's chairs, can go around side of the seat. Repeat step 9-10 for opposite side




11. Starting from the top down, loop shoulder tether hooks through shoulder D-rings and lay strap over the back of the seat. Repeat this step for the opposite side



12. Hook bottom tether hook to lower D-ring on mount



13. Tighten bottom tether hooks by pulling on loose webbing through tilt adjuster. Ensure it is fully tightened before proceeding to next step

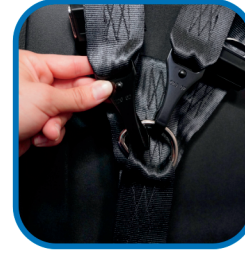

14. Hook shoulder tether hooks to the top D-ring on the mount



15. Tighten shoulder straps by pulling the loose webbing through the tilt adjuster



16. Thread vehicle seat belt through vest loops



17. Buckle seatbelt low on the lap. Make sure seatbelt is in the locked position and secured as tightly as possible

