

COVID19 SANITATION FOR MEGAPEEL

MegaPeel EX Bronze, Silver, Gold and Platinum are equipped with many safety and sanitation specifications already in place that will support your skincare practice as you return to work. These are detailed in each MegaPeel User Manual for your reference, and are highlighted here. Please note that any regulations from your state board and executive orders must be followed in addition to these sanitation measures.



1. Disposable clear divided tip and gold tip are used for each crystal microdermabrasion treatment and then discarded.



2. The Enviro Sealed™ HEPA cartridge seals in bio-contaminated waste – keeping it safely out of your treatments – using a patented multilayer HEPA cartridge.



3. The metal hand pieces and diamond tips may be cleaned and sanitized.



REVIEW OF CLEANING INSTRUCTIONS FOUND IN THE USER MANUAL

For cleaning the Silver and Gold handpieces used for crystal microdermabrasion:

- Once a week ream out the hand-piece using the needle in the spare-parts kit.
- After treating each patient/client, sanitize the hand-piece by using alcohol, CaviCide or an autoclave.

For cleaning the re-usable Diamond tips for crystal-free microdermabrasion:

- Cleaning must be performed immediately after every treatment. Immediately after each use, remove the Diamond tip from the Diamond hand-piece and rinse under hot running water. Scrub the abrasive surface of the Diamond tip with a brush (located in the cleaning kit which comes with the Diamond Tips) to remove the protein material. The Diamond tip should then be placed in the CaviCide, using either a tray or an ultrasonic unit, making sure it is completely covered and let it soak in CaviCide for 3 minutes at room temperature (68°F/20°C).

Or steam sterilization for Diamond tips

- Standard steam sterilization is based on your autoclave's maintenance parameters. Every sterilizer is different, please consult with your sterilizer's operating manual for recommended sterilization times and temperatures.
- Steam sterilization should always be used if the surface of the skin may be penetrated.

For cleaning the Diamond handpiece for crystal-free microdermabrasion:

- After treating each patient/client, sanitize the hand-piece by using alcohol, CaviCide or an autoclave.

Or steam sterilization for Diamond handpiece

- Standard steam sterilization is based on your autoclave's maintenance parameters. Every sterilizer is different, please consult with your sterilizer's operating manual for recommended sterilization times and temperatures.
- Steam sterilization should always be used if the surface of the skin may be penetrated.

The outer surfaces of the MegaPeel system may be wiped with alcohol or CaviCide wipes, including control knobs, power switch and handpiece-holder reservoir. For further protection, finger cots may be applied to cover the individual control knobs and disposed of after each treatment.

Be reassured that the aluminum oxide crystals are always new (they don't re-circulate). The used crystal and debris are always self-contained inside the waste container. Please dispose of the full waste canister properly to avoid exposure to any contaminants.