

Silicone Treatment Mask

Lightening Protocol

Pigment lightening and deep nutrient penetration is easily achieved with this protocol. The silicone mask forces toxins in the dermis to rise to the epidermis in the form of perspiration. As the toxins burn off, the remaining water grabs the skincare nutrients and drives them into the dermal layer, creating super hydration to all layers of the skin, treating where damage begins – and where new collagen production begins. Microdermabrasion as an optional step is recommended for increased results.

Enables Deep Penetration of Product Nutrients

Contains Botanical Pigment Lightening Agents

An Amazing Booster Treatment to Microderm

Items you will need:

botana-gel cleanser™

MegaPeel EX Microdermabrasion (OPTIONAL)

clari-tone serum™

C-difference™ - vitamin C lotion

silicone treatment mask

daily eclipse™ - oil free sunscreen

PROCEDURE:

1. Cleansing

Apply botana-gel cleanser™. Massage gently over face to cleanse and remove makeup. Rinse with water and pat dry.

2. MegaPeel EX Microdermabrasion Procedure (OPTIONAL)

Make sure skin is completely dry. When conducting a MegaPeel microdermabrasion treatment it is advised to make two (2) passes over the skin. The first pass should treat the entire face vertically and the second pass should treat the face horizontally. It is very important to pull the skin taut in the area being treated. The skin should be pulled in the same direction the hand piece is moving. Slowly glide the hand piece over the skin back and forth over the area. After treatment is complete, dust skin free of any particles.

3. clari-tone serum™

Immediately apply a liberal amount of the clari-tone serum™ all over the client's face.

4. C-difference™ - vitamin C lotion

Follow by a layer of C-difference™ - vitamin C lotion on top of the clari-tone serum™.

5. silicone treatment mask

After application of topicals, place the silicone treatment mask on client's face and mold the silicone over entire face.

6. Cover

Cover the silicone treatment mask with a damp hot towel, making sure that client is able to breathe comfortably. Leave silicone treatment mask on for 7-10 minutes.

7. Remove

Gently remove remaining product and set the mask aside for cleaning. OPTIONAL: Manual extractions may now be performed as needed. Apply appropriate moisturizer and eye cream. Note: Cleaning instructions on back.

8. Finish

Apply daily eclipse™ - oil free sunscreen with SPF 30 in a thin, even layer to face.

9. Home Care

Recommend adding clari-tone serum™ to nightly skin care regimen.



SAVE THESE INSTRUCTIONS

The Silicone Treatment Mask will provide approximately 40-50 applications if cared for properly. These instructions are the recommended and validated procedures for the cleaning, disinfection, or sterilization of masks for multiple patient use/reuse.

SAFETY SUMMARY

To ensure the safe disinfection of the silicone treatment mask, these instructions MUST be followed.

CLEANING THE MASK PRIOR TO DISINFECTION

1. Immediately after each use of the silicone mask, rinse thoroughly under water with mild soap to remove any excess product and organic debris.

After cleaning the mask, disinfect the mask using the disinfection method listed below.

DISINFECTING THE MASK

1. After cleaning the mask, using either a tray or an ultrasonic unit, visibly wet and soak the mask in CaviCide1 for 1 minute at room temperature (68°F/20°C).
2. Remove and rinse the mask well by running it under clean water.
3. Wipe dry thoroughly using paper towels and transfer the silicone treatment mask in between 2 paper towels. Store covered.

NOTE: Some discoloration of the silicone may occur and is acceptable. If any damage or deterioration is found, the mask should be discarded and replaced with a new mask.

Why DermaMed Solutions

A Holistic Approach to Professional Skincare & Aesthetic Equipment

Since 1998, DermaMed Solutions has provided professional skin care equipment, spa equipment and natural skincare products. In that time, we have built a reputation for innovative technology in skincare equipment and salon spa equipment along with high quality customer service.

We are committed to helping your skincare business grow. At DermaMed Solutions, we believe the condition of your skin is a reflection of your overall well-being. Achieving and maintaining healthy, beautiful skin requires a holistic approach that incorporates mechanical and topical treatments, but also considers nutrition, mindset and other lifestyle factors. Skincare regimens and treatments must be tailored to meet the unique qualities of each person. Our approach uses an integrated methodology which combines treatment modalities so you can help your clients reveal brilliant skin from the "inside out" and "outside in".

Our approach:

Mechanical Procedures: Which includes MegaPeel® Microdermabrasion, geneO+ 3-IN-1 Super Facial, Quadra Q4® Intense Pulsed Light, ClearQuest Laser Diode System, and ExcellaWave US/E-stim System.

Topical Skin Care: dmSkincare, a "clean clinical" professional retail and back bar skincare line that is paraben-phthalate-, sulfate-, petroleum-free with no artificial fragrance or colors.

Wellness: Nutritional supplements and lifestyle protocols that focus on beauty from the inside out.

Learn More

We want to be a partner in your practice building and financial success. Contact us today to begin developing a plan that is tailored to your needs, please call [610-358-4447](tel:610-358-4447) or visit us at www.DermaMedSolutions.com.

