Confidence® Plus 2 Germicidal Cleaner



Confidence Plus 2 is a multipurpose, one-step germicidal detergent that is the recommended cleaner sanitizer for MSA PPE and respiratory equipment. Confidence Plus 2 is a quaternary ammonium based, EPA-registered (#47371-130-4204) disinfectant and is effective against multiple pathogens including SARS-associated Coronavirus.

- Used for cleaning and disinfection of respirators and other PPE such as headgear, faceshields and hearing protection. May be wiped on hard surfaces of respirator cartridges and filters.
- Confidence Plus 2 may also be used as a disinfectant in schools, hospitals, nursing homes or other institutional settings.
- One bottle makes 32 gallons of disinfecting solution.
- The use-solution may be applied with a mop, sponge or spray.
- May not be used for sterilization of medical devices.
- The packaged product does not have an expiration; diluted product in solution is shelf-stable for up to 1 year.
- See respirator instruction manual for cleaning and disinfection instructions of respirators.

PART NO.	Description
10009971	Confidence Plus 2
10032737	Confidence Plus 2 (Canada)

EPA Registration #47371-130-4204





Effective against the following pathogens:

Bacteria

Pseudomonas aeruginosa1

Salmonella enterica

Staphylococcus aureus1

Enterobacter aerogenes

Enterobacter cloacae - NDM-1

Enterococcus faecalis - Vancomycin

Resistant [VRE]

Escherichia coli1

Escherichia coli - NDM-1

Klebsiella pneumoniae1

Klebsiella pneumoniae - NDM-1

Legionella pneumophila

Listeria monocytogenes

Proteus vulgaris

Salmonella typhimurium

Serratia marcescens

Staphylococcus aureus - Methicillin-resistant [MRSA]

Staphylococcus aureus - Vancomycin Intermediate

Resistant-[VISA]

Streptococcus pyogenes [Strep]

1 ATCC & antibiotic-resistant strain

Viruses

Adenovirus Type 4

Hepatitis B Virus [HBV]

Hepatitis C Virus [HCV]

Herpes Simplex Virus Type 1

Herpes Simplex Virus Type 2

HIV-1 [AIDS virus]

Influenza A Virus / Hong Kong

"in association with the organism Influenza A virus/Hong Kong"

Respiratory Syncytial Virus [RSV]

Rotavirus [WA]

Rubella virus

SARS Associated Coronavirus [SARS] [cause of Severe Acute

Respiratory Syndrome]

Vaccinia [Pox Virus]

Note: This Bulletin contains only a general description of the products shown. While product uses and performance capabilities are generally described, the products shall not, under any circumstances, be used by untrained or unqualified individuals. The products shall not be used until the product instructions/user manual, which contains detailed information concerning the proper use and care of the products, including any warnings or cautions, have been thoroughly read and understood. Specifications are subject to change without prior notice. MSA is a registered trademark of MSA Technology, LLC in the US, Europe, and other Countries. For all other trademarks visit https://us.msasafety.com/Trademarks.

MSA operates in over 40 countries worldwide. To find an MSA office near you, please visit **MSAsafety.com/offices**.