

Techline Technologies, Inc. Device Specifications WW3-003 Impaled Object



# WW3-003

# **Impaled Object**

Patent Nos. US8,221,129, US8,491,309

**TECHLINE®** Device Specifications November 4, 2021

**NSN** 6910-01-672-1813 **TECHLINETRAUMA.COM** 

## **Description-**

A wearable wound simulation that represents a piece of rebar impaled into the flesh. It wraps around an actor or manikin and attaches with hook and loop fastening. The PATENTED construction with SOFTTECH® and ROCKTECH® materials, provides realism for your training scenario.

The WW3-003 is constructed in two pieces. The main simulation represents the flesh wound, and a removable piece of replica re-bar that inserts through the wound site.

This wearable wound comes without a bleeding system.

### Construction-


The representative skin and tissue of the WW3-003 is constructed of high quality silicone rubber, with layers of coloration that provide a depth and realism to the product. In addition, the SOFTTECH® and ROCKTECH® materials have a realistic feel and are very durable and UV resistant. The skin also contains a reinforcing fabric that provides strength, and resistance to tear propagation should the skin be punctured or cut.

#### Operation-

The hook and loop fastener is snugged to fit the actor or manikin, and the flaps are fastened together in an overlapping fashion. The replica re-bar inserts through the wound site.

#### Maintenance-

The simulation should be rinsed in clean, warm water and allowed to dry before storage. The device may be washed with mild detergents like dish soap or cleaned with a disinfecting agent like isopropyl alcohol or a diluted bleach solution, as it is durable, and chemically resistant. Allow the device to dry fully before storage to prevent mold or mildew growth.

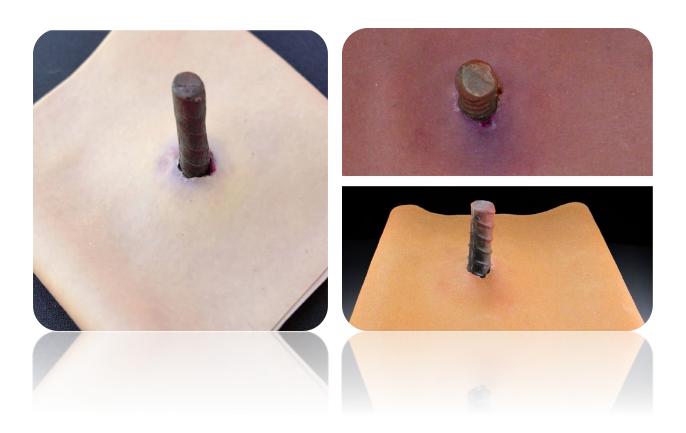

WW3-003 Impaled Object Light, Medium, and Dark Skin Tones Available