



Techline Technologies, Inc.
Device Specifications WW3-012
Wound Extender



WW3-012

Wound Extender

Patent Nos. US8,221,129, US8,491,309

TECHLINE® Device Specifications
November 4, 2021

TECHLINETRAUMA.COM

Description-

A wearable wound that extends an additional wound, to fit around different sized actors and role players. The Wound Extender permits wounds with hook and loop closures to be used on a variety of actors and allows wounds designed for use on arms and legs, to be used multiple locations on the actor. It wraps around an actor or manikin and attaches with hook and loop fastening. The PATENTED construction with SOFTTECH® materials provides realism for your training scenario.

The WW3-012 can be used for any WW3 wearable wound product (sold separately).

Construction-

The representative skin and tissue of the WW3-012 is constructed of high quality silicone rubber, with layers of coloration that provide a depth and realism to the product. In addition, the SOFTTECH® material has a realistic feel, and is very durable and UV resistant. The skin also contains a reinforcing fabric that provides strength, and resistance to tear propagation should the skin be punctured or cut.

Operation-

The hook and loop fastener is snugged to fit the actor or manikin, and the flaps are fastened together in an overlapping fashion. Use in conjunction with hook and loop closures wounds to increase the fit.

Maintenance-

The product should be rinsed in clean, warm water and allowed to dry before storage. The product may be washed with mild detergents like dish soap or cleaned with a disinfecting agent like isopropyl alcohol or a diluted bleach solution, as it is durable, and chemically resistant. Allow the product to dry fully before storage to prevent mold or mildew growth.



WW3-012 Wound Extender
Light, Medium, and Dark Skin Tones Available