Notes

Précautions

Notas

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ANSED Diagnostic Solutions LLC

1528 Walnut Street
Philadelphia, PA 19102

www.ANSEDdiagnostics.com
info@ANSEDdiagnostics.com

HD Digital Video Scope Kits User Manual

- English
- Français
- Español

Warning: before using, please read the manual carefully and follow the instructions to avoid damages and injuries.

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Function Buttons

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- Long-press turn on/off the product.
- According to the using environment, press / to adjust the brightness of LED lights for the best imaging effect, three levels adjustable.
- Up/down / : move up and down; or zoom the image for a clearer view, the amplification factor is 1.4X/1.6X/2.0X.
- Rotate : maximum 180° rotation for the best angle to view the image.
- Photograph and record preview situation, short-press to photograph, long-press to record; when recording, press to stop; when viewing file, press to play/stop the video. Support max 32GB TF card.
- OK button under preview situation, press OK to enter system settings; when viewing file, press OK to the file's detailed information.
- ESC Under preview situation, press ESC to check the file; otherwise, press ESC to exit.

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Interface Description

Description du menu

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In preview mode, press OK to enter the interface for system setting; in system setting interface, press OK to confirm, press up / down to select, press ESC to exit.

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