# harex

INSTRUCTIONS FOR USE / INSTRUCCIONES DE USO / GEBRAUCHSANWEISUNG / MODE D'EMPLOI / ISTRUZIONI PER L'USO / GEBRUIKSAANWIJZING / INSTRUÇÕES DE UTILIZAÇÃO / INSTRUKCJA UŻYTKOWANIA / HASZNÁLATI UTASÍTÁS / NÁVOD K POUŽITÍ / INSTRUCCIUNI DE UTILIZARE

Versión 04\_2023-03

**EN**
**ATTENTION**  
*Before using the Device, read the instructions carefully.*

**INTENDED USE**  
Harex® is a medical device for male urinary incontinence control. It is an external use mechanical device, made of medical grade material and latex-free.

**DESCRIPTION**  
Harex® was developed with the goal of improving the quality of life of men suffering from urinary incontinence. The design takes into account the penis anatomy and it prevents and controls involuntary urine leakage by placing it behind the glans of the penis and by applying a gentle pressure on the urethra.

The correct placing of Harex® is with the urethral pressure pad (D) on the lower part of the penis shaft, as this is where the urethra is located. The flexible design of Harex®'s fitting strap that goes around the penis and the lack of pressure at both sides of the penis, enables the blood flow.

Harex® is made of medical grade plastic materials tested not to absorb liquids.

There are no metallic components and metal detection systems (i.e. those used at airports and/or train stations) can be passed through while wearing Harex®.

The set includes a washable plastic case to transport and store the device.

**SIZE**  
- Outer dimensions: 48 mm x 52 mm (4,8 cm x 5,2 cm)  
- Fitting strap width: 18 mm (1,8 cm)  
- Distance between "D" (urethral pressure pad) in its upper position & the fitting strap (C): 8 mm (0,8 cm)  
- Weight: 18 g

### INSTRUCTIONS FOR USE

**WHERE TO PLACE HAREX®:**  
- Place it along the shaft of the penis. Recommendation: place it just behind the head of the penis (glans).  
- Important: The correct placement of the urethral pressure pad (D) is underside of the penis, where the urethra is located.

#### A. HOW TO PUT ON THE DEVICE

- Move down the adjustment lever (B) and open the fitting strap (C).
- Put the penis into the device.
- Adjust the fitting strap (C) to the diameter of the penis.
- Move the adjustment lever (B) upward to lock the fitting strap (C).
- To block the urethra, move the urethral pressure activator (A) to its upper position.

#### B. TO URINATE

- Unblock the urethra by moving the urethral pressure activator (A) to its lower position.
- Urinate.
- Upon completion, move the urethral pressure activator (A) to its upper position to block the urethra.

#### C. HOW TO REMOVE THE DEVICE

- Move the urethral pressure activator (A) to its lower position.
- Move down the adjustment lever (B) and open the fitting strap (C).
- Loosen completely the fitting strap (C) to remove the device.

**Optional:** The fitting strap (C) has a numerical ruler to mark its optimal position and thus facilitate its adjustment.

#### WARNINGS AND PRECAUTIONS

- Urinate every 2-3 hours and/or based on your physiological needs.
- Recommendation: allow the penis to rest during sleeping hours by removing Harex®. In case of overnight use, loosen the fitting strap to a lighter pressure.
- Incorrect and/or excessive use of Harex® may cause skin irritation and/or discomfort and/or slight pain.
- In case of discomfort and/or slight pain, please check that 1) the device is correctly positioned (Fig.1) and 2) there are no creases in the skin of the penis. In case of continued discomfort, try a looser fit.
- Do not use Harex® if you have open wounds and/or cutaneous irritation on the skin of the penis.
- In case of repeated urethral strictures, consult your doctor before using Harex®.
- For effective use, the length of the penis shaft should be at least 2 cm.

#### CLEANING

Wash the device by hand with soap & warm water.

#### TERMS AND CONDITIONS

Be aware that you are using Harex® at your own risk. If you have a medical issue other than urinary incontinence, please consult your doctor before using Harex®. When experiencing issues with the use of Harex®, please stop using it immediately and inform your doctor.

**Recommended to replace Harex® after 12 months of use.**

**For more information visit our website at: [www.harex-net](http://www.harex-net).**

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