

HOW TO ASSEMBLE YOUR UNCLOG EYE MASK



FRONT SIDE

The back side rest gently on the eyes.

OPENING


1

Fold the opening flap on the back side outward



2


Insert the end of the heating unit end with the cord going in through the opening.
Gently pull the cord through the hole, making sure the heating unit is flat.

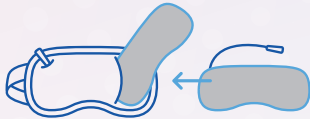

3

Insert the crystal insert above the heating unit and away from the eyes.

Be sure the heating unit and crystals are aligned and not pushed to one side or wrinkled.

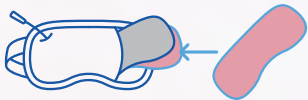

4

Unfold the end back to the normal position.
Gently shake to spread the crystal beads evenly.



WARRANTY INFORMATION

UNCLOG HEATING EYE MASK IS BACKED BY A 30-DAY MONEY-BACK GUARANTEE AND A ONE-YEAR WARRANTY



TO BE ELIGIBLE FOR THE ONE-YEAR WARRANTY:

Register your new unclog heating eye mask (warranty starts from the date of purchase) proof of purchase is required when claiming the warranty.

- 1 Register Your Product Easily Online!
- 2 Scan the Barcode below or Visit our official website at www.unclogmask.com.
- 3 Navigate to the "Product Registration" tab.
- 4 Fill in the details and click "submit".



351993 NE Elam Young
Park A. Hillsboro,
Oregon 97124



888 781-2811



service@unclogmask.com



www.unclogmask.com

UN^{clog}

UNCLOG HEATED EYE MASK

USER MANUAL

Welcome to the User Manual for our Heated Eye Mask. This manual provides essential information on how to safely and effectively use your heated eye mask for relaxation, stress relief, or eye care purposes.

Product Model: B08VCF6JH




Distributed By: Eye Comfort Care LLC
Proudly designed in the USA | Made in TAIWAN

Proudly designed in the USA Made in TAIWAN

TO DISASSEMBLE YOUR UNCLOG EYE MASK, SIMPLY REVERSE THE ASSEMBLY


RELAX AND ENJOY THE SOOTHING WARMTH THERAPY

1 **Connect the USB power source to power bank, laptop, PC, phone charger, or 5V/2A adapter.**

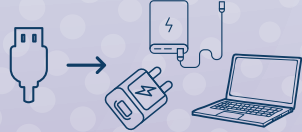

2 **Press the bottom button to power the unit ON**

When you press the 'Power' button on the controller, the screen activates, and the default settings are displayed. At this point, you'll see the timer set at 20 minutes and the lowest temperature setting # 1.




3 **Temperature button**

Continue pressing to increase or decrease the temperature as desired.



05- 122F / 50C
04- 115F / 46C
03- 109F / 43C
02- 104F / 40C
01- 97F / 36C

4 **Select the Time Selections option to choose your desired session length for therapy and relaxation. Options range from 20 to 60 minutes.**



5 **Place the mask over your eyes.**

Gently manipulate the crystal beads to position them to the desired locations like the inner and outer corners of your eyes with your fingers.

The self-fastener can be adjusted for maximum comfort.




GETTING STARTED

SAFETY PRECAUTIONS

- **Read the entire manual** before using the heated eye mask.
- **Do not use** if the eye mask appears damaged.
- **Do not immerse** the mask in water or any other liquid.
- **Do not use** the eye mask on infants, children, or individuals with limited mobility unless supervised.
- **Avoid prolonged use** to prevent skin irritation.


PACKAGE CONTENTS:

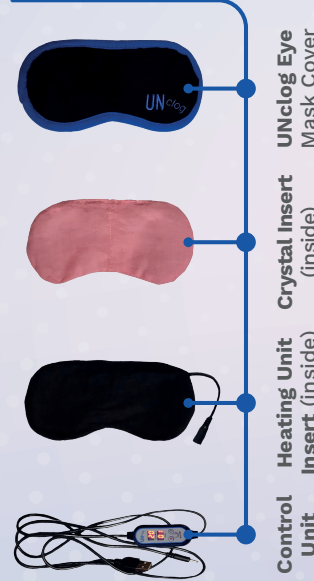

EXAMPLES OF UNCLOG USE

#3 For Daily Eye Therapy and Relaxation

#3 (or any preferred warmth) UNclog Mask controls the temperature to stay within a safe range unlike microwaveable masks that may become too hot.



#5 Moderate Dry Eye Patients

#5 50 minutes twice a day for five days followed with daily routine or as directed by your doctor.



#1 For Dermatitis/ MGD Combo Patients

#1 Use this routine therapy once to twice daily, 20 minutes duration or as directed by your doctor.




Our UNclog mask is designed to provide warmth and comfort to the eyes, helping to alleviate any blockages in the glands that produce the eye's protective barrier. Regularly using UNclog mask may improve natural tear production and overall eye comfort, reducing the need for artificial tears.

Clean your lids and lashes before using your mask. (Look for eyelid cleanser online or at your pharmacy) Removal of the excessive oil produced by your eyelashes reduces the possibility of bacterial growth.

Blink hard 5-10 times after about 20 minutes of heating. Blinking is the natural way to pump the bad oil out. Blinking stimulates and strengthens the glands.

There is a break-in period for your new Unclog Mask. At first, you might not notice any heat, then after using it a few times, you might start to feel gentle heat. That is normal. **It feels good! Enjoy!**



HOW TO CLEAN



Spot Cleaning: If you notice any stains or spills on your eye mask, gently wipe the affected area with a damp cloth. Make sure the cloth is clean and not overly wet.



Hand Wash (cover only) (see how to disassemble page)



Or in the washing machine (cover only) on a regular cycle (see how to disassemble page)

TIP: USE THE SAME SOAP YOU USE TO WASH YOUR FACE TOWELS

Your safety is our priority. Follow these guidelines for a safe and enjoyable experience. | Do not use if wiring is exposed, contact customer service at for assistance. | Avoid tugging or pulling the cord | Don't wrap the cord tightly around the mask | Do not sleep with the mask on.