
DODO AIR Face Mask

ppe GERMANY

20 Pieces

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DODO AIR face masks are produced in Berlin (Germany) by PPE Germany GmbH. They are intended for single use up to 8 hours. Please wash your hands thoroughly before putting on the mask. Please replace the mask immediately after use with a new, dry, clean and well-fitting mask. The mask is an FFP2 mask approved in the European Union and complies with the European DIN EN 149:2001+A1:2009 and the European PPE Regulation EU 2016/425. Shelf life of the product: 2 years. More information: <https://ppegermany.de/dodo-usa>

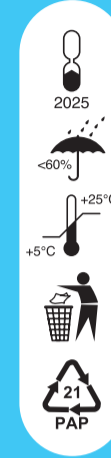<https://ppegermany.de/dodo-usa>

Improved Breathability

Adjustable shape

Less fogged glasses

FFP2

DODO AIR

Ready to go

Each mask in this box is individually sealed, making it perfect to take one at a time. Ideal for dispensers, offices, schools and reception areas.

Product name: DODO AIR face mask
EAN: 4260763950394
Product quantity: 20 pieces per box
Uses: Dust proof, professional and protective.
Storage conditions: Temperature from 5 °C to +25 °C and at a maximum humidity of 60%

Execution standard: FFP2
Manufacturer: PPE Germany GmbH, Germany
 Mertesstraße 63-115 · D-13587 Berlin
Made in Germany – synonymous of product quality, durability and reliability

20 Pieces

DODO AIR Face Mask

DODO AIR Face Mask

DODO AIR

Face Mask

Breathable · Comfortable · Disposable
German engineering and quality assurance tested

20 Pieces

white

FFP2

PRODUCED FOR USA

20 Pieces


DODO AIR Face Mask


DODO AIR Face Mask

ppe GERMANY


20 Pieces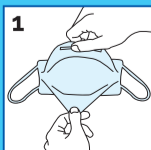


1 Place the mask in your hands with the top side facing upwards.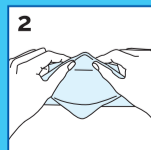

2 Put on the protective mask and press the metal part on the bridge to the shape of your nose.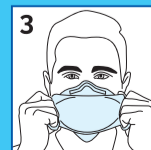

3 Pull the bottom of the mask slightly down so that the nose, mouth and chin are well covered.

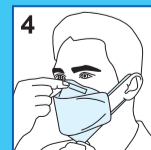4 Check the fit of the mask so that no secondary air is drawn in. Check facial hair for air leaks. If necessary, correct the fit until the mask fits comfortably and seals correctly.

FFP2

For product quality, durability and reliability