

LOVEWAX WAX WARMER AX-200

- USER MANUAL




The heater is ideal for all types of depilatory waxes.
Dissolves any type of wax, including canned waxes, wax drops or black wax.

MODEL: AX-200

Certificate No. : POCE19032609NCS, POCE19032614SCE.

Technical parameters:

- Power: 100W
- Power supply: 100W
input voltage: 220/230V
output voltage: 220/230V
- Capacity: 500ml
- Temperature range: 1-100°C
- Aluminium inner container

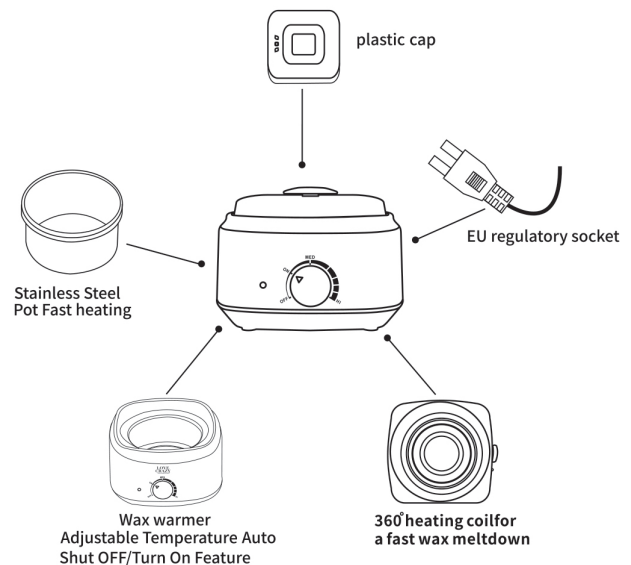

Preparing the device for operation:

1. Connect the appliance to the power supply.
2. Remove the plastic lid from the heater.
3. Place the flange ring on the rim of the heater.
4. Adjust the handle on the wax can for easier handling.
5. Place canned wax or wax drops in the heater.
6. Switch on the heater, set the temperature until the wax melts. This process for 100g of wax takes about 10 minutes, to speed it up, put a lid on the heater. The higher the temperature of the wax, the faster it will melt.
7. Once the wax has melted, reduce the temperature to a lower temperature, such that it does not burn the skin. It is important to check the temperature of the wax before applying it, adjusting the temperature if necessary. The wax must not be too hot or too cold.
8. Proceed to treatment.
9. Once the treatment is complete, remove the wax can from the unit, switch off the unit and then cover it with the lid.

Precautions:

- Ensure that the user does not have any skin diseases before performing the treatment.
- Do not directly touch the heating elements or the aluminium container.
- Do not heat an empty aluminium container.
- Connect the device to a network that complies with the device specifications.
- Do not unplug the unit from the mains when the unit is switched on and running.
- Stop using the appliance if you notice any unusual movement or noise.
- Store and use in dry areas. Keep away from water, moisture. Otherwise the device may stop working and/or be damaged.
- When you have finished using the device, unplug it from the mains supply.
- If the supply cable is damaged, it must be replaced with a new one of the same type at a service centre..
- It is forbidden to make any unauthorised changes to the device..
- Opening the device is categorically prohibited and will void the warranty and the manufacturer assumes no responsibility for the correct and safe operation of the device.
- Keep away from children.

IMPORTER

GNBLab

Michał Szewczyk, Piotr Malinowski s.c.

Piotrkowska 270

90-361 Łódź, Polska

gnb-lab.com

Made in P.R.C.